

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006490

Location: T106

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 260 psi Regulator: 250 psi

TIOGA POLICE DEPT.

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR3377 Lot #: 201610A Exp. Date: 10.4.18
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR5191 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: ICS#5 Exp. Date: 5.15.19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 03417080A3 Cyl #: 8 Exp. Date: 3.5.19

Test 1	<u>0.082</u>	Test 4	<u>0.081</u>	Test 7	<u>0.082</u>
Test 2	<u>0.081</u>	Test 5	<u>0.082</u>	Test 8	<u>0.082</u>
Test 3	<u>0.081</u>	Test 6	<u>0.082</u>	Test 9	<u>0.082</u>
Average	<u>0.082</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ehr
Breath Analyst Signature

5.8.2018
Date

Roberta Hueque-Tummit
Reviewed by

14 May 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006490
Location = TIOG 8164.14.00 09/16
05/08/2018 09:47

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:48
02 Std. Sol.	0.018	09:49
03 Room Air	0.000	09:49
04 Std. Sol.	0.017	09:50
05 Room Air	0.000	09:50
06 Std. Sol.	0.018	09:51
07 Room Air	0.000	09:52


08 Sim Temp = 34.0°C

Simul Ser No = DR3377

Std Sol No = 201610A

County = 53

Oper No. = 666666

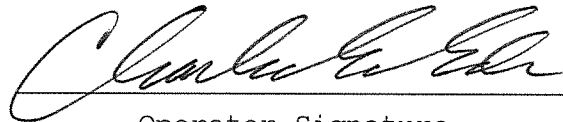

Operator Signature

CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006490
Location = TIOG 8164.14.00 09/16
05/08/2018 09:53

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:54
02 Std. Sol.	0.310	09:55
03 Room Air	0.000	09:55
04 Std. Sol.	0.310	09:56
05 Room Air	0.000	09:57
06 Std. Sol.	0.311	09:57
07 Room Air	0.000	09:58

08 Sim Temp = 34.0°C

Simul Ser No = DR5191
Std Sol No = 17350
County = 53 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006490
Location = TIOG 8164.14.00 09/16
05/08/2018 09:34

Test	AC	Time
01 Room Air	0.000	09:35
02 *Subject Test	INT*	09:36
03 Room Air	0.000	09:37

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 03/01/1982
Sub Sex = Male Weight = 120
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821449
Lot No = 03417080A3
Cyl No = 8
Expiration Date = 03/05/2019
County = 53 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferant Check
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006490
Location = TIOG 8164.14.00 09/16
05/08/2018 09:32

Test	AC	Time
01 Diagnostic	OK	09:33
02 Room Air	0.000	09:33
03 *Subject Test	RFI*	09:34
04 Room Air	0.000	09:34

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 03/01/1982
Sub Sex = Male Weight = 120
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821449
Lot No = 03417080A3
Cyl No = 8
Expiration Date = 03/05/2019
County = 53 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006490
Location = TIOG 8164.14.00 09/16
05/08/2018 09:38

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:38
02 Std. Gas	0.082	09:38
03 Room Air	0.000	09:39
04 Std. Gas	0.081	09:39
05 Room Air	0.000	09:40
06 Std. Gas	0.081	09:40
07 Room Air	0.000	09:41

Lot No = 03417080A3
Cyl No = 8
Exp Date = 03/05/2019
County = 53 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006490
Location = TIOG 8164.14.00 09/16
05/08/2018 09:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:42
02 Std. Gas	0.081	09:42
03 Room Air	0.000	09:43
04 Std. Gas	0.082	09:43
05 Room Air	0.000	09:44
06 Std. Gas	0.082	09:44
07 Room Air	0.000	09:45

Lot No = 03417080A3
Cyl No = 8
Exp Date = 03/05/2019
County = 53 Oper No. = 666666

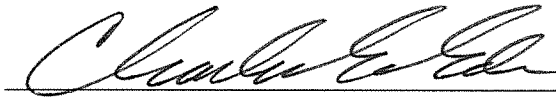

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006490
Location = TIOG 8164.14.00 09/16
05/08/2018 10:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:01
02 Std. Gas	0.082	10:01
03 Room Air	0.000	10:02
04 Std. Gas	0.082	10:02
05 Room Air	0.000	10:03
06 Std. Gas	0.082	10:03
07 Room Air	0.000	10:04

Lot No = 03417080A3
Cyl No = 8
Exp Date = 03/05/2019
County = 53 Oper No. = 666666

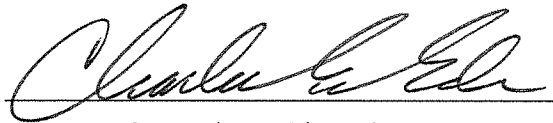

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080AC