

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006499

Location: JAME
JAMESTOWN CORRECTIONAL
CENTER

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 809 psi Regulator: 800 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR3377 Lot #: 201610A Exp. Date: 10.4.18
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR5191 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: ICS #5 Exp. Date: 5.15.19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 03417080A3 Cyl #: 4 Exp. Date: 3/5/19

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.082</u>
Test 3 <u>0.082</u>	Test 6 <u>0.081</u>	Test 9 <u>0.082</u>
Average <u>0.081</u>		

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. De
Breath Analyst Signature

5-18-2018
Date

Robert M. McGuire-Timmer
Reviewed by

24 May 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006499
Location = JAME 8164.14.00 09/16
05/18/2018 15:43

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 03417080A3
Standard Cyl #? 4
Standard Expiration? 03/05/2019
Oper No? 666666

Flow Cal. Date: 08/18/2015
Slope 678
Intercept -685002

IR Calibration Date: 08/18/2015

	3um	9um
0th Coef(*100):	-22123	-16014
1st Coef(*100):	264933	138505
2nd Coef(*100):	3281	1621
H2O adj(mg/l*10k):	704	456

***** Printer Test End *****

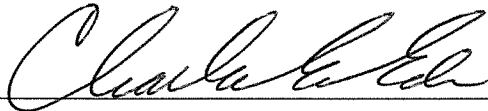

Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006499
Location = JAME 8164.14.00 09/16
05/18/2018 16:00

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:01
02 Std. Sol.	0.019	16:01
03 Room Air	0.000	16:02
04 Std. Sol.	0.018	16:03
05 Room Air	0.000	16:03
06 Std. Sol.	0.019	16:04
07 Room Air	0.000	16:05

08 Sim Temp = 34.0°C

Simul Ser No = DR3377
Std Sol No = 201610A
County = 47 Oper No. = 666666

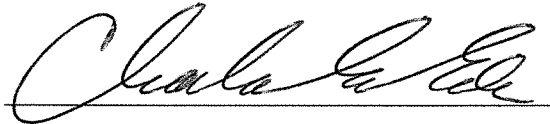

Operator Signature
CHARLES EDER

Remarks: *Low AC*
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006499
Location = JAME 8164.14.00 09/16
05/18/2018 16:06

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:06
02 Std. Sol.	0.302	16:07
03 Room Air	0.000	16:08
04 Std. Sol.	0.303	16:09
05 Room Air	0.000	16:09
06 Std. Sol.	0.303	16:10
07 Room Air	0.000	16:11

08 Sim Temp = 34.0°C

Simul Ser No = DR5191
Std Sol No = 17350
County = 47 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
 0.300 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006499
Location = JAME 8164.14.00 09/16
05/18/2018 15:45

Test	AC	Time
01 Room Air	0.000	15:46
02 *Subject Test	INT*	15:47
03 Room Air	0.000	15:48

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 4
Expiration Date = 03/05/2019
County = 47 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006499
Location = JAME 8164.14.00 09/16
05/18/2018 15:43

Test	AC	Time
01 Diagnostic	OK	15:44
02 Room Air	0.000	15:44
03 *Subject Test	RFI*	15:45
04 Room Air	0.000	15:45

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 4
Expiration Date = 03/05/2019
County = 47 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

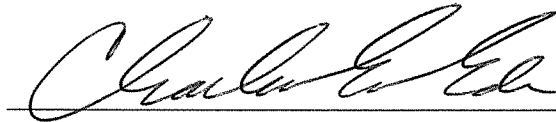

Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006499
Location = JAME 8164.14.00 09/16
05/18/2018 15:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:48
02 Std. Gas	0.081	15:49
03 Room Air	0.000	15:49
04 Std. Gas	0.081	15:50
05 Room Air	0.000	15:50
06 Std. Gas	0.082	15:51
07 Room Air	0.000	15:51

Lot No = 03417080A3
Cyl No = 4
Exp Date = 03/05/2019
County = 47 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006499
Location = JAME 8164.14.00 09/16
05/18/2018 15:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:52
02 Std. Gas	0.081	15:52
03 Room Air	0.000	15:53
04 Std. Gas	0.081	15:53
05 Room Air	0.000	15:54
06 Std. Gas	0.081	15:54
07 Room Air	0.000	15:55

Lot No = 03417080A3
Cyl No = 4
Exp Date = 03/05/2019
County = 47 Oper No. = 666666

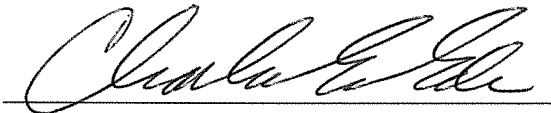

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006499
Location = JAME 8164.14.00 09/16
05/18/2018 16:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:12
02 Std. Gas	0.081	16:13
03 Room Air	0.000	16:13
04 Std. Gas	0.082	16:14
05 Room Air	0.000	16:14
06 Std. Gas	0.082	16:15
07 Room Air	0.000	16:15

Lot No = 03417080A3
Cyl No = 4
Exp Date = 03/05/2019
County = 47 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000