

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006502

Location: STUT

JAMESTOWN L.E.C.

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 1006 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR3377 Lot #: 201610A Exp. Date: 10-4-18
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR5191 Lot #: 17350 Exp. Date: 10-11-19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: ^{CEE 0.10} ~~0.10~~ AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: ICS #5 Exp. Date: 5.15.19

Display reads "Interferent Detect"

3. RFI Check (CMS Mode) Display reads "RFI Detect"

4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 0341708043 Cyl #: 3 Exp. Date: 3/5/19

Test 1	<u>0.081</u>	Test 4	<u>0.082</u>	Test 7	<u>0.082</u>
Test 2	<u>0.082</u>	Test 5	<u>0.082</u>	Test 8	<u>0.082</u>
Test 3	<u>0.082</u>	Test 6	<u>0.081</u>	Test 9	<u>0.082</u>
Average <u>0.082</u>					

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Egan
Breath Analyst Signature

5-18-2018
Date

Roberta Hueque-Nemmo
Reviewed by

24 May 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = STUT 8164.14.00 09/16
05/18/2018 14:45

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 03417080A3
Standard Cyl #? 3
Standard Expiration? 03/05/2019
Oper No? 666666

Flow Cal. Date: 08/19/2015
Slope 674
Intercept -654491

IR Calibration Date: 08/19/2015
 3um 9um

0th Coef(*100): -24446 -18103
1st Coef(*100): 267787 135417
2nd Coef(*100): 3387 1610
H2O adj(mg/l*10k): 740 513

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = STUT 8164.14.00 09/16
05/18/2018 15:03

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:03
02 Std. Sol.	0.017	15:04
03 Room Air	0.000	15:05
04 Std. Sol.	0.017	15:05
05 Room Air	0.000	15:06
06 Std. Sol.	0.017	15:06
07 Room Air	0.000	15:07

08 Sim Temp = 34.0°C

Simul Ser No = DR3377
Std Sol No = 201610A
County = 47 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = STUT 8164.14.00 09/16
05/18/2018 15:16

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:16
02 Std. Sol.	0.303	15:17
03 Room Air	0.000	15:18
04 Std. Sol.	0.303	15:18
05 Room Air	0.000	15:19
06 Std. Sol.	0.304	15:20
07 Room Air	0.000	15:20

08 Sim Temp = 34.0°C

Simul Ser No = DR5191
Std Sol No = 17350
County = 47 Oper No. = 666666

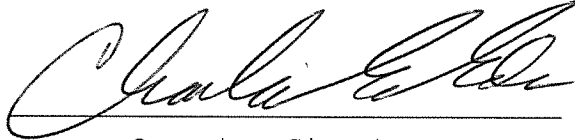

Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = STUT 8164.14.00 09/16
05/18/2018 14:53

Test	AC	Time
01 Room Air	0.000	14:53
02 *Subject Test	INT*	14:54
03 Room Air	0.000	14:54

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 3
Expiration Date = 03/05/2019
County = 47 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = STUT 8164.14.00 09/16
05/18/2018 14:50

Test	AC	Time
01 Diagnostic	OK	14:51
02 Room Air	0.000	14:52
03 *Subject Test	RFI*	14:52
04 Room Air	0.000	14:52

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 3
Expiration Date = 03/05/2019
County = 47 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = STUT 8164.14.00 09/16
05/18/2018 14:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:55
02 Std. Gas	0.081	14:56
03 Room Air	0.000	14:56
04 Std. Gas	0.082	14:56
05 Room Air	0.000	14:57
06 Std. Gas	0.082	14:57
07 Room Air	0.000	14:58

Lot No = 03417080A3

Cyl No = 3

Exp Date = 03/05/2019

County = 47

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK

0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = STUT 8164.14.00 09/16
05/18/2018 14:58

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:59
02 Std. Gas	0.082	14:59
03 Room Air	0.000	14:59
04 Std. Gas	0.082	15:00
05 Room Air	0.000	15:00
06 Std. Gas	0.081	15:01
07 Room Air	0.000	15:01

Lot No = 03417080A3
Cyl No = 3
Exp Date = 03/05/2019
County = 47 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = STUT 8164.14.00 09/16
05/18/2018 15:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:22
02 Std. Gas	0.082	15:22
03 Room Air	0.000	15:23
04 Std. Gas	0.082	15:23
05 Room Air	0.000	15:23
06 Std. Gas	0.082	15:24
07 Room Air	0.000	15:24

Lot No = 03417080A3

Cyl No = 3

Exp Date = 03/05/2019

County = 47

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080AC

Form 106-I8000