

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005947

Southwest Multi-County ^{CC} TEE READ
Location: DKPD

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 985 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR3377 ^{5143 RW} Lot #: 201610A Exp. Date: 4 Oct 18
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR5191 Lot #: 17350 Exp. Date: 11 Oct 19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: 1CSS Exp. Date: 15 May 19

Display reads "Interferent Detect"

3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC

Gas Cyl Lot #: 19817080A4 Cyl #: 10 Exp. Date: 5 Sep 19

Test 1	<u>0.078</u>	Test 4	<u>0.077</u>	Test 7	<u>0.077</u>
Test 2	<u>0.077</u>	Test 5	<u>0.077</u>	Test 8	<u>0.077</u>
Test 3	<u>0.077</u>	Test 6	<u>0.077</u>	Test 9	<u>0.077</u>
Average	<u>0.077</u>				

C. Remarks/Maintenance: Annual inspection. wrapped breath hose with electrical tape to protect it. Replaced elbow joint on calibration inlet.

Instrument is acceptable to be used in the field.

Roberta Huego-Timmo
Breath Analyst Signature

31 May 18
Date

Charles E. Ehr
Reviewed by

31 MAY 2018
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005947
Location = DKPD 8164.14.00 09/16
05/31/2018 10:03

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19817080A4
Standard Cyl #? 10
Standard Expiration? 09/05/2019
Oper No? 888888

Flow Cal. Date: 06/21/2013
Slope 693
Intercept -735360

IR Calibration Date: 04/06/2015
 3um 9um

0th Coef(*100): -28096 -26982
1st Coef(*100): 280034 136042
2nd Coef(*100): 3530 1567
H2O adj(mg/l*10k): 510 437

***** Printer Test End *****

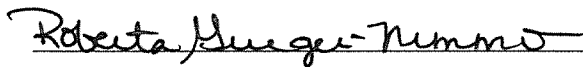

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005947
Location = DKPD 8164.14.00 09/16
05/31/2018 12:29

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:30
02 Std. Sol.	0.018	12:31
03 Room Air	0.000	12:31
04 Std. Sol.	0.019	12:32
05 Room Air	0.000	12:32
06 Std. Sol.	0.018	12:33
07 Room Air	0.000	12:34


08 Sim Temp = 34.0°C

Simul Ser No = DR5143

Std Sol No = 201610A

County = 45

Oper No. = 888888

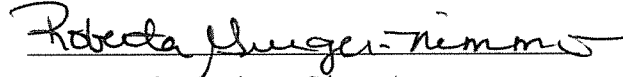

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks:

~~Calibration~~ RBN
Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005947
Location = DKPD 8164.14.00 09/16
05/31/2018 10:58

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:59
02 Std. Sol.	0.300	11:00
03 Room Air	0.000	11:00
04 Std. Sol.	0.299	11:01
05 Room Air	0.000	11:01
06 Std. Sol.	0.298	11:02
07 Room Air	0.000	11:03

08 Sim Temp = 34.0°C

Simul Ser No = DR5191
Std Sol No = 17350
County = 45 Oper No. = 888888

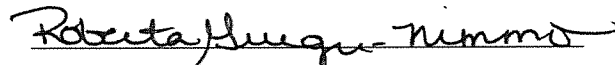
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005947
Location = DKPD 8164.14.00 09/16
05/31/2018 10:15

Test	AC	Time
01 Room Air	0.000	10:17
02 *Subject Test	INT*	10:17
03 Room Air	0.000	10:18

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 10
Expiration Date = 09/05/2019
County = 45 Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check
0.10 AC + 0.05% Acetone
0.05% Acetone
Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005947
Location = DKPD 8164.14.00 09/16
05/31/2018 10:19

Test	AC	Time
01 Diagnostic	OK	10:20
02 Room Air	RFI*	10:20
03 Room Air	0.000	10:21

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 10
Expiration Date = 09/05/2019
County = 45 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

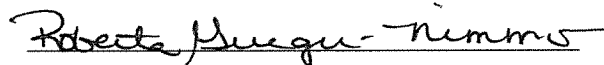

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *RFI check*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005947
Location = DKPD 8164.14.00 09/16
05/31/2018 10:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:04
02 Std. Gas	0.078	10:04
03 Room Air	0.000	10:05
04 Std. Gas	0.077	10:05
05 Room Air	0.000	10:06
06 Std. Gas	0.077	10:06
07 Room Air	0.000	10:07

Lot No = 19817080A4
Cyl No = 10
Exp Date = 09/05/2019
County = 45 Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005947
Location = DKPD 8164.14.00 09/16
05/31/2018 10:07

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:08
02 Std. Gas	0.077	10:08
03 Room Air	0.000	10:09
04 Std. Gas	0.077	10:09
05 Room Air	0.000	10:09
06 Std. Gas	0.077	10:10
07 Room Air	0.000	10:10

Lot No = 19817080A4
Cyl No = 10
Exp Date = 09/05/2019
County = 45 Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check

0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005947
Location = DKPD 8164.14.00 09/16
05/31/2018 10:10

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:11
02 Std. Gas	0.077	10:11
03 Room Air	0.000	10:12
04 Std. Gas	0.077	10:12
05 Room Air	0.000	10:13
06 Std. Gas	0.077	10:13
07 Room Air	0.000	10:14

Lot No = 19817080A4
Cyl No = 10
Exp Date = 09/05/2019
County = 45 Oper No. = 888888

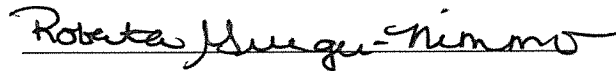

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Calibration Check*
0.080 AC

Form 106-I8000