

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006492

Location: FYPD
Fort Yates LEC

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 670 psi Regulator: 675 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR3377 Lot #: 201610A Exp. Date: 4 Oct 18
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR5191 Lot #: 17350 Exp. Date: 11 Oct 19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: 1CS5 Exp. Date: 15 May 19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 17316080A2 Cyl #: 29 Exp. Date: 5 Aug 18

Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.080</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.080</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

C. Remarks/Maintenance: Annual inspection. Replaced O-ring on breath hose.

Instrument is acceptable to be used in the field.

Roberta Hueque Nimmo
Breath Analyst Signature

30 May 18
Date

Charles E. Ed
Reviewed by

31 MAY 2018
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006492
Location = FYPD 8164.14.00 09/16
05/30/2018 09:07

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 17316080A2
Standard Cyl #? 29
Standard Expiration? 08/05/2018
Oper No? 888888

Flow Cal. Date: 08/18/2015
Slope 669
Intercept -631515

IR Calibration Date: 08/18/2015
 3um 9um

0th Coef(*100): -41685 -23145
1st Coef(*100): 269226 142194
2nd Coef(*100): 2671 1239
H2O adj(mg/l*10k): 872 437

***** Printer Test End *****

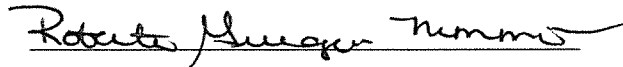

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006492
Location = FYPD 8164.14.00 09/16
05/30/2018 09:35

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:36
02 Std. Sol.	0.016	09:37
03 Room Air	0.000	09:37
04 Std. Sol.	0.017	09:38
05 Room Air	0.000	09:39
06 Std. Sol.	0.017	09:39
07 Room Air	0.000	09:40


08 Sim Temp = 34.0°C

Simul Ser No = DR3377

Std Sol No = 201610A

County = 61

Oper No. = 888888

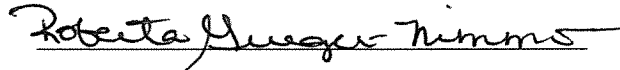

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks:

LOW AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006492
Location = FYPD 8164.14.00 09/16
05/30/2018 09:28

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:29
02 Std. Sol.	0.298	09:30
03 Room Air	0.000	09:30
04 Std. Sol.	0.297	09:31
05 Room Air	0.000	09:32
06 Std. Sol.	0.297	09:32
07 Room Air	0.000	09:33


08 Sim Temp = 34.0°C

Simul Ser No = DR5191

Std Sol No = 17350

County = 61

Oper No. = 888888

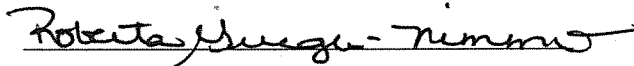

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC
0.300 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006492
Location = FYPD 8164.14.00 09/16
05/30/2018 09:24

Test	AC	Time
01 Room Air	0.000	09:24
02 *Subject Test	INT*	09:25
03 Room Air	0.000	09:26

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 29
Expiration Date = 08/05/2018
County = 61 Oper No. = 888888

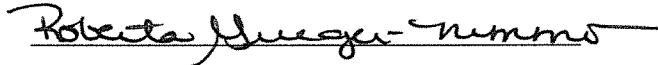

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Interferent Check*
0.10 AC + 0.05% Acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006492
Location = FYPD 8164.14.00 09/16
05/30/2018 09:21

Test	AC	Time
01 Diagnostic	OK	09:22
02 Room Air	RFI*	09:22
03 Room Air	0.000	09:23

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 17316080A2
Cyl No = 29
Expiration Date = 08/05/2018
County = 61 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

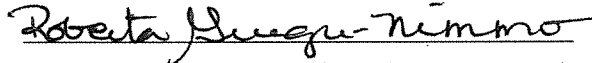

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks:

RFI Check

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006492
Location = FYPD 8164.14.00 09/16
05/30/2018 09:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:09
02 Std. Gas	0.080	09:09
03 Room Air	0.000	09:10
04 Std. Gas	0.080	09:10
05 Room Air	0.000	09:11
06 Std. Gas	0.080	09:11
07 Room Air	0.000	09:12

Lot No = 17316080A2
Cyl No = 29
Exp Date = 08/05/2018
County = 61 Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Calibration Check*
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006492
Location = FYPD 8164.14.00 09/16
05/30/2018 09:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:14
02 Std. Gas	0.080	09:14
03 Room Air	0.000	09:15
04 Std. Gas	0.080	09:15
05 Room Air	0.000	09:16
06 Std. Gas	0.080	09:16
07 Room Air	0.000	09:17

Lot No = 17316080A2
Cyl No = 29
Exp Date = 08/05/2018
County = 61 Oper No. = 888888

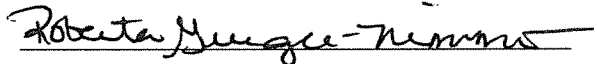
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006492
Location = FYPD 8164.14.00 09/16
05/30/2018 09:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:18
02 Std. Gas	0.080	09:18
03 Room Air	0.000	09:19
04 Std. Gas	0.080	09:19
05 Room Air	0.000	09:19
06 Std. Gas	0.080	09:20
07 Room Air	0.000	09:20

Lot No = 17316080A2
Cyl No = 29
Exp Date = 08/05/2018
County = 61 Oper No. = 888888

Roberta Grieger Nimmo
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Calibration Check*
0.080 AC