

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004943

Location: Minot Air Force Base
MNAF

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 993 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR5143 Lot #: 201610A Exp. Date: 4 Oct 18
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR5191 Lot #: 17350 Exp. Date: 11 Oct 19
 - Configure dry gas standard for the following tests (Level 1,S).
 2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: 1055 Exp. Date: 15 May 19
 Display reads "Interferent Detect"
 3. RFI Check (CMS Mode) Display reads "RFI Detect"
 4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 19817080A4 Cyl #: 8 Exp. Date: 5 Sep 19

Test 1	<u>0.080</u>	Test 4	<u>0.080</u>	Test 7	<u>0.080</u>
Test 2	<u>0.079</u>	Test 5	<u>0.081</u>	Test 8	<u>0.080</u>
Test 3	<u>0.079</u>	Test 6	<u>0.080</u>	Test 9	<u>0.080</u>
Average	<u>0.080</u>				

C. Remarks/Maintenance: Annual inspection.

Instrument is acceptable to be used in the field.

Roberta Buege-Nimmo
Breath Analyst Signature

4 Jun 18
Date

Charles E. Eder
Reviewed by

04 JUN. 2018
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = MNAF 8164.14.00 09/16
06/04/2018 11:51

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? No
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19817080A4
Standard Cyl #? 08
Standard Expiration? 09/05/2019
Oper No? 888888

Flow Cal. Date: 01/18/2010
Slope 693
Intercept -665387

IR Calibration Date: 05/21/2015
 3um 9um

0th Coef(*100): -28195 -26668
1st Coef(*100): 268915 137995
2nd Coef(*100): 103 992
H2O adj(mg/l*10k): 222 308

***** Printer Test End *****

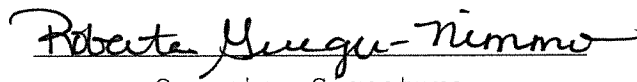

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = MNAF 8164.14.00 09/16
06/04/2018 12:19

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:20
02 Std. Sol.	0.021	12:20
03 Room Air	0.000	12:21
04 Std. Sol.	0.020	12:22
05 Room Air	0.000	12:22
06 Std. Sol.	0.020	12:23
07 Room Air	0.000	12:23


08 Sim Temp = 34.0°C

Simul Ser No = DR5143

Std Sol No = 201610A

County = 64

Oper No. = 888888

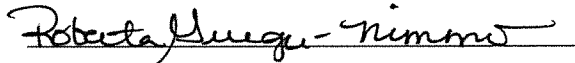

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: low AC
0.020 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = MNAF 8164.14.00 09/16
06/04/2018 12:13

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:14
02 Std. Sol.	0.300	12:14
03 Room Air	0.000	12:15
04 Std. Sol.	0.300	12:16
05 Room Air	0.000	12:16
06 Std. Sol.	0.299	12:17
07 Room Air	0.000	12:17

08 Sim Temp = 34.0°C

Simul Ser No = DR5191

Std Sol No = 17350

County = 64

Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: High AC

0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = MNAF 8164.14.00 09/16
06/04/2018 12:09

Test	AC	Time
01 Room Air	0.000	12:10
02 *Subject Test	INT*	12:10
03 Room Air	0.000	12:11

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 08
Expiration Date = 09/05/2019
County = 64 Oper No. = 888888

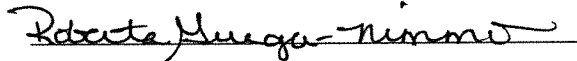

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Interferent Check*
0.10 AC + 0.05% Acetone

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = MNAF 8164.14.00 09/16
06/04/2018 11:53

Test	AC	Time
01 Diagnostic	OK	11:54
02 Room Air	0.000	11:54
03 *Subject Test	RFI*	11:55
04 Room Air	0.000	11:55

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 08
Expiration Date = 09/05/2019
County = 64 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

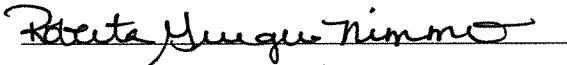
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = MNAF 8164.14.00 09/16
06/04/2018 11:58

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:58
02 Std. Gas	0.080	11:59
03 Room Air	0.000	11:59
04 Std. Gas	0.079	12:00
05 Room Air	0.000	12:00
06 Std. Gas	0.079	12:00
07 Room Air	0.000	12:01

Lot No = 19817080A4

Cyl No = 08

Exp Date = 09/05/2019

County = 64

Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = MNAF 8164.14.00 09/16
06/04/2018 12:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:02
02 Std. Gas	0.080	12:02
03 Room Air	0.000	12:03
04 Std. Gas	0.081	12:03
05 Room Air	0.000	12:04
06 Std. Gas	0.080	12:04
07 Room Air	0.000	12:05

Lot No = 19817080A4
Cyl No = 08
Exp Date = 09/05/2019
County = 64 Oper No. = 888888

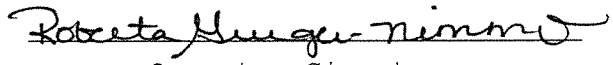
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004943
Location = MNAF 8164.14.00 09/16
06/04/2018 12:05

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:06
02 Std. Gas	0.080	12:06
03 Room Air	0.000	12:07
04 Std. Gas	0.080	12:07
05 Room Air	0.000	12:08
06 Std. Gas	0.080	12:08
07 Room Air	0.000	12:08

Lot No = 19817080A4
Cyl No = 08
Exp Date = 09/05/2019
County = 64 Oper No. = 888888

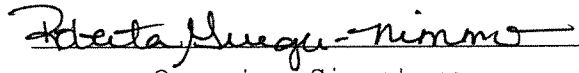

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC