

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005953

Dickey County S.O.
Location: DCSO

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 1000 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR5143 Lot #: 201610A Exp. Date: 4 Oct 18
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR5191 Lot #: 17350 Exp. Date: 11 Oct 19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: 1C55 Exp. Date: 15 May 19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 34917080A3 Cyl #: 13 Exp. Date: 5 Feb 20

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.082</u>	Test 6 <u>0.080</u>	Test 9 <u>0.081</u>
Average <u>0.081</u>		

C. Remarks/Maintenance: Annual inspection.

Instrument is acceptable to be used in the field.

Robert P. McGuinnis
Breath Analyst Signature

6 Jun 18
Date

Charles E. Edr
Reviewed by

08 JUN. 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005953
Location = DCSO 8164.14.00 09/16
06/06/2018 09:21

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

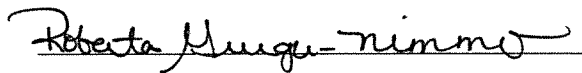
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080 *80A3*
Standard Lot #? 349170 *00A3 RLN*
Standard Cyl #? 013
Standard Expiration? 02/05/2020
Oper No? 888888

Flow Cal. Date: 06/16/2015
Slope 678
Intercept -668987

IR Calibration Date: 04/30/2015
 3um 9um

0th Coef(*100): -13435 -19950
1st Coef(*100): 257165 129445
2nd Coef(*100): 3995 1631
H2O adj(mg/l*10k): 505 548

***** Printer Test End *****


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

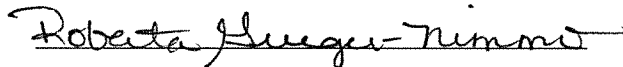
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005953
Location = DCSO 8164.14.00 09/16
06/06/2018 09:59

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:00
02 Std. Sol.	0.020	10:01
03 Room Air	0.000	10:01
04 Std. Sol.	0.020	10:02
05 Room Air	0.000	10:03
06 Std. Sol.	0.020	10:03
07 Room Air	0.000	10:04
08 Sim Temp = 34.0°C		

Simul Ser No = DR5143
Std Sol No = 201610A
County = 11 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Low AC*
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005953
Location = DCSO 8164.14.00 09/16
06/06/2018 09:52

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:53
02 Std. Sol.	0.298	09:54
03 Room Air	0.000	09:54
04 Std. Sol.	0.297	09:55
05 Room Air	0.000	09:55
06 Std. Sol.	0.297	09:56
07 Room Air	0.000	09:57

08 Sim Temp = 34.0°C

Simul Ser No = DR5191

Std Sol No = 17350

County = 11

Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks:

RW
High AC
0.300 AC

Form 106-I8000

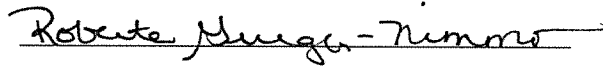
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005953
Location = DCSO 8164.14.00 09/16
06/06/2018 09:49

Test	AC	Time
01 Room Air	0.000	09:50
02 *Subject Test	INT*	09:50
03 Room Air	0.000	09:51

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 34917080A3
Cyl No = 013
Expiration Date = 02/05/2020
County = 11 Oper No. = 888888



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Interferent Check*
0.10 AC + 0.059% Acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005953
Location = DCSO 8164.14.00 09/16
06/06/2018 09:44

Test	AC	Time
01 Diagnostic	OK	09:45
02 Room Air	0.000	09:45
03 *Subject Test	RFI*	09:46
04 Room Air	0.000	09:47

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 34917080A3
Cyl No = 013
Expiration Date = 02/05/2020
County = 11 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *RFI Check*

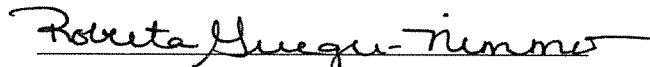
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005953
Location = DCSO 8164.14.00 09/16
06/06/2018 09:24

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:25
02 Std. Gas	0.081	09:25
03 Room Air	0.000	09:26
04 Std. Gas	0.081	09:26
05 Room Air	0.000	09:26
06 Std. Gas	0.082	09:27
07 Room Air	0.000	09:27

Lot No = 3491700^{80A3}~~00A3~~ *REN*
Cyl No = 013
Exp Date = 02/05/2020
County = 11 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Calibration Check*
0.080 AC

Form 106-I8000

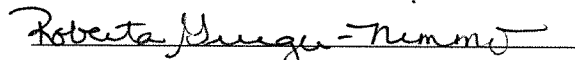
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005953
Location = DCSO 8164.14.00 09/16
06/06/2018 09:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:38
02 Std. Gas	0.081	09:38
03 Room Air	0.000	09:39
04 Std. Gas	0.081	09:39
05 Room Air	0.000	09:39
06 Std. Gas	0.080	09:40
07 Room Air	0.000	09:40

Lot No = 34917080A3
Cyl No = 013
Exp Date = 02/05/2020
County = 11 Oper No. = 888888


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC
Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005953
Location = DCSO 8164.14.00 09/16
06/06/2018 09:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:41
02 Std. Gas	0.081	09:42
03 Room Air	0.000	09:42
04 Std. Gas	0.081	09:42
05 Room Air	0.000	09:43
06 Std. Gas	0.081	09:43
07 Room Air	0.000	09:44

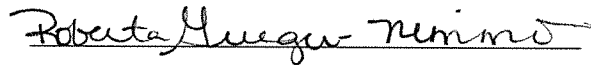
Lot No = 34917080A3

Cyl No = 013

Exp Date = 02/05/2020

County = 11

Oper No. = 888888



Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check

0.080 AC

Form 106-I8000