

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006497

Grand Forks Air Force
Location: GFAF Base

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 991 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR5143 Lot #: 201610A Exp. Date: 4 Oct 18
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR5191 Lot #: 17350 Exp. Date: 11 Oct 19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: 1055 Exp. Date: 15 May 19
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 03417080A3 Cyl #: 11 Exp. Date: 5 Mar 19

Test 1	<u>0.080</u>	Test 4	<u>0.080</u>	Test 7	<u>0.080</u>
Test 2	<u>0.081</u>	Test 5	<u>0.080</u>	Test 8	<u>0.080</u>
Test 3	<u>0.080</u>	Test 6	<u>0.080</u>	Test 9	<u>0.081</u>
Average <u>0.080</u>					

C. Remarks/Maintenance: Annual inspection.

Instrument is acceptable to be used in the field.

Robert H. Huggins - Nimmo
Breath Analyst Signature

11 Jun 18
Date

Charles E. Edin
Reviewed by

12 JUN. 18
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = GFAF 8164.14.00 09/16
06/11/2018 10:45

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

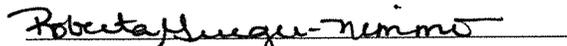
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 03417080A3
Standard Cyl #? 11
Standard Expiration? 03/05/2019
Oper No? 888888

Flow Cal. Date: 08/19/2015
Slope 657
Intercept -642041

IR Calibration Date: 08/19/2015

	3um	9um
0th Coef(*100):	-16255	-17993
1st Coef(*100):	263339	136695
2nd Coef(*100):	5198	1666
H2O adj(mg/l*10k):	766	522

***** Printer Test End *****


Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *Print Test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = GFAF 8164.14.00 09/16
06/11/2018 11:25

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:26
02 Std. Sol.	0.020	11:27
03 Room Air	0.000	11:27
04 Std. Sol.	0.020	11:28
05 Room Air	0.000	11:28
06 Std. Sol.	0.020	11:29
07 Room Air	0.000	11:30

08 Sim Temp = 34.0°C

Simul Ser No = DR5143

Std Sol No = 201610A

County = 65

Oper No. = 888888

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = GFAF 8164.14.00 09/16
06/11/2018 11:12

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:12
02 Std. Sol.	0.296	11:13
03 Room Air	0.000	11:14
04 Std. Sol.	0.296	11:14
05 Room Air	0.000	11:15
06 Std. Sol.	0.296	11:16
07 Room Air	0.000	11:16

08 Sim Temp = 34.0°C

Simul Ser No = DR5191
Std Sol No = 17350
County = 65 Oper No. = 888888

Roberta Grieger-Nimmo
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *High AC*
0.300 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = GFAF 8164.14.00 09/16
06/11/2018 11:03

Test	AC	Time
01 Room Air	0.000	11:03
02 *Subject Test	INT*	11:04
03 Room Air	0.000	11:05

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 11
Expiration Date = 03/05/2019
County = 65 Oper No. = 888888

Roberta Grieger Nimmma

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Interferent Check
0.10 AC + 0.05% Acetone

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = GFAF 8164.14.00 09/16
06/11/2018 10:59

Test	AC	Time
01 Diagnostic	OK	11:00
02 Room Air	0.000	11:01
03 *Subject Test	RFI*	11:01
04 Room Air	0.000	11:02

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 03417080A3
Cyl No = 11
Expiration Date = 03/05/2019
County = 65 Oper No. = 888888

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Roberta Grieger-Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: RFI Check

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = GFAF 8164.14.00 09/16
06/11/2018 10:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:47
02 Std. Gas	0.080	10:48
03 Room Air	0.000	10:48
04 Std. Gas	0.081	10:49
05 Room Air	0.000	10:49
06 Std. Gas	0.080	10:50
07 Room Air	0.000	10:50

Lot No = 03417080A3
Cyl No = 11
Exp Date = 03/05/2019
County = 65 Oper No. = 888888

Roberta Grieger-Nimmo
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = GFAF 8164.14.00 09/16
06/11/2018 10:50

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:51
02 Std. Gas	0.080	10:51
03 Room Air	0.000	10:52
04 Std. Gas	0.080	10:52
05 Room Air	0.000	10:53
06 Std. Gas	0.080	10:53
07 Room Air	0.000	10:54

Lot No = 03417080A3

Cyl No = 11

Exp Date = 03/05/2019

County = 65

Oper No. = 888888

Roberta Grieger Nimmo

Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: Calibration Check
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006497
Location = GFAF 8164.14.00 09/16
06/11/2018 10:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:56
02 Std. Gas	0.080	10:57
03 Room Air	0.000	10:57
04 Std. Gas	0.080	10:58
05 Room Air	0.000	10:58
06 Std. Gas	0.081	10:59
07 Room Air	0.000	10:59

Lot No = 03417080A3
Cyl No = 11
Exp Date = 03/05/2019
County = 65 Oper No. = 888888

Roberta Grieger-Nimmo
Operator Signature
ROBERTA GRIEGER-NIMMO

Remarks: *calibration check*
0.080 AC