



# INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59281 (10-2016)

Serial Number <i>80-006493</i>	Instrument Location <i>Killdeer PD Killdeer ND</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

### Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): *N/A*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Eric Brauthon</i>	Date <i>6-15-18</i>
---	------------------------

### Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified by <i>Roberta Suerger-Thumms</i>	Certified Date <i>6-20-18</i>
--	----------------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer  
North Dakota Model 8000 SN 80-006493  
Location = KDPD 8164.14.00 09/1E  
06/15/2018 21:55

\*\*\*\*\* Printer Test \*\*\*\*\*

12 char/line  
abcdefghijkl  
ABCDEFGHIJKL

16 char per line  
abcdefghijklmnop  
ABCDEFGHIJKLMN

21 chars per line  
abcdefghijklmnopqrstu  
ABCDEFGHIJKLMNQRSTU

24 characters per line  
abcdefghijklmnopqrstuvw  
ABCDEFGHIJKLMNOPQRSTUW

32 characters per line  
abcdefghijklmnopqrstuvwxy123456  
ABCDEFGHIJKLMNOPQRSTUWXY123456

42 characters per line  
abcdefghijklmnopqrstuvwxy1234567890abcdef  
ABCDEFGHIJKLMNOPQRSTUWXY1234567890ABCDEF

Current Instrument Setup  
Data Entry Mode: Enabled  
Start Test Sequence: 09090909  
Display Prelim Rslt? Yes  
Display Third Digit? Yes  
Inhib Printer(Y/N)? No  
Display Volume? No  
Disable On Memfull? Yes  
# of Print Copies? 1  
Select Std (D/W/I)? Dry  
Standard Value? 0.080  
Standard Lot #? 34917080A3  
Standard Cyl #? 15  
Standard Expiration? 02/05/2020  
Oper No? 100444

Flow Cal. Date: 05/23/2018  
Slope 640  
Intercept -624033

IR Calibration Date: 05/23/2018  
3um 9um

0th Coef(*100):	-19857	-14695
1st Coef(*100):	263752	135087
2nd Coef(*100):	3253	1458
H2O adj(ng/l*10K):	643	483

\*\*\*\*\* Test End \*\*\*\*\*

*Eric Braathen*

Operator Signature  
ERIC BRAATHEN

Remarks:

Form 106-18000


Intoxilyzer Test Record and Checklist  
NDDAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006493  
Location = KDPD      8164.14.00 09/16  
06/15/2018      22:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	22:31
02 Std. Gas	0.081	22:31
03 Room Air	0.000	22:32
04 Std. Gas	0.080	22:32
05 Room Air	0.000	22:33
06 Std. Gas	0.080	22:33
07 Room Air	0.000	22:34

Lot No = 34917080A3  
Cyl No = 15  
Exp Date = 02/05/2020  
County = 13      Oper No. = 100444



Operator Signature  
ERIC BRAATHEN

Remarks:

Form 106-18000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006493  
Location = KDPD      8164.14.00 09/16  
06/15/2018      22:26

Test	AC	Time
01 Room Air	RFI*	22:29
02 Room Air	RFI*	22:30

\*Invalid Test  
Inhibited - RFI

Sub Name = RFI TEST, INSTLL TEST  
Sub DOB = 09/09/1976  
Sub Sex = Male      Weight = 100  
Test = OTH      Cit = 11111  
Dr. Lic. = ND/1  
Lot No = 34917080A3  
Cyl No = 15  
Expiration Date = 02/05/2020  
County = 13      Oper No. = 100444

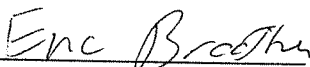

Operator Signature  
ERIC BRAATHEN

Remarks: