



Ethanol Gas Standard Cylinder Report

~~INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT~~ *REN* 10 Aug 18
NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
SFN 59282 (06/2018)

Chemical Test Operator Name (Print) Nick Schoedel		
Location Burleigh Morton Detention Center		Intoxilyzer® Serial Number 80-005952
Gas Lot Number 34917080A3	Gas Cylinder Number 36	Gas Expiration Date 2-5-20

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0.080 AC
 - B. 0.088 AC
 - C. 0.000 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Nick Schoedel</i>	Date 08-08-18
Reviewed By (Crime Laboratory Use Only) <i>Roberta Hueque-Nemms</i>	Date 08-10-18

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005952
Location = BCSO 8164.14.00 09/16
08/08/2018 12:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:19
02 Std. Gas	0.080	12:19
03 Room Air	0.000	12:20
04 Std. Gas	0.080	12:20
05 Room Air	0.000	12:21
06 Std. Gas	0.080	12:21
07 Room Air	0.000	12:22


Lot No = 34917080A3

Cyl No = 36

Exp Date = 02/05/2020

County = 08

Oper No. = 131854

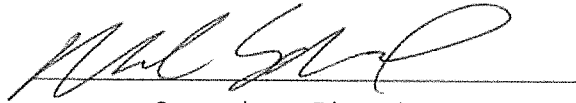

Operator Signature
NICK SCHOEDEL

Remarks:

Form 106-I8000