



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Monroe, Jeremy J</i>		
Location <i>RUGB-C</i>	Intoxilyzer® Serial Number <i>80-005951</i>	
Gas Lot Number <i>13518080A6</i>	Gas Cylinder Number <i>023</i>	Gas Expiration Date <i>08/05/2020</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).

Step 2. Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
was completed
See pg 202.
Jan 19

- A. 0.082 AC
- B. 0.081 AC
- C. 0.081 AC

- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>01/02/2019</i>
Reviewed By (Crime Laboratory Use Only) <i>Roberta Huggins-Numms</i>	Date <i>2 Jan 19</i>


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005951
Location = RUGB 8164.14.00 09/16
01/02/2019 00:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	00:47
02 Std. Gas	0.082	00:47
03 Room Air	0.000	00:48
04 Std. Gas	0.081	00:48
05 Room Air	0.000	00:49
06 Std. Gas	0.081	00:49
07 Room Air	0.000	00:49

Lot No = 13518080A6
Cyl No = 023
Exp Date = 08/05/2020
County = 35 Oper No. = 129922

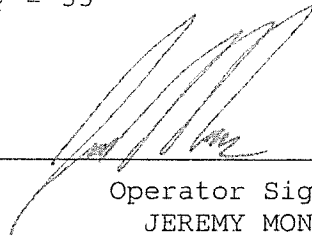

Operator Signature
JEREMY MONROE

Remarks:
Install New Gas Tank
Form 106-I8000