



ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

| | | |
|--|--|--|
| Chemical Test Operator Name (Print) <i>Darcy Aberle</i> | | |
| Location <i>Williams County LEC</i> | Intoxilyzer® Serial Number <i>80-005946</i> | |
| Gas Lot Number <i>13518080A6</i> | Gas Cylinder Number <i>28</i> | Gas Expiration Date <i>08/05/2020</i> |

Check When Done:

1. Scan/Enter Gas Cylinder Information (Level 1, Function S).

2. Perform an ACA Test (Level 1, Function C).

Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).

A. *0.082*AC

B. *0.080*AC

C. *0.080*AC

3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.

4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.

5. Send the Following to the Crime Laboratory:

A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).

B. ACA Test Record.

| | |
|---|---------------------------|
| Chemical Test Operator Signature <i>Darcy Aberle</i> | Date <i>02/19/2019</i> |
| Reviewed By (Crime Laboratory Use Only) <i>Carla E. Eber</i> | Date <i>2.21.2019</i> |


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005946
Location = WILL 8164.14.00 09/16
02/19/2019 14:07

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 14:08 |
| 02 Std. Gas | 0.082 | 14:08 |
| 03 Room Air | 0.000 | 14:09 |
| 04 Std. Gas | 0.080 | 14:09 |
| 05 Room Air | 0.000 | 14:10 |
| 06 Std. Gas | 0.080 | 14:10 |
| 07 Room Air | 0.000 | 14:11 |

Lot No = 13518080A6
Cyl No = 28
Exp Date = 08/05/2020
County = 53 Oper No. = 010101

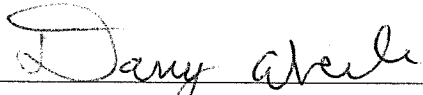

Operator Signature
DARCY ABERLE

Remarks:

Form 106-I8000