



# ETHANOL GAS STANDARD CYLINDER REPORT

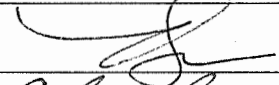
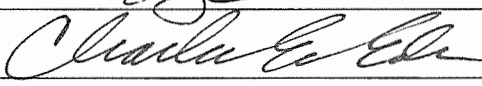
NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <u>Travis Skar</u>		
Location <u>Mountain Co SD</u>	Intoxilyzer® Serial Number <u>80-005361</u>	
Gas Lot Number <u>13518080AG</u>	Gas Cylinder Number <u>27</u>	Gas Expiration Date <u>08/05/20</u>

Check When Done:

*cee*  
*3/6/19*

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
  - A. 0.084 AC
  - B. 0.083 AC
  - C. 0.084 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature 	Date <u>2/26/19</u>
Reviewed By (Crime Laboratory Use Only) 	Date <u>3/8/2019</u>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005361  
Location = MOSO      8164.14.00 09/16  
02/26/2019      15:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:13
02 Std. Gas	0.084	15:13
03 Room Air	0.000	15:14
04 Std. Gas	0.083	15:14
05 Room Air	0.000	15:14
06 Std. Gas	0.084	15:15
07 Room Air	0.000	15:15

Lot No = 13418080A6  
Cyl No = 27  
Exp Date = 08/05/2020  
County = 31      Oper No. = 120513

  
\_\_\_\_\_  
Operator Signature  
TRAVIS SKAR

Remarks:

Form 106-I8000

**Eder, Charles E.**

---

**From:** Eder, Charles E.  
**Sent:** Wednesday, March 6, 2019 10:17 AM  
**To:** Skar, Sgt. Travis M.  
**Subject:** Ethanol Gas Standard Cylinder Report  
**Attachments:** EtOH Gas Cyl Report Mountrail County.pdf

<b>Tracking:</b>	<b>Recipient</b>	<b>Delivery</b>
	Skar, Sgt. Travis M.	Delivered: 3/6/2019 10:18 AM

Morning Travis,

I received the ethanol gas cylinder report and ACA test this morning and it needs to be amended. The lot number on Form 105-G is correct, but the lot number of the gas cylinder typed into the instrument on the ACA test is incorrect. Please review attached PDF.

Form 105-G → Gas Lot Number: 13518080A6 (Correct)  
ACA Test → Gas Lot Number: 13418080A6 (Incorrect)

Sorry for the inconvenience, I know some of the gas cylinder barcode labels in this lot are not being read by the instrument scanner.

Please re-enter the gas cylinder lot number (13518080A6) into the instrument, run a new ACA test, sign and return the ACA test to me.

Thank you for your assistance,  
Charlie

Office of Attorney General  
Crime Lab Division - Toxicology  
Charles E. Eder - State Toxicologist  
2641 East Main Avenue  
Bismarck, ND 58501-5044  
T (701) 328-6159  
F (701) 328-6185  
ceder@nd.gov

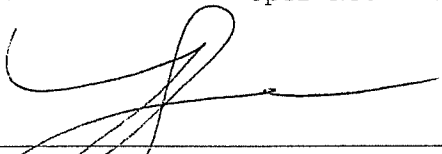
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005361  
Location = MOSO      8164.14.00 09/16  
03/08/2019      14:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:37
02 Std. Gas	0.083	14:38
03 Room Air	0.000	14:38
04 Std. Gas	0.083	14:39
05 Room Air	0.000	14:39
06 Std. Gas	0.083	14:39
07 Room Air	0.000	14:40

Lot No = 13518080A6  
Cyl No = 27  
Exp Date = 08/05/2020  
County = 31      Oper No. = 120513

  
\_\_\_\_\_  
Operator Signature  
TRAVIS SKAR

Remarks:

Form 106-I8000