



# ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Todd Lerol</i>		
Location <i>Powers Lake</i>	Intoxilyzer® Serial Number <i>80-005942</i>	
Gas Lot Number <i>34418080A2</i>	Gas Cylinder Number <i>43</i>	Gas Expiration Date <i>2/05/2021</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C). *ACA TEST SUBMITTED, CHECK BOX Empty, CEE*  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. *0.080* AC  
 B. *0.080* AC  
 C. *0.081* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

*CEE  
3/22/19*

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>3/14/19</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <i>3/22/19</i>

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005942  
Location = PLPD      8164.14.00 09/16  
03/14/2019      18:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:50
02 Std. Gas	0.080	18:51
03 Room Air	0.000	18:51
04 Std. Gas	0.080	18:52
05 Room Air	0.000	18:52
06 Std. Gas	0.081	18:52
07 Room Air	0.000	18:53

Lot No = 34418080A2  
Cyl No = 43  
Exp Date = 02/05/2021  
County = 07      Oper No. = 019709



Operator Signature  
TODD LEROL

Remarks:

ACA Test

Form 106-I8000