



ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>Joshua Nelson</i>		
Location <i>Tioga PD</i>	Intoxilyzer® Serial Number <i>80-006490</i>	
Gas Lot Number <i>3491708DA3</i>	Gas Cylinder Number <i>024</i>	Gas Expiration Date <i>2/5/2020</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0.082 AC
 - B. 0.081 AC
 - C. 0.082 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Joshua Nelson</i>	Date <i>3-19-19</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles Ede</i>	Date <i>3/22/19</i>

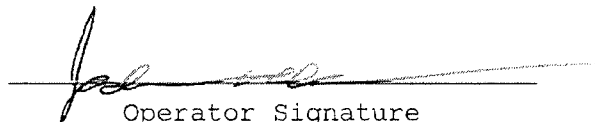
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006490
Location = TIOG 8164.14.00 09/16
03/19/2019 12:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:26
02 Std. Gas	0.082	12:26
03 Room Air	0.000	12:27
04 Std. Gas	0.081	12:27
05 Room Air	0.000	12:28
06 Std. Gas	0.082	12:28
07 Room Air	0.000	12:29

Lot No = 34917080A3
Cyl No = 24
Exp Date = 02/05/2020
County = 53 Oper No. = 130977


Operator Signature
JOSHUA NELSON

Remarks:

Form 106-I8000

**I certify this is a true
and accurate copy.**

X 