Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

Intoxilyzer [®]	8000	Serial	Number:	80-	· 00	41	<u>9</u> {	

Location: Toxム

- General Setup and Checks: Α.
 - Diagnostic Tests Pass and Instrument in Ready Mode 1.
 - 2. Breath tube heated
 - 3. Date, time and location code (Level 2,E). Re-set if necessary.
 - Service Print test (Level 1,P). Sign and attach test record. 4.
 - Tank monitor (Level 3,D,G). Display and Regulator within 50 psi. 5.
 - Display: 914 psi Regulator: 900 psi a.
- Β. Tests (Sign and attach test records):

 \mathbf{X} Configure simulator for the following tests (Level 1,S).

- Wet Calibration Check 1
 - AC (Level 1,C): Known Value ≤ 0.03 AC: 0.02 AC a. Sim. Ser #: DR3375 Lot #: 17160 Exp. Date: 6.8.19
 - \mathbb{X} High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.30° AC b. Sim. Ser #: DR7111 Lot #: 17350 Exp. Date: 10.11.19

 \bowtie Configure dry gas standard for the following tests (Level 1,S).

- 2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone Sim. Ser #: DR 7089 Lot #: ICS 5 Exp. Date: NA Display reads "Interferent Detect"
- RFI Check (CMS Mode) Display reads "RFI Detect" 3.
- Dry Calibration Check (Level 1,C): Known Value 0.080 AC 4. Gas Cyl Lot #:34418080A2 Cyl #: 45 Exp. Date: 2.5.21
 La
 Cyl #.
 Test /
 Organization

 Test 4
 0.080
 Test /
 0.079

 Test 8
 0.079
 Test 8
 0.079
 Test 1 0.080 Test 7 0.080 Test 2 0.080 Test 3 0.080 Average 0,080
- Remarks/Maintenance: REPLACED SIMULATOR RETURN O-RING. C. ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Breath Analyst Signature

Reviewed by

Date AIA

CMI, Inc. Intoxilyzer	Alcohol Analyzer
North Dakota Model 8000	SN 80-004198
Location = TOXL	8164.14.00 09/16
04/15/2019	15:38

abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

Current Instrument Setup Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 34418080A2 Standard Cyl #? 45 02/05/2021 Standard Expiration? Oper No? 666666 Flow Cal. Date: 01/09/2012

 Slope
 637

 Intercept
 -467208

 IR Calibration Date:
 05/18/2016

 3um
 9um

 0th Coef(*100):
 -19233

0011	COCL (100/.		~23030
1st	Coef(*100):	267031	132119
2nd	Coef(*100):	2449	1142
H2O	adj(mg/l*10k):	329	398

Operator Signature CHARLES EDER

Form 106-18000

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-004198Location = TOXL8164.14.00 09/1604/15/201915:40

WET CAL CHECK Test AC Time 01 Room Air 0.000 15:40 02 Std. Sol. 0.020 15:41 03 Room Air 0.000 15:42 04 Std. Sol. 0.021 15:42 05 Room Air 0.000 15:43 06 Std. Sol. 0.021 15:44 07 Room Air 0.000 15:44

08 Sim Temp = 34.0°C

Simul Ser No = DR3375 Std Sol No = 17160 County = 08 Oper No

Oper No. = 666666

Operator Signature CHARLES EDER

Low AC

0,020AC

Form 106-18000

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-004198Location = TOXL8164.14.00 09/1604/15/201915:45

Г

٦

	WET CAL CHECK			
Test	AC	Time		
01 Room Air 02 Std. Sol. 03 Room Air 04 Std. Sol. 05 Room Air 06 Std. Sol. 07 Room Air 08 Sim Temp	0.000 0.302 0.000 0.301 0.000	15:45 15:46 15:46 15:47 15:48 15:48 15:49		
Simul Ser No = DR7111 Std Sol No = 17350 County = 08 Oper No. = 666666 Oper No. = 666666				

Operator Signature CHARLES EDER

Remarks:

0.300 AC

HIGH AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-004198 Location = TOXL8164.14.00 09/16 04/15/2019 15:50 Test AC Time 01 Room Air 0.000 15:51 02 *Subject Test INT* 15:51 03 Room Air 15:52 0.000 *Invalid Test Interferent Detected Sub Name = DISCOVER, THE SPIRIT Sub DOB = 01/01/1982Sub Sex = Female Weight = 150Test = DUI Cit = INTERFERENT Dr. Lic. = ND/DIS821456Lot No = 34418080A2Cyl No = 45Expiration Date = 02/05/2021County = 08Oper No. = 666666

Operator Signature CHARLES EDER

event Check + 0.05% ACETONE

'AAC

Form 106-I8000

NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-004198 Location = TOXL 8164.14.00 09/16 04/15/2019 15:52 Test AC Time 01 Diagnostic 15:53 OK 02 Room Air 0.000 15:53 03 *Subject Test RFI* 15:54 04 Room Air 0.000 15:54 *Invalid Test Inhibited - RFI Sub Name = DISCOVER, THE SPIRIT Sub DOB = 01/01/1982Sub Sex = Female Weight = 150Test = DUI Cit = RFI CHECK Dr. Lic. = ND/DIS821456Lot No = 34418080A2Cyl No = 45Expiration Date = 02/05/2021County = 08Oper No. = 666666

Intoxilyzer Test Record and Checklist

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

leck

Form 106-18000

CMI, Inc. Intoxilyzer	Alcohol Analyzer
North Dakota Model 8000	SN 80-004198
Location = TOXL	8164.14.00 09/16
04/15/2019	15:54

DRY CAL CHECK Test AC Time 01 Room Air 0.000 15:55 02 Std. Gas 0.080 15:55 03 Room Air 0.000 15:56 04 Std. Gas 0.080 15:56 05 Room Air 0.000 15:56 06 Std. Gas 0.080 15:57 07 Room Air 0.000 15:57

Lot No = 34418080A2 Cyl No = 45 Exp Date = 02/05/2021 County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

ALIBRATION CHECK 0.080 AC

Form 106-18000

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-004198Location = TOXL8164.14.00 09/1604/15/201915:57

DRY CAL CHECK Test AC Time 01 Room Air 0.000 15:58 02 Std. Gas 0.080 15:58 03 Room Air 0.000 15:59 04 Std. Gas 0.080 15:59 05 Room Air 0.000 16:00 06 Std. Gas 0.080 16:00 07 Room Air 0.000 16:01

Lot No = 34418080A2 Cyl No = 45 Exp Date = 02/05/2021 County = 08 Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

CHECK ACIBRATION

0.080 AC

Form 106-18000

1

CMI, Inc. Intoxilyzer	Alcohol Analyzer
North Dakota Model 8000	SN 80-004198
Location = TOXL	8164.14.00 09/16
04/15/2019	16:01

DRY CAL CHECK Test AC Time 01 Room Air 0.000 16:01 02 Std. Gas 0.080 16:02 03 Room Air 0.000 16:02 04 Std. Gas 0.079 16:02 05 Room Air 0.000 16:03 06 Std. Gas 0.080 16:03 07 Room Air 0.000 16:04

Lot No = 34418080A2 Cyl No = 45 Exp Date = 02/05/2021 County = 08 Oper No. = 666666

Operator Signature CHARLES EDER

ALIBRATION CHECK 0.080 AC

Form 106-18000