OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

Intoxi	lyzer® 8000 Serial Number: <u>80 - 002669</u>	Location: ToxL	
Α.	General Setup and Checks: 1. Diagnostic Tests Pass and Instrument in Recognition 2. Breath tube heated 3. Date, time and location code (Level 2,E). Recognition 2. Print test (Level 1,P). Sign and attach test recognition 3. Display: Square psi Regulator: 900 psi	e-set if necessary. ecord.	
B.	Tests (Sign and attach test records): Configure simulator for the following tests (Level 1 Wet Calibration Check a.	0.03 AC: 0.020 AC _Exp. Date: 6.8.19 0.25 AC: 0.300 AC _Exp. Date: /0.11.19	
	 Configure dry gas standard for the following tests of the configure dry gas standard for the following tests of the configure dry gas standard for the following tests of the configure of the configuration of the configura	: 0.10 AC + 0.05% Acetone Date: NA ads "RFI Detect" /alue 0.080 AC _Exp. Date: 2.5.21 Test 7 0.079 Test 8 0.080	
C.	Remarks/Maintenance: ANNUAL INSPECTS	ON	
∯Jnst	trument is acceptable to be used in the field.	11 11 2012	
	Analyst Signature	4./6.20/9 Date NA	
Revie	Reviewed by Date		

02/18

Issuing Authority - CEE

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-002669 Location = TOXL 8164.14.00 09/16 04/16/2019 09:37

********* Printer Test ********

abcdefghijklmnopgrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

abcdefghijklmnopqrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

Current Instrument Setup

Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 34418080A2 Standard Cyl #? 45 Standard Expiration? 02/05/2021 Oper No? 666666

Flow Cal. Date: 09/09/2011 Slope 662 Intercept -510765

IR Calibration Date: 02/06/2015 3um

Oth Coef(*100): -31426 1st Coef(*100): 275748 2nd Coef(*100): 1078 -24477 139415 912

H20 adj(mg/l*10k): 429271

****** Printer Test End *******

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
04/16/2019

Alcohol Analyzer SN 80-002669 8164.14.00 09/16 09:43

WET CAL CHECK

Т	est		AC	Time
01	Room	Air	0.000	09:43
02	Std.	Sol.	0.018	09:44
03	Room	Air	0.000	09:45
04	Std.	Sol.	0.018	09:45
05	Room	Air	0.000	09:46
06	Std.	Sol.	0.019	09:47
07	Room	Air	0.000	09:47

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = DR3375 Std Sol No = 17160

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Low AC 0.020 AC Form 106-18000

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 04/16/2019 Alcohol Analyzer SN 80-002669 8164.14.00 09/16 09:48

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:49
02 Std. Sol.	0.294	09:49
03 Room Air	0.000	09:50
04 Std. Sol.	0.295	09:51
05 Room Air	0.000	09:51
06 Std. Sol.	0.296	09:52
07 Room Air	0.000	09:52

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = DR7111 Std Sol No = 17350

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

0.300 AC Form 106-18000

46H AC

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = TOXL 8164.14.00 09/16
04/16/2019 09:54

Test		AC	Time
01	Room Air	0.000	09:55
02	*Subject Test	INT*	09:55
03	Room Air	0.000	09:56

*Invalid Test

Interferent Detected

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 01/01/1982

Sub Sex = Female

Weight = 150

Test = DUI

Cit = INTERFERENT

Dr. Lic. = ND/DIS821456

Lot No = 34418080A2

Cyl No = 45

Expiration Date = 02/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

0.10AC+ 0.05% ACETONA

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-002669
Location = TOXL 8164.14.00 09/16
04/16/2019 09:56

Test	AC	Time
01 Diagnostic	OK	09:57
02 Room Air	0.000	09:58
03 *Subject Test	RFI*	09:58
04 Room Air	0.000	09:58

*Invalid Test Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 01/01/1982

Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK

Dr. Lic. = ND/DIS821456

Lot No = 34418080A2

Cyl No = 45

Expiration Date = 02/05/2021

County = 08

Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 04/16/2019

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:01
02 Std. Gas	0.079	10:01
03 Room Air	0.000	10:02
04 Std. Gas	0.079	10:02
05 Room Air	0.000	10:03
06 Std. Gas	0.079	10:03
07 Room Air	0.000	10:04

Lot No = 34418080A2

Cyl No = 45

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

Alcohol Analyzer

8164.14.00 09/16

SN 80-002669

10:00

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
04/16/2019

Alcohol Analyzer SN 80-002669 8164.14.00 09/16 10:04

DRY CAL CHECK

Te	est		AC	Time
01	Room	Air	0.000	10:04
02	Std.	Gas	0.079	10:05
03	Room	Air	0.000	10:05
04	Std.	Gas	0.079	10:05
05	Room	Air	0.000	10:06
06	Std.	Gas	0.079	10:06
07	Room	Air	0.000	10:07

Lot No = 34418080A2

Cyl No = 45

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
04/16/2019

Alcohol Analyzer SN 80-002669 8164.14.00 09/16 10:07

DRY CAL CHECK

Test		AC	Time
01 Room A	ir 0	0.000	L0:08
02 Std. G	as 0	.079	10:08
03 Room A	ir 0	.000	10:08
04 Std. G	as 0	.080	L0:09
05 Room A	ir 0	.000	L0:09
06 Std. G	as 0	.080	10:10
07 Room A	ir 0	.000	10:10

Lot No = 34418080A2

Cyl No = 45

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

Operator Signature
CHARLES EDER

AUBRATION CUERCE

Remarks:

0.080Ac Form 106-18000