

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004204

Location: TOXL

A. General Setup and Checks:

- 1.  Diagnostic Tests Pass and Instrument in Ready Mode
- 2.  Breath tube heated
- 3.  Date, time and location code (Level 2,E). Re-set if necessary.
- 4.  Print test (Level 1,P). Sign and attach test record.
- 5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 821 psi Regulator: 800 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: DR3375 Lot #: 17160 Exp. Date: 6.8.19
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: DR7111 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

- 2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR7089 Lot #: FCS 5 Exp. Date: NA  
 Display reads "Interferent Detect"
- 3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
- 4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 3441808012 Cyl #: 45 Exp. Date: 2.5.21

Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.081</u>	Test 6 <u>0.080</u>	Test 9 <u>0.081</u>
Average <u>0.080</u>		

C. Remarks/Maintenance: REPLACED SIMULATOR RETURN O-RING,  
CALIBRATED GAS CYLINDER TANK SENSOR, ANNUAL INSPECTION  
REPAVED AND TRAINED EXTERNAL BATTERY

Instrument is acceptable to be used in the field.

Charles E. Eds  
Breath Analyst Signature

4.16.19  
Date

NA  
Reviewed by

NA  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004204  
Location = TOXL                 8164.14.00 09/16  
04/16/2019                        14:54

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=  
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            34418080A2  
Standard Cyl #?            45  
Standard Expiration?       02/05/2021  
Oper No?                    666666

Flow Cal. Date:            07/13/2010  
Slope                      750  
Intercept                  -815939

IR Calibration Date:       05/12/2015  
                                    3um                      9um

-----  
0th Coef(\*100):        -23668                      -20605  
1st Coef(\*100):        259586                      131325  
2nd Coef(\*100):        3023                        1492  
H2O adj(mg/l\*10k):    445                         418

\*\*\*\*\* Printer Test End \*\*\*\*\*

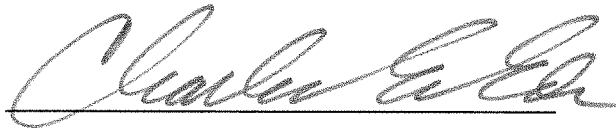

Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004204  
Location = TOXL      8164.14.00 09/16  
04/16/2019      14:55

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:56
02 Std. Sol.	0.020	14:57
03 Room Air	0.000	14:57
04 Std. Sol.	0.020	14:58
05 Room Air	0.000	14:58
06 Std. Sol.	0.020	14:59
07 Room Air	0.000	15:00


08 Sim Temp = 34.0°C

Simul Ser No = DR3375

Std Sol No = 17160

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

*Low AC*  
*0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004204  
Location = TOXL      8164.14.00 09/16  
04/16/2019      15:00

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:01
02 Std. Sol.	0.300	15:02
03 Room Air	0.000	15:02
04 Std. Sol.	0.299	15:03
05 Room Air	0.000	15:03
06 Std. Sol.	0.299	15:04
07 Room Air	0.000	15:05


08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 17350

County = 08

Oper No. = 666666

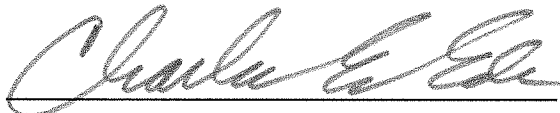

Operator Signature

CHARLES EDER

Remarks:

*HIGH AC*  
*0.300 AC*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004204  
Location = TOXL                    8164.14.00 09/16  
04/16/2019                            15:06

Test	AC	Time
01 Room Air	0.000	15:07
02 *Subject Test	INT*	15:08
03 Room Air	0.000	15:08

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 34418080A2  
Cyl No = 45  
Expiration Date = 02/05/2021  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

~~RFI CHECK~~ CEE  
INTERFERENT CHECK

Form 106-I8000

0.10 AC + 0.05% ACETONE

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004204  
Location = TOXL                    8164.14.00 09/16  
04/16/2019                            15:08

Test	AC	Time
01 Diagnostic	OK	15:09
02 Room Air	0.000	15:10
03 *Subject Test	RFI*	15:10
04 Room Air	0.000	15:10

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 34418080A2  
Cyl No = 45  
Expiration Date = 02/05/2021  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004204  
Location = TOXL      8164.14.00 09/16  
04/16/2019      15:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:11
02 Std. Gas	0.080	15:11
03 Room Air	0.000	15:12
04 Std. Gas	0.081	15:12
05 Room Air	0.000	15:13
06 Std. Gas	0.081	15:13
07 Room Air	0.000	15:14

Lot No = 34418080A2  
Cyl No = 45  
Exp Date = 02/05/2021  
County = 08      Oper No. = 666666

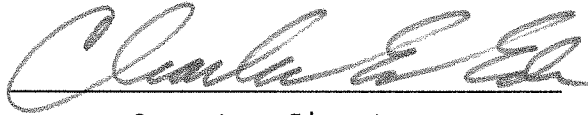

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004204  
Location = TOXL      8164.14.00 09/16  
04/16/2019      15:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:14
02 Std. Gas	0.080	15:15
03 Room Air	0.000	15:15
04 Std. Gas	0.080	15:16
05 Room Air	0.000	15:16
06 Std. Gas	0.080	15:17
07 Room Air	0.000	15:17

Lot No = 34418080A2  
Cyl No = 45  
Exp Date = 02/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004204  
Location = TOXL      8164.14.00 09/16  
04/16/2019      15:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:18
02 Std. Gas	0.081	15:18
03 Room Air	0.000	15:19
04 Std. Gas	0.080	15:19
05 Room Air	0.000	15:19
06 Std. Gas	0.081	15:20
07 Room Air	0.000	15:20

Lot No = 34418080A2  
Cyl No = 45  
Exp Date = 02/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000