

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006491

Location: BELLAH P.D.
(BEPD)

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 1001 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR3375 Lot #: 17160 Exp. Date: 6.8.19
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR7111 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR7089 Lot #: ICS5 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 34917080A3 Cyl #: 21 Exp. Date: 2.5.20

Test 1 <u>0.082</u>	Test 4 <u>0.081</u>	Test 7 <u>0.080</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.082</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>
Average <u>0.081</u>		

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Breath Analyst Signature

Charles E. Eder

NA

Date

4.18.2019

NA

Reviewed by

Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006491
Location = BEPD 8164.14.00 09/16
04/18/2019 15:28

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:28
02 Std. Sol.	0.020	15:29
03 Room Air	0.000	15:30
04 Std. Sol.	0.020	15:30
05 Room Air	0.000	15:31
06 Std. Sol.	0.020	15:32
07 Room Air	0.000	15:32

08 Sim Temp = 34.0°C

Simul Ser No = DR3375
Std Sol No = 17160
County = 29 Oper No. = 666666

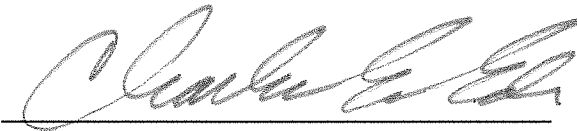

Operator Signature
CHARLES EDER

Remarks: *LOW AC*
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006491
Location = BEPD 8164.14.00 09/16
04/18/2019 15:34

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:35
02 Std. Sol.	0.302	15:36
03 Room Air	0.000	15:36
04 Std. Sol.	0.304	15:37
05 Room Air	0.000	15:37
06 Std. Sol.	0.303	15:38
07 Room Air	0.000	15:39


08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 17350

County = 29

Oper No. = 666666

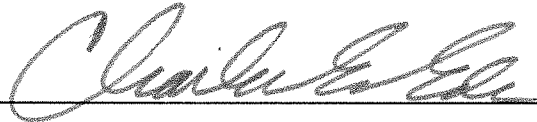

Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006491
Location = BEPD 8164.14.00 09/16
04/18/2019 15:16

Test	AC	Time
01 Room Air	0.000	15:17
02 *Subject Test	INT*	15:17
03 Room Air	0.000	15:18

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 34917080A3
Cyl No = 21
Expiration Date = 02/05/2020
County = 29 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006491
Location = BEPD 8164.14.00 09/16
04/18/2019 15:19

Test	AC	Time
01 Diagnostic	OK	15:20
02 Room Air	0.000	15:20
03 *Subject Test	RFI*	15:20
04 Room Air	0.000	15:21

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 34917080A3
Cyl No = 21
Expiration Date = 02/05/2020
County = 29 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006491
Location = BEPD 8164.14.00 09/16
04/18/2019 15:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:12
02 Std. Gas	0.082	15:13
03 Room Air	0.000	15:13
04 Std. Gas	0.081	15:14
05 Room Air	0.000	15:14
06 Std. Gas	0.082	15:15
07 Room Air	0.000	15:15

Lot No = 34917080A3
Cyl No = 21
Exp Date = 02/05/2020
County = 29 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006491
Location = BEPD 8164.14.00 09/16
04/18/2019 15:22

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:22
02 Std. Gas	0.081	15:23
03 Room Air	0.000	15:23
04 Std. Gas	0.081	15:24
05 Room Air	0.000	15:24
06 Std. Gas	0.081	15:25
07 Room Air	0.000	15:25

Lot No = 34917080A3
Cyl No = 21
Exp Date = 02/05/2020
County = 29 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECK*
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006491
Location = BEPD 8164.14.00 09/16
04/18/2019 15:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:42
02 Std. Gas	0.080	15:42
03 Room Air	0.000	15:43
04 Std. Gas	0.081	15:43
05 Room Air	0.000	15:44
06 Std. Gas	0.081	15:44
07 Room Air	0.000	15:44

Lot No = 34917080A3
Cyl No = 21
Exp Date = 02/05/2020
County = 29 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080AC
Form 106-I8000