

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005361

Location: MOUNTAIN TRAIL

COUNTY S.D.
(MOSO)

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 1029 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR3375 Lot #: 17160 Exp. Date: 6.8.19
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR7111 Lot #: 1711-000 Exp. Date: 10.11.19
17350

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR7089 Lot #: JCS 5 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 13518080A6 Cyl #: 27 Exp. Date: 8.5.20

Test 1 <u>0.083</u>	Test 4 <u>0.081</u>	Test 7 <u>0.082</u>
Test 2 <u>0.082</u>	Test 5 <u>0.082</u>	Test 8 <u>0.082</u>
Test 3 <u>0.082</u>	Test 6 <u>0.083</u>	Test 9 <u>0.082</u>
Average <u>0.082</u>		

C. Remarks/Maintenance: REPLACED BREATH HOSE O-RING
REPLACED SIMULATOR RETURN O-RING
ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Christina Ech
Breath Analyst Signature

5.3.2019

Date

NA

NA

Reviewed by

Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = MOSO 8164.14.00 09/16
05/03/2019 13:31

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:32
02 Std. Sol.	0.020	13:32
03 Room Air	0.000	13:33
04 Std. Sol.	0.021	13:34
05 Room Air	0.000	13:34
06 Std. Sol.	0.020	13:35
07 Room Air	0.000	13:35

08 Sim Temp = 34.0°C

Simul Ser No = DR3375

Std Sol No = 17160

County = 31

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = MOSO 8164.14.00 09/16
05/03/2019 13:47

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:48
02 Std. Sol.	0.299	13:49
03 Room Air	0.000	13:49
04 Std. Sol.	0.299	13:50
05 Room Air	0.000	13:51
06 Std. Sol.	0.299	13:51
07 Room Air	0.000	13:52

08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 17350

County = 31

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = MOSO 8164.14.00 09/16
05/03/2019 13:58

Test	AC	Time
01 Room Air	0.000	13:59
02 *Subject Test	INT*	14:00
03 Room Air	0.000	14:00

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 13518080A6
Cyl No = 27
Expiration Date = 08/05/2020
County = 31 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = MOSO 8164.14.00 09/16
05/03/2019 14:00

Test	AC	Time
01 Diagnostic	OK	14:02
02 Room Air	0.000	14:02
03 *Subject Test	RFI*	14:02
04 Room Air	0.000	14:03

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 13518080A6
Cyl No = 27
Expiration Date = 08/05/2020
County = 31 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

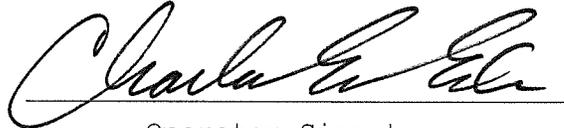

Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = MOSO 8164.14.00 09/16
05/03/2019 13:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:26
02 Std. Gas	0.083	13:26
03 Room Air	0.000	13:27
04 Std. Gas	0.082	13:27
05 Room Air	0.000	13:28
06 Std. Gas	0.082	13:28
07 Room Air	0.000	13:28

Lot No = 13518080A6
Cyl No = 27
Exp Date = 08/05/2020
County = 31 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = MOSO 8164.14.00 09/16
05/03/2019 13:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:55
02 Std. Gas	0.081	13:55
03 Room Air	0.000	13:56
04 Std. Gas	0.082	13:56
05 Room Air	0.000	13:57
06 Std. Gas	0.083	13:57
07 Room Air	0.000	13:58

Lot No = 13518080A6
Cyl No = 27
Exp Date = 08/05/2020
County = 31 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005361
Location = MOSO 8164.14.00 09/16
05/03/2019 14:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:04
02 Std. Gas	0.082	14:04
03 Room Air	0.000	14:04
04 Std. Gas	0.082	14:05
05 Room Air	0.000	14:05
06 Std. Gas	0.082	14:06
07 Room Air	0.000	14:06

Lot No = 13518080A6
Cyl No = 27
Exp Date = 08/05/2020
County = 31 Oper No. = 666666

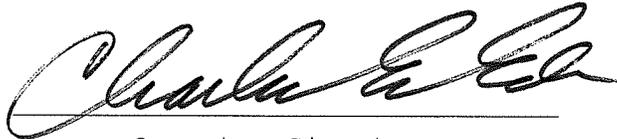

Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000