OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

Into	xilyzer [®] 8000 Serial Number: _.	80-006502	Location: STUTS MAN GUN!
Α.	General Setup and Checks 1.	s: Pass and Instrument in I	Ready Mode Re-set if necessary. t record. Regulator within 50 psi.
B.	Sim. Ser #: D b. High AC (L Sim. Ser #: D Configure dry gas stand 2. Interferent Check Sim. Ser #: DR 70 8 Display reads "Interferent Check (CMS) RFI Check (CMS) RFI Check (CMS) Dry Calibration Check Gas Cyl Lot #: 1351 Test 1 Test 2 Test 3 Average O.08 Average	the following tests (Level ck evel 1,C): Known Value R3375 Lot #: 17166 evel 1,C): Known Value R7111 Lot #: 17356 ard for the following test (Level 1,B): Known Value (Level 1,B): Known Value (Level 1,B): Known Value (Level 1,C): Known Value (Leve	≤ 0.03 AC:
C.	Remarks/Maintenance:	INNUAL INSPE	_C770N
M Ins	strument is acceptable to be u	weed in the field.	
	Chalas	El	5.7.2019
Breat	th Analyst Signature NA		Date N A
Revie	ewed by	49 - 43 - 17 - 17 - 18 - 18 - 18 - 18 - 18 - 18	Date

02/18

Issuing Authority - CEE

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006502
Location = STUT 8164.14.00 09/16
05/07/2019 14:06

******** Printer Test *******

abcdefghijklmnopqrstuvwxyz1234567890-= |
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 13518080A6 Standard Cyl #? 34 Standard Expiration? 08/05/2020 Oper No? 666666

Flow Cal. Date: 08/19/2015 Slope 674 Intercept -654491

IR Calibration Date: 08/19/2015
3um 9um

Oth Coef(*100): -24446 -18103 1st Coef(*100): 267787 135417 2nd Coef(*100): 3387 1610 H2O adj(mg/l*10k): 740 513

******* Printer Test End *******

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

RINT TEST

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = STUT
05/07/2019

Alcohol Analyzer SN 80-006502 8164.14.00 09/16 14:08

WET CAL CHECK

T.	est		AC	Time
01	Room	Air	0.000	14:09
02	Std.	Sol.	0.019	14:10
03	Room	Air	0.000	14:10
04	Std.	Sol.	0.019	14:11
05	Room	Air	0.000	14:12
06	Std.	Sol.	0.019	14:12
07	Room	Air	0.000	14:13

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = DR3375 Std Sol No = 17160 County = 47

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

LOW AC

0.020AC

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = STUT
05/07/2019

Alcohol Analyzer SN 80-006502 8164.14.00 09/16 14:13

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:14
02 Std. Sol.	0.298	14:15
03 Room Air	0.000	14:15
04 Std. Sol.	0.299	14:16
05 Room Air	0.000	14:17
06 Std. Sol.	0.300	14:17
07 Room Air	0.000	14:18

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = DR7111 Std Sol No = 17350 County = 47

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

47614 AC

CMI, Inc. Intoxilyzer North Dakota Model 8000 SN 80-006502 Location = STUT 05/07/2019

Alcohol Analyzer 8164.14.00 09/16 14:20

Test	AC	Time
01 Room Air	0.000	14:20
02 *Subject Test	INT*	14:21
03 Room Air	0.000	14:21

*Invalid Test Interferent Detected

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 01/01/1982

Sub Sex = Female

Weight = 150

Test = DUI

Cit = INTERFERENT

Dr. Lic. = ND/DIS821456

Lot No = 13518080A6

Cyl No = 34

Expiration Date = 08/05/2020

County = 47

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = STUT
05/07/2019

Alcohol Analyzer SN 80-006502 8164.14.00 09/16 14:21

Test .	AC	Time
01 Diagnostic	OK	14:22
02 Room Air	0.000	14:23
03 *Subject Test	RFI*	14:23
04 Room Air	0.000	14:23

*Invalid Test Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 01/01/1982

Sub Sex = Female

Weight = 150

Test = DUI

Cit = RFI CHECK

Dr. Lic. = ND/DIS821456

Lot No = 13518080A6

Cyl No = 34

Expiration Date = 08/05/2020

County = 47

Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = STUT 05/07/2019

Alcohol Analyzer SN 80-006502 8164.14.00 09/16 14:24

DRY CAL CHECK

Test			AC	Time
01	Room	Air	0.000	14:24
02	Std.	Gas	0.082	14:25
03	Room	Air	0.000	14:25
04	Std.	Gas	0.082	14:25
05	Room	Air	0.000	14:26
06	Std.	Gas	0.082	14:26
07	Room	Air	0.000	14:27

Lot No = 13518080A6

Cyl No = 34

Exp Date = 08/05/2020

County = 47

Oper No. = 666666

Operator Signature
CHARLES EDER
ALIBRATISM CHECK

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = STUT 05/07/2019

Alcohol Analyzer SN 80-006502 8164.14.00 09/16 14:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:28
02 Std. Gas	0.082	14:28
03 Room Air	0.000	14:29
04 Std. Gas	0.083	14:29
05 Room Air	0.000	14:29
06 Std. Gas	0.083	14:30
07 Room Air	0.000	14:30

Lot No = 13518080A6

Cyl No = 34

Exp Date = 08/05/2020

County = 47

Oper No. = 666666

CHARLES EDER

AUBRATION CHECK Operator Signature

Remarks:

Form 106-I8000

0.080 AC

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = STUT 05/07/2019

Alcohol Analyzer SN 80-006502 8164.14.00 09/16 14:30

DRY CAL CHECK

Test			AC	Time
01	Room	Air	0.000	14:31
02	Std.	Gas	0.082	14:31
03	Room	Air	0.000	14:32
04	Std.	Gas	0.083	14:32
05	Room	Air	0.000	14:33
06	Std.	Gas	0.082	14:33
07	Room	Air	0.000	14:34

Lot No = 13518080A6

Cyl No = 34

Exp Date = 08/05/2020

County = 47

Oper No. = 666666

Remarks:

CHARLES EDER

CACIBRATION CHECK

0.080 AC