

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006494

Location: BIA-BELCOURT
TURNE MT. AGENCY
(BELC)

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 999 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR3375 Lot #: 17160 Exp. Date: 6.8.19
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR7111 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: R7089 Lot #: ICS 5 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 34418080A2 Cyl #: 37 Exp. Date: 2.5.21
Test 1 0.082 Test 4 0.082 Test 7 0.082
Test 2 0.082 Test 5 0.082 Test 8 0.082
Test 3 0.082 Test 6 0.081 Test 9 0.082
Average 0.082

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Eds
Breath Analyst Signature

5.9.2019
Date

NA

NA

Reviewed by

Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = BELC 8164.14.00 09/16
05/09/2019 13:57

WET CAL CHECK

| Test | AC | Time |
|--------------|-------|-------|
| 01 Room Air | 0.000 | 13:57 |
| 02 Std. Sol. | 0.019 | 13:58 |
| 03 Room Air | 0.000 | 13:58 |
| 04 Std. Sol. | 0.020 | 13:59 |
| 05 Room Air | 0.000 | 14:00 |
| 06 Std. Sol. | 0.020 | 14:00 |
| 07 Room Air | 0.000 | 14:01 |

08 Sim Temp = 34.0°C

Simul Ser No = DR3375

Std Sol No = 17160

County = 63

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = BELC 8164.14.00 09/16
05/09/2019 14:01

WET CAL CHECK

| Test | AC | Time |
|--------------|-------|-------|
| 01 Room Air | 0.000 | 14:02 |
| 02 Std. Sol. | 0.297 | 14:03 |
| 03 Room Air | 0.000 | 14:03 |
| 04 Std. Sol. | 0.299 | 14:04 |
| 05 Room Air | 0.000 | 14:05 |
| 06 Std. Sol. | 0.300 | 14:05 |
| 07 Room Air | 0.000 | 14:06 |

08 Sim Temp = 34.0°C

Simul Ser No = DR7111
Std Sol No = 17350
County = 63 Oper No. = 666666

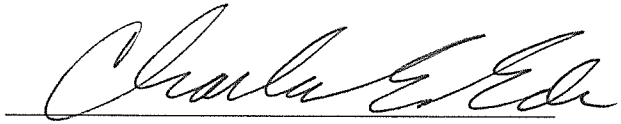

Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = BELC 8164.14.00 09/16
05/09/2019 13:47

| Test | AC | Time |
|------------------|-------|-------|
| 01 Diagnostic | OK | 13:48 |
| 02 Room Air | 0.000 | 13:48 |
| 03 *Subject Test | RFI* | 13:49 |
| 04 Room Air | 0.000 | 13:49 |

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 34418080A2
Cyl No = 37
Expiration Date = 02/05/2021
County = 63 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = BELC 8164.14.00 09/16
05/09/2019 13:42

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 13:42 |
| 02 Std. Gas | 0.082 | 13:43 |
| 03 Room Air | 0.000 | 13:43 |
| 04 Std. Gas | 0.082 | 13:44 |
| 05 Room Air | 0.000 | 13:44 |
| 06 Std. Gas | 0.082 | 13:45 |
| 07 Room Air | 0.000 | 13:45 |

Lot No = 34418080A2
Cyl No = 37
Exp Date = 02/05/2021
County = 63 Oper No. = 666666

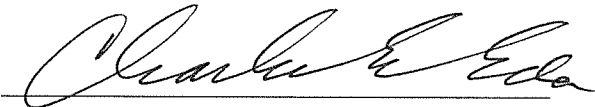

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = BELC 8164.14.00 09/16
05/09/2019 13:49

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 13:50 |
| 02 Std. Gas | 0.082 | 13:50 |
| 03 Room Air | 0.000 | 13:51 |
| 04 Std. Gas | 0.082 | 13:51 |
| 05 Room Air | 0.000 | 13:52 |
| 06 Std. Gas | 0.081 | 13:52 |
| 07 Room Air | 0.000 | 13:53 |

Lot No = 34418080A2
Cyl No = 37
Exp Date = 02/05/2021
County = 63 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006494
Location = BELC 8164.14.00 09/16
05/09/2019 14:07

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 14:07 |
| 02 Std. Gas | 0.082 | 14:08 |
| 03 Room Air | 0.000 | 14:08 |
| 04 Std. Gas | 0.082 | 14:09 |
| 05 Room Air | 0.000 | 14:09 |
| 06 Std. Gas | 0.082 | 14:10 |
| 07 Room Air | 0.000 | 14:10 |

Lot No = 34418080A2
Cyl No = 37
Exp Date = 02/05/2021
County = 63 Oper No. = 666666

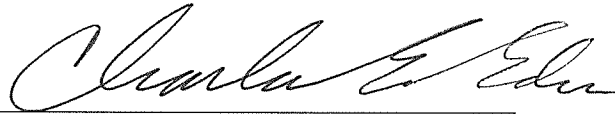

Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECK*
0.080 AC

Form 106-I8000