

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006490

Location: TIOGA P.D.  
(TIOG)

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 997 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: DR3375 Lot #: 17160 Exp. Date: 6.8.19
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: DR7111 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR7089 Lot #: ICS 5 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 34917080A3 Cyl #: 24 Exp. Date: 2.5.2020

Test 1 <u>0.082</u>	Test 4 <u>0.081</u>	Test 7 <u>0.080</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.082</u>	Test 6 <u>0.081</u>	Test 9 <u>0.080</u>
Average <u>0.081</u>		

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Eh  
Breath Analyst Signature

5.9.2019  
Date

NA  
Reviewed by

NA  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006490  
Location = TIOG      8164.14.00 09/16  
05/09/2019      08:05

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:      Enabled  
Start Test Sequence:      DABACABA  
Display Prelim Rslt?      Yes  
Display Third Digit?      Yes  
Inhib Printer(Y/N)?      No  
Display Volume?      No  
Disable On Memfull?      Yes  
# of Print Copies?      1  
Select Std (D/W/I)?      Dry  
Standard Value?      0.080  
Standard Lot #?      34917080A3  
Standard Cyl #?      24  
Standard Expiration?      02/05/2020  
Oper No?      666666

Flow Cal. Date:      08/18/2015  
Slope      675  
Intercept      -635799

IR Calibration Date:      08/18/2015

	3um	9um
0th Coef(*100):	-18073	-15729
1st Coef(*100):	265337	133887
2nd Coef(*100):	3965	1560
H2O adj(mg/l*10k):	767	457

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

**PRINT TEST**

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006490  
Location = TIOG      8164.14.00 09/16  
05/09/2019      08:27

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:28
02 Std. Sol.	0.017	08:29
03 Room Air	0.000	08:29
04 Std. Sol.	0.018	08:30
05 Room Air	0.000	08:30
06 Std. Sol.	0.018	08:31
07 Room Air	0.000	08:32

08 Sim Temp = 34.0°C

Simul Ser No = DR3375  
Std Sol No = 17160  
County = 53      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

Low AC  
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006490  
Location = TIOG      8164.14.00 09/16  
05/09/2019      08:42

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:43
02 Std. Sol.	0.300	08:44
03 Room Air	0.000	08:44
04 Std. Sol.	0.302	08:45
05 Room Air	0.000	08:46
06 Std. Sol.	0.302	08:46
07 Room Air	0.000	08:47

08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 17350

County = 53

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

HIGH AC  
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006490  
Location = TIOG                    8164.14.00 09/16  
05/09/2019                            08:13

Test	AC	Time
01 Room Air	0.000	08:14
02 *Subject Test	INT*	08:15
03 Room Air	0.000	08:15

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 34917080A3  
Cyl No = 24  
Expiration Date = 02/05/2020  
County = 53                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: **INTERFERENT CHECK**  
**0.10AC + 0.05% ACETONE**

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006490  
Location = TIOG                      8164.14.00 09/16  
05/09/2019                              08:20

Test	AC	Time
01 Diagnostic	OK	08:21
02 Room Air	0.000	08:22
03 *Subject Test	RFI*	08:22
04 Room Air	0.000	08:22

\*Invalid Test  
  Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                      Weight = 150  
Test = DUI                              Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 34917080A3  
Cyl No = 24  
Expiration Date = 02/05/2020  
County = 53                              Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

*RFI CHECK*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006490  
Location = TIOG      8164.14.00 09/16  
05/09/2019      08:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:09
02 Std. Gas	0.082	08:09
03 Room Air	0.000	08:10
04 Std. Gas	0.081	08:10
05 Room Air	0.000	08:11
06 Std. Gas	0.082	08:11
07 Room Air	0.000	08:12

Lot No = 34917080A3  
Cyl No = 24  
Exp Date = 02/05/2020  
County = 53      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *CALIBRATION CHECK*  
*0.080 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006490  
Location = TIOG      8164.14.00 09/16  
05/09/2019      08:16

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:16
02 Std. Gas	0.081	08:16
03 Room Air	0.000	08:17
04 Std. Gas	0.081	08:17
05 Room Air	0.000	08:18
06 Std. Gas	0.081	08:18
07 Room Air	0.000	08:19

Lot No = 34917080A3  
Cyl No = 24  
Exp Date = 02/05/2020  
County = 53

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*CALIBRATION CHECK*  
*0.080 AC*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006490  
Location = TIOG      8164.14.00 09/16  
05/09/2019      08:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:49
02 Std. Gas	0.080	08:49
03 Room Air	0.000	08:50
04 Std. Gas	0.081	08:50
05 Room Air	0.000	08:51
06 Std. Gas	0.080	08:51
07 Room Air	0.000	08:52

Lot No = 34917080A3  
Cyl No = 24  
Exp Date = 02/05/2020  
County = 53      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080AC

Form 106-I8000