

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006506

Location: BOTTINEAU COUNTY  
S.O. (BOTT)

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 903 psi Regulator: 900 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
    - a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: DR3375 Lot #: 17160 Exp. Date: 6.8.19
    - b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: DR7111 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR7089 Lot #: ICS 5 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 19817080A4 Cyl #: 9 Exp. Date: 9/5/2019

Test 1	<u>0.081</u>	Test 4	<u>0.082</u>	Test 7	<u>0.081</u>
Test 2	<u>0.082</u>	Test 5	<u>0.081</u>	Test 8	<u>0.081</u>
Test 3	<u>0.082</u>	Test 6	<u>0.081</u>	Test 9	<u>0.082</u>
Average	<u>0.081</u>				

C. Remarks/Maintenance: REPLACED BREATH HOSE O-RING,  
TAPED TEAR STARTING IN PROTECTIVE COVER OF BREATH HOSE,  
ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ehr  
Breath Analyst Signature

5.9.2019  
Date

NA  
Reviewed by

NA  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer           Alcohol Analyzer  
North Dakota Model 8000       SN 80-006506  
Location = BOTT               8164.14.00 09/16  
05/09/2019                       12:09

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

Data Entry Mode:           Enabled  
Start Test Sequence:       DABACABA  
Display Prelim Rslt?       Yes  
Display Third Digit?       Yes  
Inhib Printer(Y/N)?       No  
Display Volume?           No  
Disable On Memfull?       Yes  
# of Print Copies?         1  
Select Std (D/W/I)?       Dry  
Standard Value?           0.080  
Standard Lot #?           19817080A4  
Standard Cyl #?           9  
Standard Expiration?      09/05/2019  
Oper No?                  666666

Flow Cal. Date:           08/20/2015  
Slope                      657  
Intercept                 -627881

IR Calibration Date:      08/20/2015  
                            3um                      9um

-----  
0th Coef(\*100):       -22470               -18919  
1st Coef(\*100):       266505               136109  
2nd Coef(\*100):       3702                  1644  
H2O adj(mg/l\*10k):   709                   510

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:                PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006506  
Location = BOTT      8164.14.00 09/16  
05/09/2019      12:26

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:27
02 Std. Sol.	0.018	12:27
03 Room Air	0.000	12:28
04 Std. Sol.	0.019	12:29
05 Room Air	0.000	12:29
06 Std. Sol.	0.020	12:30
07 Room Air	0.000	12:30

08 Sim Temp = 34.0°C

Simul Ser No = DR3375

Std Sol No = 17160

County = 05

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Low AC*  
*0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006506  
Location = BOTT      8164.14.00 09/16  
05/09/2019      12:38

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:39
02 Std. Sol.	0.303	12:39
03 Room Air	0.000	12:40
04 Std. Sol.	0.303	12:41
05 Room Air	0.000	12:41
06 Std. Sol.	0.303	12:42
07 Room Air	0.000	12:42

08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 17350

County = 05

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*HIGH AC*

*0.300 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006506  
Location = BOTT                    8164.14.00 09/16  
05/09/2019                            12:20

Test	AC	Time
01 Room Air	0.000	12:21
02 *Subject Test	INT*	12:21
03 Room Air	0.000	12:22

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 9  
Expiration Date = 09/05/2019  
County = 05                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Interferent Check*  
*0.10AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006506  
Location = BOTT                    8164.14.00 09/16  
05/09/2019                            12:14

Test	AC	Time
01 Diagnostic	OK	12:15
02 Room Air	0.000	12:16
03 *Subject Test	RFI*	12:16
04 Room Air	0.000	12:16

\*Invalid Test  
  Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 19817080A4  
Cyl No = 9  
Expiration Date = 09/05/2019  
County = 05                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

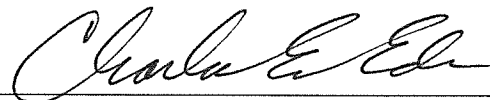

Operator Signature  
CHARLES EDER

Remarks:

*RFI CHECK*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006506  
Location = BOTT      8164.14.00 09/16  
05/09/2019      12:10

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:11
02 Std. Gas	0.081	12:11
03 Room Air	0.000	12:12
04 Std. Gas	0.082	12:12
05 Room Air	0.000	12:13
06 Std. Gas	0.082	12:13
07 Room Air	0.000	12:14

Lot No = 19817080A4  
Cyl No = 9  
Exp Date = 09/05/2019  
County = 05      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006506  
Location = BOTT      8164.14.00 09/16  
05/09/2019      12:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:17
02 Std. Gas	0.082	12:17
03 Room Air	0.000	12:18
04 Std. Gas	0.081	12:18
05 Room Air	0.000	12:19
06 Std. Gas	0.081	12:19
07 Room Air	0.000	12:20

Lot No = 19817080A4  
Cyl No = 9  
Exp Date = 09/05/2019  
County = 05      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006506  
Location = BOTT      8164.14.00 09/16  
05/09/2019      12:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:45
02 Std. Gas	0.081	12:45
03 Room Air	0.000	12:45
04 Std. Gas	0.081	12:46
05 Room Air	0.000	12:46
06 Std. Gas	0.082	12:47
07 Room Air	0.000	12:47

Lot No = 19817080A4  
Cyl No = 9  
Exp Date = 09/05/2019  
County = 05      Oper No. = 666666

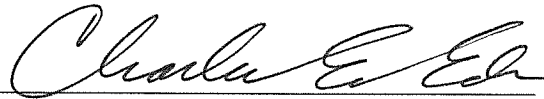

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000