

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005944

Location: STEELE COUNTY
S.O. (FINL)

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 990 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR5131 Lot #: 201810D Exp. Date: 10.24.20
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR7111 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR7089 Lot #: ICS 5 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 34917080A3 Cyl #: 1 Exp. Date: 2.5.20

Test 1	<u>0.080</u>	Test 4	<u>0.081</u>	Test 7	<u>0.080</u>
Test 2	<u>0.081</u>	Test 5	<u>0.081</u>	Test 8	<u>0.081</u>
Test 3	<u>0.082</u>	Test 6	<u>0.081</u>	Test 9	<u>0.081</u>
Average	<u>0.081</u>				

C. Remarks/Maintenance: REPLACED BREATH HOSE O-RING,
ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ed
Breath Analyst Signature

5.14.2019
Date

NA

NA

Reviewed by

Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = FINL 8164.14.00 09/16
05/14/2019 14:32

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 34917080A3
Standard Cyl #? 1
Standard Expiration? 02/05/2020
Oper No? 666666

Flow Cal. Date: 06/20/2013
Slope 677
Intercept -702620

IR Calibration Date: 06/09/2017
3um 9um

	3um	9um
0th Coef(*100):	-16158	-17815
1st Coef(*100):	266780	136962
2nd Coef(*100):	1117	1089
H2O adj(mg/l*10k):	317	364

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = FINL 8164.14.00 09/16
05/14/2019 14:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:45
02 Std. Sol.	0.020	14:45
03 Room Air	0.000	14:46
04 Std. Sol.	0.021	14:47
05 Room Air	0.000	14:47
06 Std. Sol.	0.021	14:48
07 Room Air	0.000	14:48


08 Sim Temp = 34.0°C

Simul Ser No = DR5131

Std Sol No = 201810D

County = 46

Oper No. = 666666

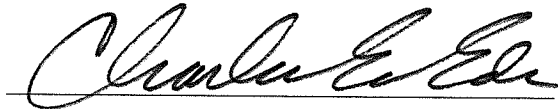

Operator Signature

CHARLES EDER

Remarks:

Low AC

0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = FINL 8164.14.00 09/16
05/14/2019 14:49

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:50
02 Std. Sol.	0.301	14:51
03 Room Air	0.000	14:51
04 Std. Sol.	0.302	14:52
05 Room Air	0.000	14:53
06 Std. Sol.	0.303	14:53
07 Room Air	0.000	14:54


08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 17350

County = 46

Oper No. = 666666

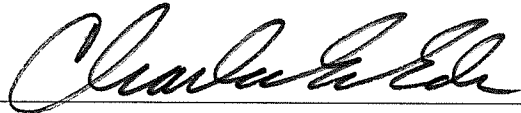

Operator Signature

CHARLES EDER

Remarks:

HIGH AC

0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = FINL 8164.14.00 09/16
05/14/2019 14:38

Test	AC	Time
01 Room Air	0.000	14:39
02 *Subject Test	INT*	14:39
03 Room Air	0.000	14:40

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 34917080A3
Cyl No = 1
Expiration Date = 02/05/2020
County = 46 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = FINL 8164.14.00 09/16
05/14/2019 14:36

Test	AC	Time
01 Diagnostic	OK	14:37
02 Room Air	0.000	14:37
03 *Subject Test	RFI*	14:37
04 Room Air	0.000	14:38

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 34917080A3
Cyl No = 1
Expiration Date = 02/05/2020
County = 46 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

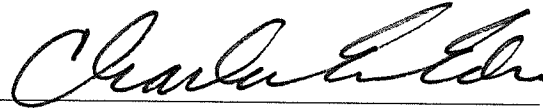

Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = FINL 8164.14.00 09/16
05/14/2019 14:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:33
02 Std. Gas	0.080	14:33
03 Room Air	0.000	14:34
04 Std. Gas	0.081	14:34
05 Room Air	0.000	14:35
06 Std. Gas	0.082	14:35
07 Room Air	0.000	14:36

Lot No = 34917080A3
Cyl No = 1
Exp Date = 02/05/2020
County = 46 Oper No. = 666666

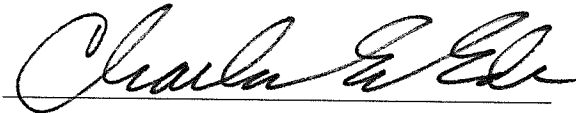

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = FINL 8164.14.00 09/16
05/14/2019 14:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:41
02 Std. Gas	0.081	14:41
03 Room Air	0.000	14:42
04 Std. Gas	0.081	14:42
05 Room Air	0.000	14:42
06 Std. Gas	0.081	14:43
07 Room Air	0.000	14:43

Lot No = 34917080A3
Cyl No = 1
Exp Date = 02/05/2020
County = 46 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC
Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005944
Location = FINL 8164.14.00 09/16
05/14/2019 14:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:55
02 Std. Gas	0.080	14:55
03 Room Air	0.000	14:56
04 Std. Gas	0.081	14:56
05 Room Air	0.000	14:57
06 Std. Gas	0.081	14:57
07 Room Air	0.000	14:58

Lot No = 34917080A3

Cyl No = 1

Exp Date = 02/05/2020

County = 46

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK

0.080 AC

Form 106-I8000