Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

| Intoxil | yzer® 8000 Serial Number: 80-00 4945 Location: 10xL |
|---------|--|
| Α. | General Setup and Checks: 1. Diagnostic Tests Pass and Instrument in Ready Mode 2. Diagnostic Tests Pass and Instrument in Ready Mode 3. Diagnostic Tests Pass and Instrument in Ready Mode 4. Diagnostic Tests Pass and Instrument in Ready Mode 5. Diagnostic Tests Pass and Instrument in Ready Mode 6. Diagnostic Tests Pass and Instrument in Ready Mode 7. Diagnostic Tests Pass and Instrument in Ready Mode 8. Diagnostic Tests Pass and Instrument in Ready Mode 9. Diagnostic Tests Pass and Instrument |
| B. | Tests (Sign and attach test records): ☐ Configure simulator for the following tests (Level 1,S). 1. Wet Calibration Check a. ☐ Low AC (Level 1,C): Known Value ≤ 0.03 AC: O·ORO AC Sim. Ser #: DR3378 Lot #: 2017 128 Exp. Date: 12·12·19 b. ☐ High AC (Level 1,C): Known Value ≥ 0.25 AC: O·300 AC Sim. Ser #: DR7350 Lot #: 201803 14 Exp. Date: 3·22·20 |
| | Configure dry gas standard for the following tests (Level 1,S). Interferent Check (Level 1,B): Known Value: 0·/0 AC + 0.05% Acetone Sim. Ser #: DR 345/ Lot #: ICS 5 Exp. Date: NA Display reads "Interferent Detect" MRFI Check (CMS Mode) |
| C. | Remarks/Maintenance: ANNUAL INSPECTION |
| | |
| | rument is acceptable to be used in the field. Analyst Signature AA AA AA AA AA AA AA AA AA |
| Reviev | wed by Date |

02/18

Issuing Authority - CEE

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
06/11/2019 11:43

******* Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

Current Instrument Setup

Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 34418080A2 Standard Cyl #? 45 Standard Expiration? 02/05/2021 Oper No? 666666

Flow Cal. Date: 09/26/2012 Slope 622 Intercept -485836

IR Calibration Date: 04/22/2016
3um 9um

 0th Coef(*100):
 -23427
 -26406

 1st Coef(*100):
 266989
 144341

 2nd Coef(*100):
 2085
 1078

 H2O adj(mg/l*10k):
 539
 400

******* Printer Test End ********

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 06/11/2019 Alcohol Analyzer SN 80-004945 8164.14.00 09/16 11:47

WET CAL CHECK

| Test | AC | Time |
|--------------|-------|-------|
| 01 Room Air | 0.000 | 11:48 |
| 02 Std. Sol. | 0.019 | 11:48 |
| 03 Room Air | 0.000 | 11:49 |
| 04 Std. Sol. | 0.018 | 11:50 |
| 05 Room Air | 0.000 | 11:50 |
| 06 Std. Sol. | 0.018 | 11:51 |
| 07 Room Air | 0.000 | 11:51 |

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = DR3378 Std Sol No = 201712B

County = 08

Oper No. = 666666

C 0.020 AC

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 06/11/2019 Alcohol Analyzer SN 80-004945 8164.14.00 09/16 11:53

WET CAL CHECK

| Test | AC | Time |
|--------------|-------|-------|
| 01 Room Air | 0.000 | 11:54 |
| 02 Std. Sol. | 0.300 | 11:55 |
| 03 Room Air | 0.000 | 11:55 |
| 04 Std. Sol. | 0.302 | 11:56 |
| 05 Room Air | 0.000 | 11:57 |
| 06 Std. Sol. | 0.302 | 11:57 |
| 07 Room Air | 0.000 | 11:58 |

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = DR7350 Std Sol No = 201803H

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

0,300AC

Form 106-I8000

HIGH AC

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
06/11/2019 11:59

| Test | AC | Time |
|------------------|-------|-------|
| 01 Room Air | 0.000 | 11:59 |
| 02 *Subject Test | INT* | 12:00 |
| 03 Room Air | 0.000 | 12:01 |

*Invalid Test

Interferent Detected

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150

Test = DUI Cit = INTERFERENT

Dr. Lic. = ND/DIS921456

Lot No = 34418080A2

Cyl No = 45

Remarks:

Expiration Date = 02/05/2021

County = 08 Oper No. = 666666

Operator Signature CHARLES EDER

1

D. 10AC+ D.US! ACETONE

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
06/11/2019 12:01

| Test | | AC | Time |
|------|---------------|-------|-------|
| 01 | Diagnostic | OK | 12:02 |
| 02 | Room Air | 0.000 | 12:02 |
| 03 | *Subject Test | RFI* | 12:02 |
| 04 | Room Air | 0.000 | 12:03 |

*Invalid Test Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK

Dr. Lic. = ND/DIS921456

Lot No = 34418080A2

Cyl No = 45

Expiration Date = 02/05/2021

County = 08

Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/11/2019

Alcohol Analyzer SN 80-004945 8164.14.00 09/16 12:03

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 12:04 |
| 02 Std. Gas | 0.080 | 12:04 |
| 03 Room Air | 0.000 | 12:05 |
| 04 Std. Gas | 0.079 | 12:05 |
| 05 Room Air | 0.000 | 12:05 |
| 06 Std. Gas | 0.079 | 12:06 |
| 07 Room Air | 0.000 | 12:06 |

Lot No = 34418080A2

Cyl No = 45

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

ALBRATION CHECK

Remarks:

0-080 Ac Form 106-18000

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/11/2019

Alcohol Analyzer SN 80-004945 8164.14.00 09/16 12:07

DRY CAL CHECK

| Test | | | AC | Time |
|------|------|-----|-------|-------|
| 01 | Room | Air | 0.000 | 12:08 |
| 02 | Std. | Gas | 0.079 | 12:08 |
| 03 | Room | Air | 0.000 | 12:08 |
| 04 | Std. | Gas | 0.080 | 12:09 |
| 05 | Room | Air | 0.000 | 12:09 |
| 06 | Std. | Gas | 0.079 | 12:10 |
| 07 | Room | Air | 0.000 | 12:10 |

Lot No = 34418080A2

Cyl No = 45

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

PALIBRATION CHECK

Remarks:

0-080AC Form 106-18000

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/11/2019

Alcohol Analyzer SN 80-004945 8164.14.00 09/16 12:11

DRY CAL CHECK

| Test | | | AC | Time |
|------|------|-----|-------|-------|
| 01 | Room | Air | 0.000 | 12:11 |
| 02 | Std. | Gas | 0.080 | 12:12 |
| 03 | Room | Air | 0.000 | 12:12 |
| 04 | Std. | Gas | 0.079 | 12:13 |
| 05 | Room | Air | 0.000 | 12:13 |
| 06 | Std. | Gas | 0.080 | 12:13 |
| 07 | Room | Air | 0.000 | 12:14 |

Lot No = 34418080A2

Cyl No = 45

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

ALIBRATION CHECK

Remarks:

0.080 AC