OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

Intoxi	ilyzer® 8000 Serial Number: 80-009956 Location: To X L				
Α.	General Setup and Checks: 1.				
B.	Tests (Sign and attach test records): Configure simulator for the following tests (Level 1,S). Wet Calibration Check a.				
	2. [X Interferent Check (Level 1,B): Known Value: 0.70 AC + 0.05% Acetone				
	Sim. Ser #: <u>DR3451</u> Lot #: <u>TCS 5</u> Exp. Date: <u>NA</u>				
	Display reads "Interferent Detect" 3. A RFI Check (CMS Mode) Display reads "RFI Detect" 4. Dry Calibration Check (Level 1,C): Known Value O.080 AC Gas Cyl Lot #: 3441808042 Cyl #: 45 Exp. Date: 2.5.21 Test 1 0.080 Test 4 0.079 Test 7 0.079 Test 2 0.080 Test 5 0.079 Test 8 0.080 Test 3 0.079 Test 6 0.080 Test 9 0.079 Average 0.079				
C.	Remarks/Maintenance: ANNUAL FNSPECTION				
X Inst	trument is acceptable to be used in the field.				
Breath	Breath Analyst Signature Date NA Date				
Revie	Reviewed by Date				

02/18

Issuing Authority - CEE

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.14.00 09/16
06/13/2019 10:47

******* Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

Current Instrument Setup

Enabled Data Entry Mode: Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 34418080A2 Standard Cyl #? 45 Standard Expiration? 02/05/2021 Oper No? 666666

Flow Cal. Date: 06/13/2019
Slope 606
Intercept -586035

IR Calibration Date: 06/13/2019
3um 9um

Oth Coef(*100): -19986 -21891
1st Coef(*100): 263289 136002
2nd Coef(*100): 2309 1332
H2O adj(mg/l*10k): 544 502

******* Printer Test End *******

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

PRINT TEST

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 06/13/2019 Alcohol Analyzer SN 80-004956 8164.14.00 09/16 10:48

WET CAL CHECK

Test			AC	Time
01	Room	Air	0.000	10:48
02	Std.	Sol.	0.014	10:49
03	Room	Air	0.000	10:49
04	Std.	Sol.	0.014	10:50
05	Room	Air	0.000	10:51
06	Std.	Sol.	0.014	10:51
07	Room	Air	0.000	10:52

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = DR5113 Std Sol No = 201805C

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Low AC

Remarks:

Form 106-I8000

0.015 AC

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/13/2019

Alcohol Analyzer SN 80-004956 8164.14.00 09/16 10:58

WET CAL CHECK

Te	est		AC	Time
01	Room	Air	0.000	10:59
02	Std.	Sol.	0.248	11:00
03	Room	Air	0.000	11:00
04	Std.	Sol.	0.248	11:01
05	Room	Air	0.000	11:01
06	Std.	Sol.	0.249	11:02
07	Room	Air	0.000	11:03

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = DR7351 Std Sol No = 201803G

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

0,250 AC

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.14.00 09/16
06/13/2019 11:04

Test	AC	Time
01 Room Air	0.000	11:05
02 *Subject Test	INT*	11:06
03 Room Air	0.000	11:06

*Invalid Test

Interferent Detected

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150

Test = DUI Cit = INTERFERENT

Dr. Lic. = ND/DIS921456

Lot No = 34418080A2

Cyl No = 45

Expiration Date = 02/05/2021

County = 08

Oper No. = 666666

Operator Signature

nterferent Check 10AC +0.05% ACETOWE

Remarks:

Form 106-I8000

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004956
Location = TOXL 8164.14.00 09/16
06/13/2019 11:07

Test		AC	Time
01	Diagnostic	OK	11:07
02	Room Air	0.000	11:08
03	*Subject Test	RFI*	11:08
04	Room Air	0.000	11:08

*Invalid Test Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK

Dr. Lic. = ND/DIS921456

Lot No = 34418080A2

Cyl No = 45

Expiration Date = 02/05/2021

County = 08

Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/13/2019

Alcohol Analyzer SN 80-004956 8164:14.00 09/16 11:09

DRY CAL CHECK

	Τe	est		AC	Time
()1	Room	Air	0.000	11:09
()2	Std.	Gas	0.080	11:10
()3	Room	Air	0.000	11:10
()4	Std.	Gas	0.080	11:10
()5	Room	Air	0.000	11:11
()6	Std.	Gas	0.079	11:11
(7	Room	Air	0.000	11:12

Lot No = 34418080A2

Cyl No = 45

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

arks: CALIBRATION CWECK

0.080 40

Form 106-I8000

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 06/13/2019 Alcohol Analyzer SN 80-004956 8164.14.00 09/16 11:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:13
02 Std. Gas	0.079	11:13
03 Room Air	0.000	11:13
04 Std. Gas	0.079	11:14
05 Room Air	0.000	11:14
06 Std. Gas	0.080	11:15
07 Room Air	0.000	11:15

Lot No = 34418080A2

Cyl No = 45

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

Operator Signature

Remarks:

0.080AC Form 106-18000

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/13/2019

SN 80-004956 8164.14.00 09/16 11:15

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:16
02 Std. Gas	0.079	11:16
03 Room Air	0.000	11:17
04 Std. Gas	0.080	11:17
05 Room Air	0.000	11:18
06 Std. Gas	0.079	11:18
07 Room Air	0.000	11:18

Lot No = 34418080A2

Cyl No = 45

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

PALIBRATION CHECK

Operator Signature CHARLES EDER

Remarks:

0.080AC Form 106-I8000