

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004205

Location: ToxL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 451 psi Regulator: 450 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: DR3378 Lot #: 201712B Exp. Date: 12.12.19
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.250 AC  
Sim. Ser #: DR7351 Lot #: 201803G Exp. Date: 3.22.20

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR3451 Lot #: ICS 5 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 34418080 A2 Cyl #: 45 Exp. Date: 2.5.21

Test 1 <u>0.080</u>	Test 4 <u>0.079</u>	Test 7 <u>0.079</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.079</u>	Test 6 <u>0.079</u>	Test 9 <u>0.079</u>
Average <u>0.079</u>		

C. Remarks/Maintenance: ANNUAL INSPECTION,

WILL BE USED IN CLASSROOM ONLY UNTIL  
FLOW MONITOR/SENSOR CAN BE REPAIRED.

Instrument is acceptable to be used in the field.

Breath Analyst Signature

Charles E. Edie

NA

Date

6/14/2019

NA

Reviewed by

Date



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.14.00 09/16  
06/14/2019      15:51

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:52
02 Std. Sol.	0.019	15:53
03 Room Air	0.000	15:53
04 Std. Sol.	0.019	15:54
05 Room Air	0.000	15:54
06 Std. Sol.	0.020	15:55
07 Room Air	0.000	15:56

08 Sim Temp = 34.0°C

Simul Ser No = DR3378  
Std Sol No = 201712B  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

Low AC  
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.14.00 09/16  
06/14/2019      16:02

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:02
02 Std. Sol.	0.245	16:03
03 Room Air	0.000	16:04
04 Std. Sol.	0.245	16:04
05 Room Air	0.000	16:05
06 Std. Sol.	0.245	16:06
07 Room Air	0.000	16:06

08 Sim Temp = 34.0°C

Simul Ser No = DR7351  
Std Sol No = 201803G  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.250 AC

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004205  
Location = TOXL                    8164.14.00 09/16  
06/14/2019                            16:14

Test	AC	Time
01 Diagnostic	OK	16:15
02 Room Air	0.000	16:15
03 *Subject Test	RFI*	16:15
04 Room Air	0.000	16:16

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 45  
Expiration Date = 02/05/2021  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: *RFI CHECK*

*A. CEE*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.14.00 09/16  
06/14/2019      16:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:09
02 Std. Gas	0.080	16:09
03 Room Air	0.000	16:09
04 Std. Gas	0.080	16:10
05 Room Air	0.000	16:10
06 Std. Gas	0.079	16:11
07 Room Air	0.000	16:11

Lot No = 34418080A2  
Cyl No = 45  
Exp Date = 02/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.14.00 09/16  
06/14/2019      16:16

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:17
02 Std. Gas	0.079	16:17
03 Room Air	0.000	16:17
04 Std. Gas	0.080	16:18
05 Room Air	0.000	16:18
06 Std. Gas	0.079	16:19
07 Room Air	0.000	16:19

Lot No = 34418080A2  
Cyl No = 45  
Exp Date = 02/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AK

Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004205  
Location = TOXL      8164.14.00 09/16  
06/14/2019      16:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:20
02 Std. Gas	0.079	16:20
03 Room Air	0.000	16:21
04 Std. Gas	0.080	16:21
05 Room Air	0.000	16:22
06 Std. Gas	0.079	16:22
07 Room Air	0.000	16:23

Lot No = 34418080A2  
Cyl No = 45  
Exp Date = 02/05/2021  
County = 08      Oper No. = 666666

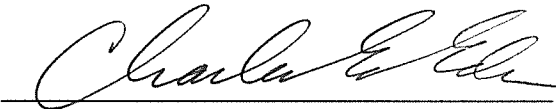

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000