Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004/99 Location: 70XL
A. General Setup and Checks: 1. 以 Diagnostic Tests Pass and Instrument in Ready Mode 2. 以 Breath tube heated 3. 反 Date, time and location code (Level 2,E). Re-set if necessary. 4. 以 Print test (Level 1,P). Sign and attach test record. 5. 反 Tank monitor (Level 3,D,G). Display and Regulator within 50 psi. a. Display: イイグ psi Regulator: グラウ psi
 B. Tests (Sign and attach test records):
Configure dry gas standard for the following tests (Level 1,S).
2. Interferent Check (Level 1,B): Known Value: O-P AC + 0.05% Acetone Sim. Ser #: DR3451 Lot #: TCS 5 Exp. Date: NA Display reads "Interferent Detect" 3. IN RFI Check (CMS Mode) IN Display reads "RFI Detect" 4. Interferent Check (CMS Mode) IN Display reads "RFI Detect" 4. Interferent Check (Level 1,C): Exp. Date: O-080 AC
C. Remarks/Maintenance: ANNUAL INSPECTION
Instrument is acceptable to be used in the field.
(leader 2 Ed 6/17/2019
Breath Analyst Signature Date '
Reviewed by Date
Teviewed by Date

02/18

Issuing Authority - CEE

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.14.00 09/16
06/17/2019 11:58

******* Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

Current Instrument Setup

Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 34418080A2 Standard Cyl #? 45 Standard Expiration? 02/05/2021 Oper No? 666666

Flow Cal. Date: 06/17/2019 Slope 593 Intercept -481610

IR Calibration Date: 06/17/2019
3um 9um

Oth Coef(*100): -17512 -19237 1st Coef(*100): 277207 136841 2nd Coef(*100): 1620 1027 H2O adj(mg/l*10k): 211 375

******* Printer Test End ********

Operator Signature CHARLES EDER

Remarks: PRINT TEST

Form 106-I8000

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/17/2019

Alcohol Analyzer SN 80-004199 8164.14.00 09/16 12:47

WET CAL CHECK

Te	est		AC	Time
01	Room	Air	0.000	12:48
02	Std.	Sol.	0.020	12:49
03	Room	Air	0.000	12:49
04	Std.	Sol.	0.020	12:50
05	Room	Air	0.000	12:50
06	Std.	Sol.	0.020	12:51
07	Room	Air	0.000	12:51

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = DR3378 Std Sol No = 201712B

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Low Ac

Remarks:

0.020 AC

Form 106-I8000

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 06/17/2019 Alcohol Analyzer SN 80-004199 8164.14.00 09/16 12:58

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:58
02 Std. Sol.	0.298	12:59
03 Room Air	0.000	13:00
04 Std. Sol.	0.299	13:01
05 Room Air	0.000	13:01
06 Std. Sol.	0.299	13:02
07 Room Air	0.000	13:02

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = DR7089 Std Sol No = 17350

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

0.300 AC Form 106-18000

HIGH AC

CMI, Inc. Intoxilyzer	Alcohol Analyzer
North Dakota Model 8000	SN 80-004199
Location = TOXL	8164.14.00 09/16
06/17/2019	13:04

Test	AC	Time
01 Room Air	0.000	13:05
02 *Subject Test	INT*	13:05
03 Room Air	0.000	13:06

*Invalid Test

Interferent Detected

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150

Test = DUI Cit = INTERFERENT

Dr. Lic. = ND/DIS921456

Lot No = 34418080A2

Cyl No = 45

Expiration Date = 02/05/2021

County = 08 Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Totalerant Clock
0.10AC + 0.05/, 4CETONE
FORM 106-18000

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.14.00 09/16
06/17/2019 13:06

Test	AC	Time
01 Diagnostic	OK	13:07
02 Room Air	0.000	13:08
03 *Subject Test	RFI*	13:08
04 Room Air	0.000	13:08

*Invalid Test Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150 Test = DUI Cit = RFI CHECK

Dr. Lic. = ND/DIS921456

Lot No = 34418080A2

Cyl No = 45

Expiration Date = 02/05/2021

County = 08

Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/17/2019

Alcohol Analyzer SN 80-004199 8164.14.00 09/16 13:09

DRY CAL CHECK

Te	est		AC	Time
01	Room	Air	0.000	13:09
02	Std.	Gas	0.080	13:10
03	Room	Air	0.000	13:10
04	Std.	Gas	0.079	13:10
05	Room	Air	0.000	13:11
06	Std.	Gas	0.080	13:11
07	Room	Air	0.000	13:12

Lot No = 34418080A2

Cyl No = 45

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

0.080 AC

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/17/2019

Alcohol Analyzer SN 80-004199 8164.14.00 09/16 13:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:13
02 Std. Gas	0.080	13:13
03 Room Air	0.000	13:14
04 Std. Gas	0.080	13:14
05 Room Air	0.000	13:15
06 Std. Gas	0.079	13:15
07 Room Air	0.000	13:16

Lot No = 34418080A2

Cyl No = 45

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

Operator Signature
CHARLES EDER

ALIBRATION CHECK

Remarks:

Form 106-I8000

0.080 AC

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 06/17/2019

DRY CAL CHECK

$T\epsilon$	est		AC	Time
01	Room	Air	0.000	13:16
02	Std.	Gas	0.079	13:17
03	Room	Air	0.000	13:17
04	Std.	Gas	0.079	13:18
05	Room	Air	0.000	13:18
06	Std.	Gas	0.080	13:19
07	Room	Air	0.000	13:19

Lot No = 34418080A2

Cyl No = 45

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

Alcohol Analyzer

8164.14.00 09/16

SN 80-004199

13:16

Operator Signature
CHARLES EDER
ALIBRATION (WECK

Remarks:

Form 106-I8000

0.080 AC