OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

	MICALIZER GOOD MOI ZOTION		
Intoxil	yzer® 8000 Serial Number: 80-004954 Location: ToxL		
Α.	General Setup and Checks: 1.		
В.	Tests (Sign and attach test records): Configure simulator for the following tests (Level 1,S). Wet Calibration Check a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC Sim. Ser #: DR 337 B Lot #: 201712 B Exp. Date: 12.12.19 b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC Sim. Ser #: DR 7350 Lot #: 201803 H Exp. Date: 3.22.20		
	Configure dry gas standard for the following tests (Level 1,S).		
	2. Interferent Check (Level 1,B): Known Value: 0-10 AC + 0.05% Acetone Sim. Ser #: DR3451 Lot #: ICS 5 Exp. Date: NA Display reads "Interferent Detect"		
C.	Remarks/Maintenance: ANNUAL INSPECTION		
Breath	trument is acceptable to be used in the field. 6.18.2019 Analyst Signature NA Date		
Revie	Reviewed by Date		

02/18 Issuing Authority - CEE

SN 80-004954

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004954
Location = TOXL 8164.14.00 09/16
06/18/2019 11:08

******** Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

Current Instrument Setup

Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Yes Display Third Digit? Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 34418080A2 Standard Cyl #? 40 Standard Expiration? 02/05/2021 Oper No? 666666

Flow Cal. Date: 08/27/2012 Slope 670 Intercept -570610

IR Calibration Date: 05/31/2017
3um 9um

Oth Coef(*100): -18765 -20238 1st Coef(*100): 256075 130805 2nd Coef(*100): 2112 1219 H2O adj(mg/l*10k): 442 407

******* Printer Test End ********

Operator Signature CHARLES EDER

Remarks: PRINT TEST

Form 106-I8000

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/18/2019

Alcohol Analyzer SN 80-004954 8164.14.00 09/16 11:09

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:09
02 Std. Sol.	0.019	11:10
03 Room Air	0.000	11:11
04 Std. Sol.	0.019	11:11
05 Room Air	0.000	11:12
06 Std. Sol.	0.019	11:13
07 Room Air	0.000	11:13

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = DR3378 Std Sol No = 201712B

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Low AC

Remarks:

0.020 Ac

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 06/18/2019

Alcohol Analyzer SN 80-004954 8164.14.00 09/16 11:14

WET CAL CHECK

Test		AC	Time
01 Room	Air	0.000	11:15
02 Std.	Sol.	0.295	11:16
03 Room	Air	0.000	11:16
04 Std.	Sol.	0.297	11:17
05 Room	Air	0.000	11:17
06 Std.	Sol.	0.297	11:18
07 Room	Air	0.000	11:19

08 Sim Temp = 34.0°C

Simul Ser No = DR7350 Std Sol No = 201803H

County = 08

Remarks:

Oper No. = 666666

Operator Signature CHARLES EDER

76H AC

0.300 A

Form 106-I8000

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004954
Location = TOXL 8164.14.00 09/16
06/18/2019 11:20

Test	AC	Time
01 Room Air	0.000	11:21
02 *Subject Test	INT*	11:22
03 Room Air	0.000	11:22

*Invalid Test

Interferent Detected

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT

Dr. Lic. = ND/DIS921456

Lot No = 34418080A2

Cyl No = 40

Expiration Date = 02/05/2021

County = 08 Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Form 106-18000

CMI, Inc. Intoxilyzer	Alcohol Analyzer
North Dakota Model 8000	SN 80-004954
Location = TOXL	8164.14.00 09/16
06/18/2019	11:23

Tes	t	AC	Time
01 D	iagnostic	OK	11:23
02 R	oom Air	0.000	11:24
03 *	Subject Test	RFI*	11:24
04 R	oom Air	0.000	11:24

^{*}Invalid Test Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK

Dr. Lic. = ND/DIS921456

Lot No = 34418080A2

Cyl No = 40

Expiration Date = 02/05/2021

County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 06/18/2019 Alcohol Analyzer SN 80-004954 8164.14.00 09/16 11:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:25
02 Std. Gas	0.080	11:25
03 Room Air	0.000	11:26
04 Std. Gas	0.081	11:26
05 Room Air	0.000	11:27
06 Std. Gas	0.080	11:27
07 Room Air	0.000	11:28

Lot No = 34418080A2

Cyl No = 40

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

0.080 AC

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/18/2019

Alcohol Analyzer SN 80-004954 8164.14.00 09/16 11:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:34
02 Std. Gas	0.081	11:34
03 Room Air	0.000	11:35
04 Std. Gas	0.081	11:35
05 Room Air	0.000	11:35
06 Std. Gas	0.080	11:36
07 Room Air	0.000	11:36

Lot No = 34418080A2

Cyl No = 40

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

0.080 AC

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/18/2019

Alcohol Analyzer SN 80-004954 8164.14.00 09/16 11:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:37
02 Std. Gas	0.081	11:37
03 Room Air	0.000	11:38
04 Std. Gas	0.080	11:38
05 Room Air	0.000	11:39
06 Std. Gas	0.080	11:39
07 Room Air	0.000	11:40

Lot No = 34418080A2

Cyl No = 40

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

ALIBRATION CHECK

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

0-080AC