Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

ntoxilyzer® 8000 Serial Number: 80-004/96 Location: ToxL
 General Setup and Checks: 1.
Tests (Sign and attach test records): ☐ Configure simulator for the following tests (Level 1,S). 1. Wet Calibration Check a. ☐ Low AC (Level 1,C): Known Value ≤ 0.03 AC: ☐ -0.20 AC Sim. Ser #: ☐ DR3378 Lot #: ☐ 0.7128 Exp. Date: ☐ 2.12.19 b. ☐ High AC (Level 1,C): Known Value ≥ 0.25 AC: ☐ 0.300 AC Sim. Ser #: ☐ R7350 Lot #: ☐ 201803 H Exp. Date: ☐ 3.22.20
🕅 Configure dry gas standard for the following tests (Level 1,S).
2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone Sim. Ser #: DR3451 Lot #: TC5 5 Exp. Date: NA Display reads "Interferent Detect" 3. Interferent Detect" 3. Interferent Check (CMS Mode)
Remarks/Maintenance: ANNUAL INSPECTION, REPLACED AND TRAINED EXTERNAL BATTERY, TANK SENSOR CALIBRATION
Instrument is acceptable to be used in the field. 6-19.2019
eath Analyst Signature NA Date
eviewed by Date

02/18 Issuing Authority - CEE

SN 80-004196

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.14.00 09/16
06/19/2019 11:07

******** Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

Current Instrument Setup

Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 34418080A2 Standard Cyl #? 40 Standard Expiration? 02/05/2021 Oper No? 666666

Flow Cal. Date: 06/04/2009 Slope 727

Intercept -600779

IR Calibration Date: 06/02/2015
3um 9um

Oth Coef(*100): -19524 -13707
1st Coef(*100): 263393 129371
2nd Coef(*100): 2924 1442
H2O adj(mq/l*10k): 392 311

******* Printer Test End *******

Operator Signature CHARLES EDER

Remarks: TRINT TEST

Form 106-I8000

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/19/2019

Alcohol Analyzer SN 80-004196 8164.14.00 09/16 11:12

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:13
02 Std. Sol.	0.019	11:14
03 Room Air	0.000	11:14
04 Std. Sol.	0.019	11:15
05 Room Air	0.000	11:16
06 Std. Sol.	0.019	11:16
07 Room Air	0.000	11:17

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = DR3378 Std Sol No = 201712B

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

LOW AC

Remarks:

Form 106-I8000

0.020Ac

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 06/19/2019 Alcohol Analyzer SN 80-004196 8164.14.00 09/16 11:18

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:18
02 Std. Sol.	0.297	11:19
03 Room Air	0.000	11:19
04 Std. Sol.	0.300	11:20
05 Room Air	0.000	11:21
06 Std. Sol.	0.300	11:22
07 Room Air	0.000	11:22

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = DR7350 Std Sol No = 201803H

County = 08

Oper No. = 666666

Operator Signature

CHARLES EDER

Remarks:

Form 106-I8000

0.300 4

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.14.00 09/16
06/19/2019 11:24

Test	AC	Time
01 Room Air	0.000	11:24
02 *Subject Test	INT*	11:25
03 Room Air	0.000	11:26

*Invalid Test

Interferent Detected

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150

Test = DUI Cit = INTERFERENT

Dr. Lic. = ND/DIS921456

Lot No = 34418080A2

Cyl No = 40

Expiration Date = 02/05/2021

County = 08 Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Interferent Check
0.10AC + 0.05! ACETONE
Form 106-18000

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004196
Location = TOXL 8164.14.00 09/16
06/19/2019 11:26

Test	AC	Time
01 Diagnostic	OK	11:27
02 Room Air	0.000	11:27
03 *Subject Test	RFI*	11:27
04 Room Air	0.000	11:28

*Invalid Test Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150 Test = DUI Cit = RFI CHECK

Dr. Lic. = ND/DIS921456

Lot No = 34418080A2

Cyl No = 40

Expiration Date = 02/05/2021

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/19/2019

Alcohol Analyzer SN 80-004196 8164.14.00 09/16 11:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:28
02 Std. Gas	0.080	11:29
03 Room Air	0.000	11:29
04 Std. Gas	0.079	11:30
05 Room Air	0.000	11:30
06 Std. Gas	0.080	11:31
07 Room Air	0.000	11:31

Lot No = 34418080A2

Cyl No = 40

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

ALIBRATION CHECK

0.080 AC

Form 106-I8000

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/19/2019

SN 80-004196 8164.14.00 09/16 11:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:32
02 Std. Gas	0.080	11:32
03 Room Air	0.000	11:33
04 Std. Gas	0.080	11:33
05 Room Air	0.000	11:34
06 Std. Gas	0.079	11:34
07 Room Air	0.000	11:35

Lot No = 34418080A2

Cyl No = 40

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

Operator Signature
CHARLES EDER

CALIBRATION CAECK

Remarks:

Form 106-I8000

0.080 AC

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 06/19/2019 Alcohol Analyzer SN 80-004196 8164.14.00 09/16 11:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:36
02 Std. Gas	0.080	11:36
03 Room Air	0.000	11:37
04 Std. Gas	0.080	11:37
05 Room Air	0.000	11:37
06 Std. Gas	0.080	11:38
07 Room Air	0.000	11:38

Lot No = 34418080A2

Cyl No = 40

Exp Date = 02/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

0-080 AC