

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-005362

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 426 psi Regulator: 425 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR337B Lot #: 18020 Exp. Date: 1.9.20
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR7350 Lot #: 2018034 Exp. Date: 3.22.20

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: FCS5 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 34418080A2 Cyl #: 40 Exp. Date: 2.5.21

Test 1	<u>0.079</u>	Test 4	<u>0.079</u>	Test 7	<u>0.080</u>
Test 2	<u>0.080</u>	Test 5	<u>0.080</u>	Test 8	<u>0.080</u>
Test 3	<u>0.079</u>	Test 6	<u>0.080</u>	Test 9	<u>0.079</u>
Average <u>0.080</u>					

C. Remarks/Maintenance: ANNUAL INSPECTION, REPLACED AND TRAINED EXTERNAL BATTERY

Instrument is acceptable to be used in the field.

Charles E. Ehr
Breath Analyst Signature

6.25.2019
Date

NA

NA

Reviewed by

Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005362
Location = TOXL 8164.14.00 09/16
06/25/2019 14:38

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%\$^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%\$^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 03417080A3
Standard Cyl #? 27
Standard Expiration? 03/05/2019
Oper No? 666666

Flow Cal. Date: 04/04/2018
Slope 623
Intercept -543872

IR Calibration Date: 04/07/2015
 3um 9um

0th Coef(*100): -22003 -17935
1st Coef(*100): 254570 134851
2nd Coef(*100): 4320 2032
H2O adj (mg/l*10k): 466 435

***** Printer Test End *****

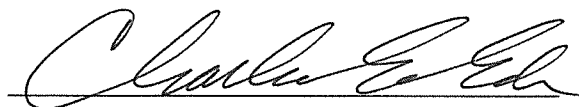

Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005362
Location = TOXL 8164.14.00 09/16
06/25/2019 14:45

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:45
02 Std. Sol.	0.017	14:50
03 Room Air	0.000	14:51
04 Std. Sol.	0.018	14:51
05 Room Air	0.000	14:52
06 Std. Sol.	0.018	14:53
07 Room Air	0.000	14:53

08 Sim Temp = 34.0°C

Simul Ser No = DR3378
Std Sol No = 18020
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-18000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005362
Location = TOXL 8164.14.00 09/16
06/25/2019 15:08

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:08
02 Std. Sol.	0.297	15:09
03 Room Air	0.000	15:10
04 Std. Sol.	0.298	15:11
05 Room Air	0.000	15:11
06 Std. Sol.	0.298	15:12
07 Room Air	0.000	15:12

08 Sim Temp = 34.0°C

Simul Ser No = DR7350
Std Sol No = 201803H
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005362
Location = TOXL 8164.14.00 09/16
06/25/2019 15:14

Test	AC	Time
01 Room Air	0.000	15:15
02 *Subject Test	INT*	15:15
03 Room Air	0.000	15:16

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 40
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005362
Location = TOXL 8164.14.00 09/16
06/25/2019 15:16

Test	AC	Time
01 Diagnostic	OK	15:17
02 Room Air	0.000	15:17
03 *Subject Test	RFI*	15:17
04 Room Air	0.000	15:18

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 40
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005362
Location = TOXL 8164.14.00 09/16
06/25/2019 15:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:18
02 Std. Gas	0.079	15:19
03 Room Air	0.000	15:19
04 Std. Gas	0.080	15:20
05 Room Air	0.000	15:20
06 Std. Gas	0.079	15:21
07 Room Air	0.000	15:21

Lot No = 34418080A2
Cyl No = 40
Exp Date = 02/05/2021
County = 08 Oper No. = 666666

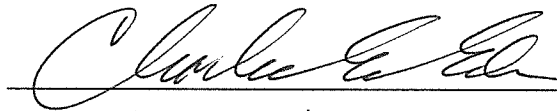

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005362
Location = TOXL 8164.14.00 09/16
06/25/2019 15:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:22
02 Std. Gas	0.079	15:22
03 Room Air	0.000	15:23
04 Std. Gas	0.080	15:23
05 Room Air	0.000	15:24
06 Std. Gas	0.080	15:24
07 Room Air	0.000	15:25

Lot No = 34418080A2
Cyl No = 40
Exp Date = 02/05/2021
County = 08 Oper No. = 666666


Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005362
Location = TOXL 8164.14.00 09/16
06/25/2019 15:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:25
02 Std. Gas	0.080	15:26
03 Room Air	0.000	15:26
04 Std. Gas	0.080	15:27
05 Room Air	0.000	15:27
06 Std. Gas	0.079	15:28
07 Room Air	0.000	15:28

Lot No = 34418080A2
Cyl No = 40
Exp Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000