

**INTOXILYZER® 8000 INSPECTION**

Intoxilyzer® 8000 Serial Number: 80-004200

Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 463 psi Regulator: 450 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
    - a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: DR3378 Lot #: 18020 Exp. Date: 1.9.20
    - b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: DR7089 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR3451 Lot #: ICS5 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 34418080A2 Cyl #: 40 Exp. Date: 2.5.21

Test 1	<u>0.080</u>	Test 4	<u>0.079</u>	Test 7	<u>0.080</u>
Test 2	<u>0.080</u>	Test 5	<u>0.079</u>	Test 8	<u>0.080</u>
Test 3	<u>0.080</u>	Test 6	<u>0.079</u>	Test 9	<u>0.079</u>
Average	<u>0.080</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION, REPLACED BATTERY PACK,  
REPLACED AND TRAINED EXTERNAL BATTERY.  
BATTERY STATUS IS NOW BEING DISPLAYED.

Instrument is acceptable to be used in the field.

Charles E. Ed  
Breath Analyst Signature

6.25.2019  
Date

NA

NA

Reviewed by

Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004200  
Location = TOXL                 8164.14.00 09/16  
06/25/2019                        11:11

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?


Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            34418080A2  
Standard Cyl #?            40  
Standard Expiration?       02/05/2021  
Oper No?                    666666  
  
Flow Cal. Date:             06/25/2019  
Slope                        614  
Intercept                   -551153

IR Calibration Date:        06/25/2019  
                                    3um                      9um

-----  
0th Coef(\*100):            -18792                    -22164  
1st Coef(\*100):            267001                    140777  
2nd Coef(\*100):            1903                      1083  
H2O adj(mg/l\*10k):        403                        445

\*\*\*\*\* Printer Test End \*\*\*\*\*

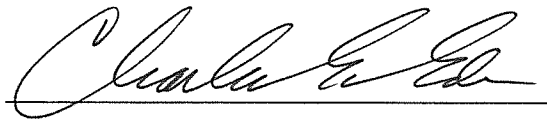

Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.14.00 09/16  
06/25/2019      11:13

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:14
02 Std. Sol.	0.019	11:14
03 Room Air	0.000	11:15
04 Std. Sol.	0.020	11:16
05 Room Air	0.000	11:16
06 Std. Sol.	0.020	11:17
07 Room Air	0.000	11:17


08 Sim Temp = 34.0°C

Simul Ser No = DR3378

Std Sol No = 18020

County = 08

Oper No. = 666666

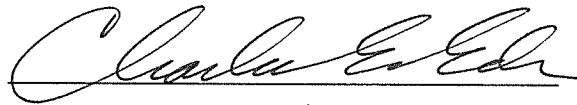

Operator Signature

CHARLES EDER

Remarks:

Low AC  
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.14.00 09/16  
06/25/2019      11:24

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:24
02 Std. Sol.	0.295	11:25
03 Room Air	0.000	11:25
04 Std. Sol.	0.295	11:26
05 Room Air	0.000	11:27
06 Std. Sol.	0.296	11:27
07 Room Air	0.000	11:28

08 Sim Temp = 34.0°C

Simul Ser No = DR7089

Std Sol No = 17350

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

HIGH AC  
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.14.00 09/16  
06/25/2019      11:31

Test	AC	Time
01 Room Air	0.000	11:31
02 *Subject Test	INT*	11:32
03 Room Air	0.000	11:32

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 40  
Expiration Date = 02/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10 AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-004200  
Location = TOXL                    8164.14.00 09/16  
06/25/2019                            11:33

Test	AC	Time
01 Diagnostic	OK	11:34
02 Room Air	0.000	11:34
03 *Subject Test	RFI*	11:34
04 Room Air	0.000	11:35

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 40  
Expiration Date = 02/05/2021  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: *RFI CHECK*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.14.00 09/16  
06/25/2019      11:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:36
02 Std. Gas	0.080	11:36
03 Room Air	0.000	11:37
04 Std. Gas	0.080	11:37
05 Room Air	0.000	11:38
06 Std. Gas	0.080	11:38
07 Room Air	0.000	11:38

Lot No = 34418080A2  
Cyl No = 40  
Exp Date = 02/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.14.00 09/16  
06/25/2019      11:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:40
02 Std. Gas	0.079	11:40
03 Room Air	0.000	11:40
04 Std. Gas	0.079	11:41
05 Room Air	0.000	11:41
06 Std. Gas	0.079	11:42
07 Room Air	0.000	11:42

Lot No = 34418080A2  
Cyl No = 40  
Exp Date = 02/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004200  
Location = TOXL      8164.14.00 09/16  
06/25/2019      11:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:43
02 Std. Gas	0.080	11:44
03 Room Air	0.000	11:44
04 Std. Gas	0.080	11:44
05 Room Air	0.000	11:45
06 Std. Gas	0.079	11:45
07 Room Air	0.000	11:46

Lot No = 34418080A2  
Cyl No = 40  
Exp Date = 02/05/2021  
County = 08      Oper No. = 666666

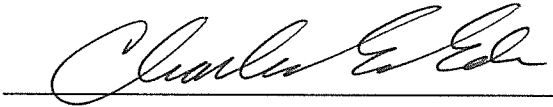

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000