



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION
Toxicology Section/Breath Alcohol Program
SFN 59281 (03-2014)

Serial Number 80-005940	Instrument Location NEWT
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u>update at Crime Lab and Returned</u>	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): W/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date 6-25-19
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date 6.26.19

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *[Signature]* Certified Date: **6.26.19**

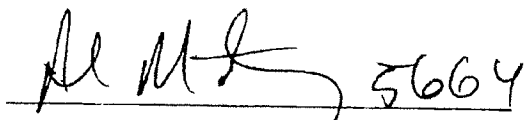
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005940
Location = NEWT 8164.14.00 09/16
06/25/2019 14:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:55
02 Std. Gas	0.077	14:56
03 Room Air	0.000	14:56
04 Std. Gas	0.078	14:57
05 Room Air	0.000	14:57
06 Std. Gas	0.077	14:58
07 Room Air	0.000	14:58

Lot No = 34917080A3
Cyl No = 19
Exp Date = 02/05/2020
County = 60 Oper No. = 131739

 5664

Operator Signature
ANDRES MARTINEZ

Remarks: 

Form 106-I8000

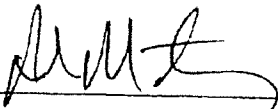
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-005940
Location = NEWT 8164.14.00 09/16
06/25/2019 14:59

Test	AC	Time
01 Room Air	0.000	15:00
02 *Subject Test	RFI*	15:01
03 Room Air	0.000	15:01

*Invalid Test
Inhibited - RFI

Sub Name = TEST, TEST TEST
Sub DOB = 09/09/1990
Sub Sex = Male Weight = 232
Test = OHV Cit = NA
Dr. Lic. = ND/NA
Lot No = 34917080A3
Cyl No = 19
Expiration Date = 02/05/2020
County = 60 Oper No. = 131739

 5604
Operator Signature
ANDRES MARTINEZ

Remarks:

Install

Form 106-18000