

INTOXILYZER® 8000 INITIAL INSPECTION

Intoxilyzer® 8000 Serial Number: 80-003071

**A. GENERAL Setup and Checks:**

- 1.  Power On
  - 5 min. countdown during initial warm-up
  - 1 min. countdown to begin test after standby mode
- 2.  Display Intensity (Ctrl + ↑ or ↓)
- 3. Password Security:
  - Level 1 (Operator)  Level 2 (Field Inspector)  Level 3 (Lab Personnel)
- 4.  Printer Setup (Level 3,S,P)
- 5.  Forms Load (CMI, Inc., CoBrA Software)

**B. LEVEL 1 Setup and Checks:**

- 1.  Internal Printer Test and Paper Feed (Level 1,P). Attach record.
- 2.  External Printer Test (Level 1,P). Attach record.
- 3. Barcode Reader (Level 1,B):
  - Operator ID  ND License  R key: Refusal. Attach record.

**C. LEVEL 2 Setup and Checks:**

- 1.  Continuous Air Blank (Level 2,A)
- 2.  Diagnostic (Level 2,D). Attach record.
- 3.  Prelim Data Entry (Level 2,E)
- 4. Ser Number/Version Info (Level 2,V):
  - Inst Ser#: 80-003071 DSP Ver: 1362.14
  - Core Ver: 8164.14 CPLD Ver: 7

**D. LEVEL 3 Setup and Checks:**

- 1. DVM Monitor (Level 3,D,D): 3: 13569 s: 1 c: 47.00  
9: 13754 s: 3 B: 45 0  
(RFI indicator with keyed radio): 1
- 2. Function (F) Keys:  F5 – Tone  F9 - Fan  
 F6 – Green LED  F10 – Pump  
 F7 – Red LED  F11 – Solenoid
- 3. Atmospheric Monitor (Level 3,D,A): Atmo Sensor, P: 952 CF: 1.06
- 4. Flow Monitor (Level 3,D,F): Flow Sensor, R: 154 F: 0.000 L/s

5. Temperature Monitor (Level 3,D,T): C: 47.0 A: 46.0 I: 0.0  
B: 44.9 V: 0-0
6. Tank Monitor (Level 3,D,G): P (display): 419 psi P (gauge): 425 psi
7.  General Setup (Level 3,S,G)
8.  Configure Start Test (Level 3,S,T)
9.  Network (Level 3,S,N)

**E. TESTS (attach all test records):**

**Configure wet bath simulator standards before running tests (Level 1,S):**

1. Zero AC (Level 1,C):  
Sim ser #: DR3841 Lot #: MILLI Q H<sub>2</sub>O Exp date: NA  
Known value: 0.000 AC  
Results: 0.000 AC 0.000 AC 0.000 AC
2. Low AC (Level 1,C):  
Sim ser #: DR3378 Lot #: 18020 Exp date: 1.9.20  
Known value: 0.020 AC  
Results: 0.019 AC 0.018 AC 0.018 AC
3. High AC (Level 1,C):  
Sim ser #: DR7089 Lot #: 17350 Exp date: 10.11.19  
Known value: 0.300 AC 2 CEE  
Results: 0.292 AC 0.295 AC 0.294 AC

**Configure dry gas standard before running tests (Level 1,S):**

4. Ethanol + 0.05% Acetone (Level 1,B):  
Sim ser #: DR3451 Lot #: ICS 5 Exp date: NA  
Known value: 0.10AC + 0.05% ACETONE  
 Display reads "Interferent Detect"
5. Acetone (Level 1,B):  
Sim ser #: DR 5188 Lot #: ACE 1 Exp date: NA  
Known value: 0.05% ACETONE  
 Display reads "Interferent Detect"
6. Methanol (Level 1,B):  
Sim ser #: DR3455 Lot #: MET 2 Exp date: NA  
Known value: 0.10% METHANOL  
 Display reads "Interferent Detect"
7. Isopropanol (Level 1,B):  
Sim ser #: DR5189 Lot #: IPA 2 Exp date: NA  
Known value: 0.10% ISOPROPRANOL  
 Display reads "Interferent Detect"

- 8.  Ambient Fail (Level 1,B): Display and test record read correct.
- 9.  Invalid Sample (Level 1,B): Display and test record read correct.
- 10.  Sequence Aborted (Level 1,B): Display and test record read correct.
- 11.  ABA (Level 1,B): Test record reads correct.
- 12.  Calibration Check Out of Tolerance (CMS mode): Display and test record read correct.
- 13.  RFI Detected (B,C, or CMS mode): Display and test record read correct.
- 14.  Tank Pressure Below Minimum (CMS mode): Display reads correct.
- 15.  Deficient Samples (CMS mode): Display and test record read correct.
- 16.  Improper Sample (CMS mode): Display and test record read correct.
- 17.  Difference Too Great (CMS mode): Display and test record read correct.
- 18.  CMS (CMS mode): Test record reads correct.
- 19. Stability Test (Level 3,T,S):

Ave: 0.0791 Std Dev: 0.0005 Rel Std Dev: 0.65%

Maintenance/Repairs/Notes: INITIAL INSPECTION DUE TO OPENING  
UP INSTRUMENT TO REPLACE BATTERY  
FOR REAL TIME CLOCK (RTC).

Instrument is acceptable to be used in the field.

Charles E. Edwards  
Field Inspector's Signature

6-27-19  
Date

NA  
Reviewed by

NA  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer  
North Dakota Model 8000 SN 80-003071  
Location = TOXL 8164.14.00 09/16  
06/27/2019 13:46

\*\*\*\*\* Printer Test \*\*\*\*\*

12 char/line  
abcdefghijkl  
ABCDEFGHIJKL

16 char per line  
abcdefghijklmnop  
ABCDEFGHIJKLMN

21 chars per line  
abcdefghijklmnopqrstu  
ABCDEFGHIJKLMNOPQR

24 characters per line  
abcdefghijklmnopqrstuwx  
ABCDEFGHIJKLMNOPQRSTUWX

32 characters per line  
abcdefghijklmnopqrstuwx123456  
ABCDEFGHIJKLMNOPQRSTUWXYZ123456

42 characters per line  
abcdefghijklmnopqrstuwx1234567890abcdef  
ABCDEFGHIJKLMNOPQRSTUWXYZ1234567890ABCDEFGHI

Current Instrument Setup

Data Entry Mode: Enabled  
Start Test Sequence: DABACABA  
Display Prelim Rslt? Yes  
Display Third Digit? Yes  
Inhib Printer(Y/N)? No  
Display Volume? No  
Disable On Memfull? Yes  
# of Print Copies? 1  
Select Std (D/W/I)? Dry  
Standard Value? 0.080  
Standard Lot #? 20016080A2  
Standard Cyl #? 48  
Standard Expiration? 09/05/2018  
Oper No? 666666

Flow Cal. Date: 07/09/2010  
Slope 806  
Intercept -994737

IR Calibration Date: 04/10/2015  
3um 9um  
-----  
0th Coef(\*100): -11328 -10938  
1st Coef(\*100): 268087 134278  
2nd Coef(\*100): 3093 1368  
H2O adj(ng/l\*10K): 242 233

\*\*\*\*\* Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks: **INTERNAL  
PRINT  
TEST**  
Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-003071  
Location = TOXL                    8164.14.00 09/16  
06/27/2019    13:48

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            20016080A2  
Standard Cyl #?            48  
Standard Expiration?       09/05/2018  
Oper No?                    666666

Flow Cal. Date:            07/09/2010  
Slope                        806  
Intercept                   -994737

IR Calibration Date:       04/10/2015

|                    | 3um    | 9um    |
|--------------------|--------|--------|
| 0th Coef(*100):    | -11328 | -10938 |
| 1st Coef(*100):    | 268087 | 134278 |
| 2nd Coef(*100):    | 3093   | 1368   |
| H2O adj(mg/l*10k): | 242    | 233    |

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:    *EXTERNAL PRINT*  
*TEST*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
06/27/2019      13:51

| Test             | AC    | Time  |
|------------------|-------|-------|
| 01 Room Air      | 0.000 | 13:51 |
| 02 *Subject Test | REF*  | 13:51 |
| 03 Room Air      | 0.000 | 13:52 |

\*Subject Test Refused

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = R KEY REFUSAL  
Dr. Lic. = ND/DIS921456  
Lot No = 20016080A2  
Cyl No = 48  
Expiration Date = 09/05/2018  
County = 08      Oper No. = 666666

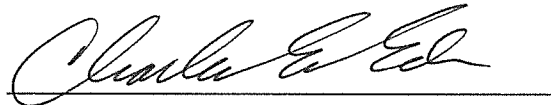

Operator Signature  
CHARLES EDER

Remarks:      *R Key REFUSAL*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
06/27/2019      13:53

DIAGNOSTIC

|                           |      |
|---------------------------|------|
| Voltage/Current Test      | Pass |
| RAM Test                  | Pass |
| EEPROM Checksum Test      | Pass |
| Real Time Clock Test      | Pass |
| DSP Test                  | Pass |
| Analytical Stability Test | Pass |
| Modem Test                | Pass |
| Temp Regulation Test      | Pass |

Oper No. = 666666

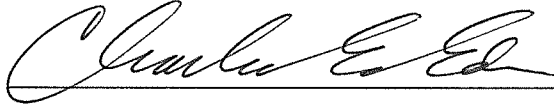

Operator Signature  
CHARLES EDER

Remarks:

DIAGNOSTIC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
06/27/2019      14:05

WET CAL CHECK

| Test         | AC    | Time  |
|--------------|-------|-------|
| 01 Room Air  | 0.000 | 14:06 |
| 02 Std. Sol. | 0.000 | 14:07 |
| 03 Room Air  | 0.000 | 14:07 |
| 04 Std. Sol. | 0.000 | 14:08 |
| 05 Room Air  | 0.000 | 14:08 |
| 06 Std. Sol. | 0.000 | 14:09 |
| 07 Room Air  | 0.000 | 14:09 |

08 Sim Temp = 34.0°C

Simul Ser No = DR3841  
Std Sol No = MILLIQ H2O  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *ZERO AC*  
*0.000 AC*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
06/27/2019      14:11

WET CAL CHECK

| Test         | AC    | Time  |
|--------------|-------|-------|
| 01 Room Air  | 0.000 | 14:11 |
| 02 Std. Sol. | 0.019 | 14:12 |
| 03 Room Air  | 0.000 | 14:13 |
| 04 Std. Sol. | 0.018 | 14:13 |
| 05 Room Air  | 0.000 | 14:14 |
| 06 Std. Sol. | 0.018 | 14:15 |
| 07 Room Air  | 0.000 | 14:15 |

08 Sim Temp = 34.0°C

Simul Ser No = DR3378  
Std Sol No = 18020  
County = 08      Oper No. = 666666

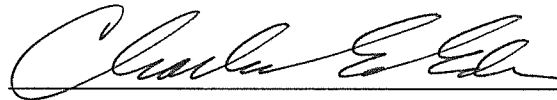

Operator Signature  
CHARLES EDER

Remarks:

Low AC  
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
06/27/2019      14:28

WET CAL CHECK

| Test         | AC    | Time  |
|--------------|-------|-------|
| 01 Room Air  | 0.000 | 14:29 |
| 02 Std. Sol. | 0.292 | 14:30 |
| 03 Room Air  | 0.000 | 14:30 |
| 04 Std. Sol. | 0.295 | 14:31 |
| 05 Room Air  | 0.000 | 14:31 |
| 06 Std. Sol. | 0.294 | 14:32 |
| 07 Room Air  | 0.000 | 14:33 |

08 Sim Temp = 34.0°C

Simul Ser No = DR7089

Std Sol No = 17350

County = 08

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003071  
Location = TOXL                    8164.14.00 09/16  
06/27/2019                            14:49

| Test             | AC    | Time  |
|------------------|-------|-------|
| 01 Room Air      | 0.000 | 14:50 |
| 02 *Subject Test | INT*  | 14:50 |
| 03 Room Air      | 0.000 | 14:51 |

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 40  
Expiration Date = 02/05/2021  
County = 08                            Oper No. = 666666

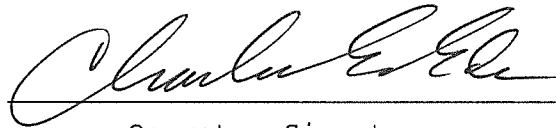

Operator Signature  
CHARLES EDER

Remarks: *Interferent*  
*0.10AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003071  
Location = TOXL                    8164.14.00 09/16  
06/27/2019                            14:52

| Test             | AC    | Time  |
|------------------|-------|-------|
| 01 Room Air      | 0.000 | 14:53 |
| 02 *Subject Test | INT*  | 14:53 |
| 03 Room Air      | 0.000 | 14:54 |

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 40  
Expiration Date = 02/05/2021  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent*  
*0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003071  
Location = TOXL                    8164.14.00 09/16  
06/27/2019                            14:55

| Test             | AC    | Time  |
|------------------|-------|-------|
| 01 Room Air      | 0.000 | 14:56 |
| 02 *Subject Test | INT*  | 14:56 |
| 03 Room Air      | 0.000 | 14:57 |

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 40  
Expiration Date = 02/05/2021  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent*  
*0.10% METHANOL*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003071  
Location = TOXL                    8164.14.00 09/16  
06/27/2019                            14:58

| Test             | AC    | Time  |
|------------------|-------|-------|
| 01 Room Air      | 0.000 | 14:59 |
| 02 *Subject Test | INT*  | 14:59 |
| 03 Room Air      | 0.000 | 15:00 |

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 40  
Expiration Date = 02/05/2021  
County = 08                            Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent*  
*0.10% ISOPROPANOL*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
06/27/2019      15:05

| Test        | AC    | Time  |
|-------------|-------|-------|
| 01 Room Air | AMB*  | 15:06 |
| 02 Room Air | 0.000 | 15:07 |

\*Invalid Test  
Check Ambient Conditions

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = AMBIENT FAIL  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 40  
Expiration Date = 02/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *AMBIENT FAIL*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003071  
Location = TOXL                    8164.14.00 09/16  
06/27/2019                            15:07

| Test              | AC    | Time  |
|-------------------|-------|-------|
| 01 Room Air       | 0.000 | 15:08 |
| 02 Invalid Sample | X.XXX | 15:08 |
| 03 Room Air       | 0.000 | 15:09 |

\*Invalid Test - Mouth Alcohol

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INVALID SAMPLE  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 40  
Expiration Date = 02/05/2021  
County = 08                            Oper No. = 666666

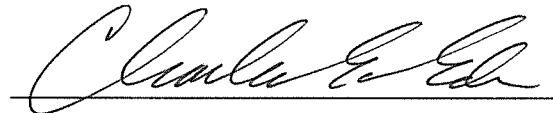

Operator Signature  
CHARLES EDER

Remarks:                    *INVALID SAMPLE*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003071  
Location = TOXL                 8164.14.00 09/16  
06/27/2019                        15:13

| Test             | AC    | Time  |
|------------------|-------|-------|
| 01 Room Air      | 0.000 | 15:14 |
| 02 *Subject Test | ABT*  | 15:14 |
| 03 Room Air      | 0.000 | 15:15 |

\*Sequence Aborted

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                 Weight = 150  
Test = DUI                        Cit = SEQUENCE ABORT  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 40  
Expiration Date = 02/05/2021  
County = 08                        Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *SEQUENCE ABORTED*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
06/27/2019      15:16

| Test              | AC    | Time  |
|-------------------|-------|-------|
| 01 Room Air       | 0.000 | 15:17 |
| 02 Subject Test 1 | 0.081 | 15:17 |
| 03 Room Air       | 0.000 | 15:18 |
| 04 Reported AC    | 0.081 | 15:17 |

No RFI Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = ABA MODE  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 40  
Expiration Date = 02/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *ABA MODE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
06/27/2019      15:18

| Test              | AC     | Time  |
|-------------------|--------|-------|
| 01 Diagnostic     | OK     | 15:19 |
| 02 Room Air       | 0.000  | 15:20 |
| 03 Subject Test 1 | 0.081  | 15:20 |
| 04 Room Air       | 0.000  | 15:23 |
| 05 Std. Gas       | 0.087* | 15:24 |
| 06 Room Air       | 0.000  | 15:24 |

\*Cal Check Out of Tolerance

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = CAL CHK OUT TOL  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 40  
Expiration Date = 02/05/2021  
County = 08      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

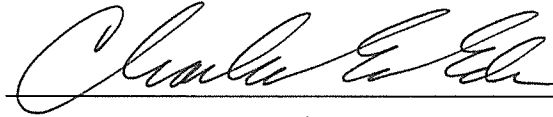

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
OUT OF TOLERANCE

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
06/27/2019      15:25

| Test              | AC    | Time  |
|-------------------|-------|-------|
| 01 Diagnostic     | OK    | 15:25 |
| 02 Room Air       | 0.000 | 15:26 |
| 03 Subject Test 1 | 0.081 | 15:26 |
| 04 Room Air       | 0.000 | 15:29 |
| 05 Std. Gas       | 0.079 | 15:30 |
| 06 Room Air       | 0.000 | 15:31 |
| 07 *Subject Test  | RFI*  | 15:31 |
| 08 Room Air       | 0.000 | 15:32 |

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = RFI DETECT  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 40  
Expiration Date = 02/05/2021  
County = 08      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: *RFI DETECTED*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003071  
Location = TOXL                    8164.14.00 09/16  
06/27/2019                            15:33

| Test             | AC     | Time  |
|------------------|--------|-------|
| 01 Diagnostic    | OK     | 15:34 |
| 02 Room Air      | 0.000  | 15:34 |
| 03 *Subject Test | 0.000* | 15:37 |
| 04 Room Air      | 0.000  | 15:40 |
| 05 Std. Gas      | 0.078  | 15:41 |
| 06 Room Air      | 0.000  | 15:42 |
| 07 *Subject Test | 0.000* | 15:45 |
| 08 Room Air      | 0.000  | 15:46 |

No RFI Detected

\*Deficient Sample - Value Printed was  
Highest Obtained

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = DEFICIENT SAMP  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 40  
Expiration Date = 02/05/2021  
County = 08                            Oper No. = 666666

20 minute waiting period ascertained? Y  
Clean Mouthpiece used and disposed? Y  
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the  
instructions displayed by the Intoxilyzer  
in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: DEFICIENT  
SAMPLES

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003071  
Location = TOXL                    8164.14.00 09/16  
06/27/2019                            15:47

| Test          | AC    | Time  |
|---------------|-------|-------|
| 01 Diagnostic | OK    | 15:48 |
| 02 Room Air   | 0.000 | 15:48 |
| 03 Reference  | IPS*  | 15:49 |
| 04 Room Air   | 0.000 | 15:49 |

\*Improper Sample

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = IMPROPER SAMPLE  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 40  
Expiration Date = 02/05/2021  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: *IMPROPER SAMPLE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-003071  
Location = TOXL      8164.14.00 09/16  
06/27/2019      15:50

| Test              | AC     | Time  |
|-------------------|--------|-------|
| 01 Diagnostic     | OK     | 15:51 |
| 02 Room Air       | 0.000  | 15:51 |
| 03 Subject Test 1 | 0.149* | 15:52 |
| 04 Room Air       | 0.000  | 15:54 |
| 05 Std. Gas       | 0.080  | 15:55 |
| 06 Room Air       | 0.000  | 15:57 |
| 07 Subject Test 2 | 0.098* | 15:57 |
| 08 Room Air       | 0.000  | 15:58 |

\*Difference Too Great

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = DIFF TOO GREAT  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 40  
Expiration Date = 02/05/2021  
County = 08      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

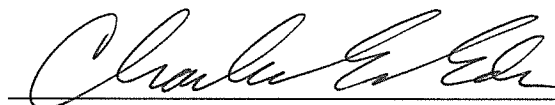

Operator Signature  
CHARLES EDER

Remarks: *DIFFERENCE  
TOO GREAT*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-003071  
Location = TOXL                    8164.14.00 09/16  
06/27/2019                            15:58

| Test              | AC    | Time  |
|-------------------|-------|-------|
| 01 Diagnostic     | OK    | 15:59 |
| 02 Room Air       | 0.000 | 16:00 |
| 03 Subject Test 1 | 0.098 | 16:00 |
| 04 Room Air       | 0.000 | 16:03 |
| 05 Std. Gas       | 0.078 | 16:04 |
| 06 Room Air       | 0.000 | 16:05 |
| 07 Subject Test 2 | 0.096 | 16:06 |
| 08 Room Air       | 0.000 | 16:07 |
| 09 Reported AC    | 0.096 | 16:06 |

Difference OK  
No RFI Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = CMS MODE  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 40  
Expiration Date = 02/05/2021  
County = 08                            Oper No. = 666666

20 minute waiting period ascertained? Y  
Clean Mouthpiece used and disposed? Y  
Clean Mouthpiece used and disposed? Y

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks: *CMS MODE*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-003071  
Location = TOXL                    8164.14.00 09/16  
06/27/2019                            16:28

| Test                | AC     | Time  |
|---------------------|--------|-------|
| 01 Room Air         | 0.000  | 16:29 |
| 02 Std. Gas         | 0.079  | 16:29 |
| 03 Room Air         | 0.000  | 16:30 |
| 04 Std. Gas         | 0.079  | 16:30 |
| 05 Room Air         | 0.000  | 16:31 |
| 06 Std. Gas         | 0.078  | 16:31 |
| 07 Room Air         | 0.000  | 16:32 |
| 08 Std. Gas         | 0.079  | 16:32 |
| 09 Room Air         | 0.000  | 16:32 |
| 10 Std. Gas         | 0.079  | 16:33 |
| 11 Room Air         | 0.000  | 16:33 |
| 12 Std. Gas         | 0.079  | 16:34 |
| 13 Room Air         | 0.000  | 16:34 |
| 14 Std. Gas         | 0.080  | 16:35 |
| 15 Room Air         | 0.000  | 16:35 |
| 16 Std. Gas         | 0.080  | 16:36 |
| 17 Room Air         | 0.000  | 16:36 |
| 18 Std. Gas         | 0.078  | 16:36 |
| 19 Room Air         | 0.000  | 16:37 |
| 20 Std. Gas         | 0.079  | 16:37 |
| 21 Room Air         | 0.000  | 16:38 |
| 22 Std. Gas         | 0.079  | 16:38 |
| 23 Room Air         | 0.000  | 16:39 |
| 24 Std. Gas         | 0.079  | 16:39 |
| 25 Room Air         | 0.000  | 16:40 |
| 26 Std. Gas         | 0.079  | 16:40 |
| 27 Room Air         | 0.000  | 16:41 |
| 28 Std. Gas         | 0.079  | 16:41 |
| 29 Room Air         | 0.000  | 16:42 |
| 30 Std. Gas         | 0.079  | 16:42 |
| 31 Room Air         | 0.000  | 16:42 |
| 32 Std. Gas         | 0.080  | 16:43 |
| 33 Room Air         | 0.000  | 16:43 |
| 34 Std. Gas         | 0.079  | 16:44 |
| 35 Room Air         | 0.000  | 16:44 |
| 36 Std. Gas         | 0.079  | 16:45 |
| 37 Room Air         | 0.000  | 16:45 |
| 38 Std. Gas         | 0.079  | 16:45 |
| 39 Room Air         | 0.000  | 16:46 |
| 40 Std. Gas         | 0.079  | 16:46 |
| 41 Room Air         | 0.000  | 16:47 |
| 42 Cal Check Stats: |        |       |
| 43 Average          | 0.0791 |       |
| 44 Std Dev          | 0.0005 |       |
| 45 Rel Std Dev      | 0.65%  |       |



Operator Signature  
CHARLES EDER

Remarks:

STABILITY TEST

Form 106-I8000

0.080 AC

CEE

LOT: 34418080 A~~1~~2

CYL: 40

EXP: 2.5.21