



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <b>KEVIN ELDRIUGE</b>		
Location <b>DICKINSON LEC</b>	Intoxilyzer® Serial Number <b>80-005947</b>	
Gas Lot Number <b>34418080A2</b>	Gas Cylinder Number <b>11</b>	Gas Expiration Date <b>02/05/2021</b>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. **0.079** AC  
 B. **0.078** AC  
 C. **0.078** AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:  
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).  
 B. ACA Test Record.

Chemical Test Operator Signature	Date <b>8/31/19</b>
Reviewed By (Crime Laboratory Use Only)	Date <b>9/3/19</b>


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005947  
Location = DKPD      8164.14.00 09/16  
08/31/2019      14:50

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:51
02 Std. Gas	0.079	14:51
03 Room Air	0.000	14:52
04 Std. Gas	0.078	14:52
05 Room Air	0.000	14:53
06 Std. Gas	0.078	14:53
07 Room Air	0.000	14:54

Lot No = 34418080A2  
Cyl No = 11  
Exp Date = 02/05/2021  
County = 45      Oper No. = 130935

  
Operator Signature  
KEVIN ELDRIDGE

Remarks:

Form 106-I8000