

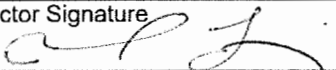


**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (06/2018)

Serial Number 80-005952	Instrument Location BMDC
Reason for Install/Repair	
<input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u>Check up- Testing    Print Test, ACA Test, RFI Test</u>	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): None Needed
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature  4834	Date 11/26/2019
-----------------------------------------------------------------------------------------------------------------------	--------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By 	Certified Date 12/3/2019
--------------------------------------------------------------------------------------------------------------	-----------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-005952  
Location = BMDC                8164.14.00 09/16  
11/26/2019                      14:55

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?         1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            34917080A3  
Standard Cyl #?            36  
Standard Expiration?       02/05/2020  
Oper No?                    130315

Flow Cal. Date:            06/21/2013  
Slope                      734  
Intercept                  -865305

IR Calibration Date:        06/21/2013  
                                    3um                      9um

-----  
0th Coef(\*100):            -21281                      -23057  
1st Coef(\*100):            292132                      137092  
2nd Coef(\*100):            2334                        1231  
H2O adj(mg/l\*10k):        497                         482

\*\*\*\*\* Printer Test End \*\*\*\*\*

 4834

Operator Signature  
JARED LEMIEUX

Remarks:

Form 106-I8000

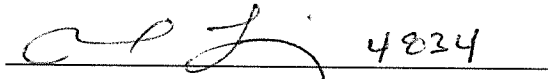
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005952  
Location = BMDC      8164.14.00 09/16  
11/26/2019      14:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:56
02 Std. Gas	0.082	14:57
03 Room Air	0.000	14:57
04 Std. Gas	0.082	14:57
05 Room Air	0.000	14:58
06 Std. Gas	0.082	14:58
07 Room Air	0.000	14:59

Lot No = 34917080A3  
Cyl No = 36  
Exp Date = 02/05/2020  
County = 08      Oper No. = 130315

  
Operator Signature  
JARED LEMIEUX

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

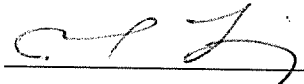
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005952  
Location = BMDC      8164.14.00 09/16  
11/26/2019      15:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	15:01
02 Room Air	0.000	15:02

\*Invalid Test  
Inhibited - RFI

Lot No = 34917080A3  
Cyl No = 36  
Exp Date = 02/05/2020  
County = 08      Oper No. = 130315

 4834  
Operator Signature  
JARED LEMIEUX

Remarks:

Form 106-I8000