



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (06/2018)

Serial Number <b>80-005944</b>	Instrument Location <b>FINL</b>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input checked="" type="checkbox"/> Other (Specify) <u>Returned from ND Crime Lab for regular maintenance</u>	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): \_\_\_\_\_
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test. **RECEIVED ALL**

Field Inspector Signature 	Date <b>1/28/2020</b>
-------------------------------	--------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By 	Certified Date <b>1.31.2020</b>
---------------------------	------------------------------------

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58507

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 3000      SN 80-005944  
Location = FINL      8:64.14.00 09/16  
01/28/2020      18:48

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890- |  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$%^&\*()\_+?

Current Instrument Setup

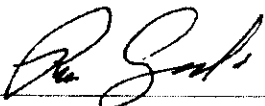
Data Entry Mode:      Enabled  
Start Test Sequence:      DABACABA  
Display Prelim Rslt?      Yes  
Display Third Digit?      Yes  
Inhib Printer(Y/N)?      No  
Display Volume?      No  
Disable On Memfull?      Yes  
# of Print Copies?      1  
Select Std (D/W/I)?      Dry  
Standard Value?      0.080  
Standard Lot #?      34917080A3  
Standard Cyl #?      1  
Standard Expiration?      02/05/2020  
Oper No?      130363

Flow Cal. Date:      06/20/2013  
Slope      677  
Intercept      -702620

IR Calibration Date:      06/09/2017  
  3um                          9um

0th Coef(*100):	-16158	-17815
1st Coef(*100):	266780	136962
2nd Coef(*100):	1117	1089
H2O adj(mg/l*10k):	317	364

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
Operator Signature  
RICK CZAPIEWSKI

Remarks:

Form 106-18000

Intoxilyzer Test Record and Checklist  
NDDAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005944  
Location = FINL      8164.14.00 09/16  
01/28/2020      18:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:52
02 Std. Gas	0.083	18:52
03 Room Air	0.000	18:53
04 Std. Gas	0.082	18:53
05 Room Air	0.000	18:54
06 Std. Gas	0.082	18:54
07 Room Air	0.000	18:54

Lot No = 34917080A3  
Cyl No = 1  
Exp Date = 02/05/2020  
County = 46      Oper No. = 130363



Operator Signature  
RICK CZAPIEWSKI

Remarks:

Form 106-18000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005944  
Location = FINL      8164.14.00 09/16  
01/28/2020      18:56

Test	AC	Time
01 Room Air	RFI*	18:58
02 Room Air	0.000	18:58

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, TEST T  
Sub DOB = 11/22/1999  
Sub Sex = Male      Weight = 222  
Test = DUI      Cit = NA  
Dr. Lic. = ND/NA  
Lot No = 34917080A3  
Cyl No = 1  
Expiration Date = 02/05/2020  
County = 46      Oper No. = 130363



Operator Signature  
RICK CZAPIEWSKI

Remarks:

Form 106-18000