

ETHANOL GAS STANDARD CYLINDER REPORT NORTH DAKOTA OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM SFN 59282 (08/2018)

Chemical Test Operator Name (Print)	Idan Baker	
Location Berleh PD 116 (entral AU		Intoxilyzer® Serial Number
Gas Lot Number 241/90/80/4.2	Gas Cylinder Number	Gas Expiration Date

Check When Done:

- 1, Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).

Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).

- A. 0.042 AC
- B. 0.082 AC
- C. 0.084 AC

23. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.

- 4, File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature	1745	Date 02/05/20
Reviewed By (Crime Laboratory Use Only)	Charles Ech	Date 02/06/20

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-006491Location = BEPD8164.14.00 09/1602/05/202015:20

			DRY CAL CHECK	
Т	est		AC	Time
01	Room	Air	0.000	15:21
02	Std.	Gas	0.082	15:22
03	Room	Air	0.000	15:22
04	Std.	Gas	0.082	15:23
05	Room	Air	0.000	15:23
06	Std.	Gas	0.082	15:23
07	Room	Air	0.000	15:24

Lot No = 24119080A1 Cyl No = 20 Exp Date = 11/05/2021 County = 29 Oper No. = 131958

R 1745

Operator Signature ADAM BAKER

Remarks:

Form 106-18000