



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) CHRISTOPHER SECREST		
Location Renville Sheriff's office 205 MAIN ST E MOHALL, ND 58226	Intoxilyzer® Serial Number 80-006505	
Gas Lot Number 24119080A1	Gas Cylinder Number 34	Gas Expiration Date 11/05/2021

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. **0.082** AC
 B. **0.082** AC
 C. **0.082** AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature 	Date 02/10/2020
Reviewed By (Crime Laboratory Use Only) 	Date 2/11/2020

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006505
Location = MOHL 8164.14.00 09/16
02/07/2020 17:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:36
02 Std. Gas	0.082	17:36
03 Room Air	0.000	17:37
04 Std. Gas	0.082	17:37
05 Room Air	0.000	17:38
06 Std. Gas	0.082	17:38
07 Room Air	0.000	17:38

Lot No = 24119080A1
Cyl No = 34
Exp Date = 11/05/2021
County = 38 Oper No. = 130839



Operator Signature
CHRISTOPHER SECREST

Remarks:

Form 106-I8000

I, the undersigned Sheriff in and for ~~Beaville~~ ^{Beaville} County, North Dakota, do certify that the foregoing is a true copy of an original document remaining on file and record in my office.

WITNESS my hand and seal of said Court this
7th day of FEBRUARY, 2020.

 , Sheriff