

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004945

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 451 psi Regulator: 450 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.015 AC
Sim. Ser #: MP3064 Lot #: 201805c Exp. Date: 5.30.20
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP3069 Lot #: 201902D Exp. Date: 2.20.21

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: ICS 5 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 34418080A2 Cyl #: 5 Exp. Date: 2.5.21

Test 1	<u>0.081</u>	Test 4	<u>0.081</u>	Test 7	<u>0.081</u>
Test 2	<u>0.081</u>	Test 5	<u>0.081</u>	Test 8	<u>0.081</u>
Test 3	<u>0.081</u>	Test 6	<u>0.081</u>	Test 9	<u>0.080</u>
Average	<u>0.081</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ed
Breath Analyst Signature

4/7/2020
Date

NA

NA

Reviewed by _____

Date _____

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/07/2020 11:59

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:00
02 Std. Sol.	0.013	12:01
03 Room Air	0.000	12:01
04 Std. Sol.	0.014	12:02
05 Room Air	0.000	12:02
06 Std. Sol.	0.013	12:03
07 Room Air	0.000	12:04

08 Sim Temp = 34.0°C

Simul Ser No = MP3064
Std Sol No = 201805C
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
 0.015 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/07/2020 11:54

Test	AC	Time
01 Room Air	0.000	11:55
02 *Subject Test	INT*	11:55
03 Room Air	0.000	11:56

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 5
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/07/2020 11:57

Test	AC	Time
01 Diagnostic	OK	11:58
02 Room Air	0.000	11:58
03 *Subject Test	RFI*	11:58
04 Room Air	0.000	11:59

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 5
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/07/2020 12:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:11
02 Std. Gas	0.081	12:11
03 Room Air	0.000	12:12
04 Std. Gas	0.081	12:12
05 Room Air	0.000	12:13
06 Std. Gas	0.081	12:13
07 Room Air	0.000	12:14

Lot No = 34418080A2
Cyl No = 5
Exp Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/07/2020 12:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:14
02 Std. Gas	0.081	12:15
03 Room Air	0.000	12:15
04 Std. Gas	0.081	12:16
05 Room Air	0.000	12:16
06 Std. Gas	0.081	12:16
07 Room Air	0.000	12:17

Lot No = 34418080A2
Cyl No = 5
Exp Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004945
Location = TOXL 8164.14.00 09/16
04/07/2020 12:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:18
02 Std. Gas	0.081	12:18
03 Room Air	0.000	12:19
04 Std. Gas	0.081	12:19
05 Room Air	0.000	12:19
06 Std. Gas	0.080	12:20
07 Room Air	0.000	12:20

Lot No = 34418080A2
Cyl No = 5
Exp Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000