

INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT NORTH DAKOTA OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM SFN 59281 (06/2018)

Serial Number	Instrument Location		
80-005947	STARK LEC/DKPD		
Reason for Install/Repair			
Install After Receiving From Crime Laboratory			
Kother (Specify) Repair (equest due to tone not working			
Check When Done:	\cup		
1. Surge Protector Installed/Property Grounded.			
2. Telephone Line Connected to Intoxilyzer® 8000.			
3. Breath Tube Heated.			
4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).			
5. Scan/Enter Gas Cylinder Information (Level 1, Function S).			
6. Run Tests:			
A. Print Test (Level1, Function P).			
B. ACA Test (Level 1, Function C).			
C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).			
7. Repair and/or Maintenance Performed (if any): N/A (TODE WOLKED Dropelly)			
8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.			
9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.			
10. Send the Following to the Crime Laboratory: Completed Into 104-G), Print Test, ACA Test, and RFI Test.	xilyzer® 8000 Installation and Repair Checkout (SFN59281, Form		
Field Inspector Signature	Date 4/10/2020		
Crime Laboratory Use Only			
This installation has been reviewed and the instrument is approved to be the date the Field Inspector performed the installation. This approved to Silver and the installation of the second se	used for the analysis of breath to determine alcohol concentration from		

the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By	Charles El	Certified Date 4/16/20

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-005947 Location = DKPD 8164.14.00 09/16 04/10/2020 01:32 abcdefghijklmnopqrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +? abcdefghijklmnopgrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +? Current Instrument Setup Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? Standard Cyl #? 34418080A2 11 Standard Expiration? 02/05/2021 Oper No? 130935 Flow Cal. Date: 06/21/2013 Slope 693 -735360 Intercept IR Calibration Date: 04/06/2015 3um 9um _____ _____ 0th Coef(*100): -28096 -26982 1st Coef(*100): 280034 2nd Coef(*100): 3530 136042 1567 H2O adj(mg/l*10k): 510 437

Operator \$⁄ignature KEVIN ÆLDRIDGE

Remarks:

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-005947 Location = DKPD 8164.14.00 09/16 01:32 04/10/2020

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.080 0.000 0.080 0.000 0.079	01:33 01:33 01:34 01:34 01:35 01:35 01:35

Lot No = 34418080A2Cyl No = 11Exp Date = 02/05/2021County = 45Oper No. = 130935

Operator Signature

KEVIN ELDRIDGE

Remarks:

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 Alcohol Analyzer CMI, Inc. Intoxilyzer North Dakota Model 8000 SN 80-005947 8164.14.00 09/16 Location = DKPD 01:36 04/10/2020 AC Time Test 01:38 01 Room Air 0.000 01:38 RFI* 02 *Subject Test 01:39 03 Room Air 0.000 *Invalid Test Inhibited - RFI Sub Name = TEST, TEST NA Sub DOB = 01/01/1999Sub Sex = Male Weight = 210 Cit = 0000000000Test = OTHDr. Lic. = ND/NALot No = 34418080A2Cyl No = 11Expiration Date = 02/05/2021Oper No. = 130935County = 45

Operator ∕\$ígnature KEVIN ÉLDRIDGE

1/12/01/1

Remarks:

Form 106-18000