Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

Intoxilvzer[®] 8000 Serial Number: 80-004204

Location: TOXL

- Α. General Setup and Checks:
 - Diagnostic Tests Pass and Instrument in Ready Mode 1.
 - Breath tube heated 2.
 - Date, time and location code (Level 2,E). Re-set if necessary. 3.
 - Print test (Level 1,P). Sign and attach test record. 4.
 - Tank monitor (Level 3, D, G). Display and Regulator within 50 psi. 5.
 - Display: 38 9 psi Regulator: 375 psi
- Β. Tests (Sign and attach test records):

 \mathbf{X} Configure simulator for the following tests (Level 1,S).

- Wet Calibration Check 1.
 - X Low AC (Level 1,C): Known Value ≤ 0.03 AC: O_0/S AC a. Sim. Ser #: MP3069 Lot #: 201805C Exp. Date: 5.30.20
 - X High AC (Level 1,C): Known Value ≥ 0.25 AC: O.300 AC b. Sim. Ser #: MR 3069 Lot #: 201902 D Exp. Date: 2.20.21

 $C_{\mathcal{H}}^{\mathcal{H}}$ Configure dry gas standard for the following tests (Level 1,S).

- AC + 0.05% Acetone 2 Sim. Ser #: DR3451 Lot #: ICS 5 Exp. Date: NA Display reads "Interferent Detect"
- XRFI Check (CMS Mode) XDisplay reads "RFI Detect" 3.
- CEE X Dry Calibration Check (Level 1, C): Known Value 0.080 AC 4. Gas Cyl Lot #: $34418080A_2$ Cyl #: 5 Exp. Date: $2 \cdot 5 \cdot 26 \cdot 21$ Test 1 0,080 Test 4 0,080 Test 7 0,080

	10014 0.000	
Test 2 0.080	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 0.080	Test 6 0.08/	Test 9 0.081
Average 0.080	2	

Remarks/Maintenance: ANNUAL INSPECTION C.

Instrument is acceptable to be used in the field.

Breath Analyst Signature

Reviewed by

02/18 Issuing Authority - CEE <u>7.15.20</u> Date 11 A

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-004204 Location = TOXL 8164.14.00 09/16 04/15/2020 10:43 abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +? abcdefghijklmnopgrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +? Current Instrument Setup Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 34418080A2 Standard Cyl #? 5 Standard Expiration? 02/05/2021 Oper No? 666666 Flow Cal. Date: 04/15/2020 Slope 657 Intercept -673032 IR Calibration Date: 04/15/2020 3um 9um Oth Coef(*100):-310941st Coef(*100):2653312nd Coef(*100):1555 -28730 134630 1069 H2O adj(mg/l*10k): 516483

Operator Signature CHARLES EDER

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-004204Location = TOXL8164.14.00 09/1604/15/202010:49

	WET CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Sol. 03 Room Air 04 Std. Sol. 05 Room Air 06 Std. Sol. 07 Room Air	0.000 0.013 0.000 0.012 0.000 0.013 0.000	10:50 10:51 10:51 10:52 10:52 10:53 10:54

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = MP3064 Std Sol No = 201805C County = 08

Oper No. = 666666

M C

Operator Signature CHARLES EDER

Low AC 0.015AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-004204Location = TOXL8164.14.00 09/1604/15/202010:55

	WET CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Sol. 03 Room Air 04 Std. Sol. 05 Room Air 06 Std. Sol. 07 Room Air	0.000 0.293 0.000 0.295 0.000 0.294 0.000	10:55 10:56 10:57 10:57 10:58 10:59 10:59

08 Sim Temp = 34.0°C

inca

Operator Signature CHARLES EDER

HIGH

0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-004204 Location = TOXL 8164.14.00 09/16 04/15/2020 10:44 Test AC Time 01 Room Air 0.000 10:45 02 *Subject Test INT* 10:45 03 Room Air 0.000 10:46 *Invalid Test Interferent Detected Sub Name = DISCOVER, THE SPIRIT Sub DOB = 02/01/1992Sub Sex = Female Weight = 150Test = DUI Cit = INTERFERENT Dr. Lic. = ND/DIS921456Lot No = 34418080A2Cyl No = 5Expiration Date = 02/05/2021

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

County = 08

J Check 0.05% ACETONE lven

LOA +

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-004204 Location = TOXL 8164.14.00 09/16 04/15/2020 10:47 Test AC Time 01 Diagnostic OK 10:47 02 Room Air 0.000 10:48 03 *Subject Test RFI* 10:48 04 Room Air 0.000 10:49 *Invalid Test Inhibited - RFI Sub Name = DISCOVER, THE SPIRIT Sub DOB = 02/01/1992Sub Sex = Female Weight = 150Test = DUICit = RFI CHECK Dr. Lic. = ND/DIS921456Lot No = 34418080A2Cyl No = 5Expiration Date = 02/05/2021County = 08Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

Remarks:

CHECK

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-004204 Location = TOXL 8164.14.00 09/16 04/15/2020 11:01

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.080 0.000 0.080 0.000 0.080 0.080	11:01 11:02 11:02 11:03 11:03 11:03 11:04

Lot No = 34418080A2Cyl No = 5Exp Date = 02/05/2021County = 08

Oper No. = 666666

Operator Signature CHARLES EDER MLIBRATION CHECK

0.080 AC Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-004204 Location = TOXL 8164.14.00 09/16 04/15/2020 11:04

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.080 0.000 0.080 0.000 0.081 0.000	11:05 11:05 11:06 11:06 11:07 11:07 11:08

Lot No = 34418080A2Cyl No = 5Exp Date = 02/05/2021County = 08

Oper No. = 666666

Operator Signature

HARLES EDER ALIBRATION CHECK

0.080 AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-004204Location = TOXL8164.14.00 09/1604/15/202011:08

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.080 0.000 0.080 0.000 0.081 0.000	11:08 11:09 11:09 11:10 11:10 11:10 11:11

Lot No = 34418080A2 Cyl No = 5 Exp Date = 02/05/2021 County = 08 Oper

Oper No. = 666666

UU

Operator Signature CHARLES EDER

ALIBRATION CAECK

0.080 AC_

Form 106-18000