

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004201

Location: ToxL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 395 psi Regulator: 400 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq$  0.03 AC: 0.015 AC  
Sim. Ser #: MP3064 Lot #: 201805C Exp. Date: 5.30.20
- b.  High AC (Level 1,C): Known Value  $\geq$  0.25 AC: 0.300 AC  
Sim. Ser #: MP3069 Lot #: 201902D Exp. Date: 2.20.21

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR3451 Lot #: IC55 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 34418080A2 Cyl #: 5 Exp. Date: 2.5.21

Test 1	<u>0.080</u>	Test 4	<u>0.081</u>	Test 7	<u>0.081</u>
Test 2	<u>0.081</u>	Test 5	<u>0.080</u>	Test 8	<u>0.081</u>
Test 3	<u>0.080</u>	Test 6	<u>0.081</u>	Test 9	<u>0.081</u>
Average	<u>0.081</u>				

C. Remarks/Maintenance: REPLACED SIMULATOR RETURN O-RING,  
REPLACED BREATH HOSE O-RING, ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ehr  
Breath Analyst Signature

4.15.20  
Date

NA  
Reviewed by

NA  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004201  
Location = TOXL                    8164.14.00 09/16  
04/15/2020                            12:15

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup

Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            34418080A2  
Standard Cyl #?            5  
Standard Expiration?        02/05/2021  
Oper No?                    666666

Flow Cal. Date:            04/25/2018  
Slope                        580  
Intercept                    -164747

IR Calibration Date:        04/26/2018  
                                  3um                            9um

-----  
0th Coef(\*100):            -26833                        -28328  
1st Coef(\*100):            272526                        132392  
2nd Coef(\*100):            3966                            1397  
H2O adj(mg/l\*10k):        561                            514

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004201  
Location = TOXL      8164.14.00 09/16  
04/15/2020      12:21

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:21
02 Std. Sol.	0.013	12:22
03 Room Air	0.000	12:23
04 Std. Sol.	0.013	12:23
05 Room Air	0.000	12:24
06 Std. Sol.	0.013	12:25
07 Room Air	0.000	12:25

08 Sim Temp = 34.0°C

Simul Ser No = MP3064

Std Sol No = 201805C

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

Low AC  
0.015 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004201  
Location = TOXL      8164.14.00 09/16  
04/15/2020      12:26

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:27
02 Std. Sol.	0.297	12:27
03 Room Air	0.000	12:28
04 Std. Sol.	0.300	12:29
05 Room Air	0.000	12:29
06 Std. Sol.	0.301	12:30
07 Room Air	0.000	12:30


08 Sim Temp = 34.0°C

Simul Ser No = MP3069

Std Sol No = 201902D

County = 08

Oper No. = 666666

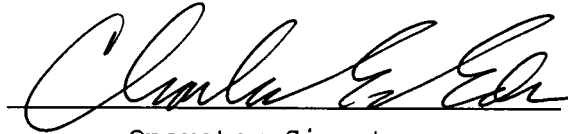

Operator Signature

CHARLES EDER

Remarks:

HIGH AC  
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004201  
Location = TOXL                    8164.14.00 09/16  
04/15/2020                            12:16

Test	AC	Time
01 Room Air	0.000	12:16
02 *Subject Test	INT*	12:17
03 Room Air	0.000	12:18

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = INTERFERENT  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 5  
Expiration Date = 02/05/2021  
County = 08                            Oper No. = 666666

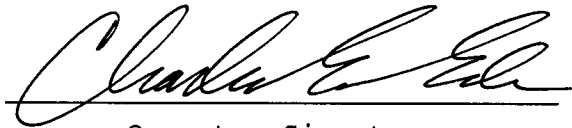

Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10 AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-004201  
Location = TOXL                8164.14.00 09/16  
04/15/2020                        12:19

Test	AC	Time
01 Diagnostic	OK	12:19
02 Room Air	0.000	12:20
03 *Subject Test	RFI*	12:20
04 Room Air	0.000	12:20

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS921456  
Lot No = 34418080A2  
Cyl No = 5  
Expiration Date = 02/05/2021  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

*RFI CHECK*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004201  
Location = TOXL      8164.14.00 09/16  
04/15/2020      12:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:32
02 Std. Gas	0.080	12:33
03 Room Air	0.000	12:33
04 Std. Gas	0.081	12:34
05 Room Air	0.000	12:34
06 Std. Gas	0.080	12:35
07 Room Air	0.000	12:35

Lot No = 34418080A2  
Cyl No = 5  
Exp Date = 02/05/2021  
County = 08      Oper No. = 566666

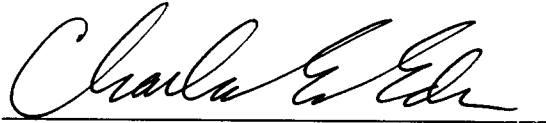

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004201  
Location = TOXL      8164.14.00 09/16  
04/15/2020      12:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:36
02 Std. Gas	0.081	12:36
03 Room Air	0.000	12:37
04 Std. Gas	0.080	12:37
05 Room Air	0.000	12:38
06 Std. Gas	0.081	12:38
07 Room Air	0.000	12:38

Lot No = 34418080A2  
Cyl No = 5  
Exp Date = 02/05/2021  
County = 08      Oper No. = 666666

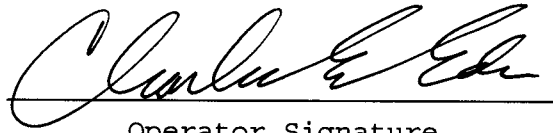

Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION CHECK  
0.080 AC

Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-004201  
Location = TOXL      8164.14.00 09/16  
04/15/2020      12:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:39
02 Std. Gas	0.081	12:40
03 Room Air	0.000	12:40
04 Std. Gas	0.081	12:41
05 Room Air	0.000	12:41
06 Std. Gas	0.081	12:41
07 Room Air	0.000	12:42

Lot No = 34418080A2  
Cyl No = 5  
Exp Date = 02/05/2021  
County = 08      Oper No. = 666666

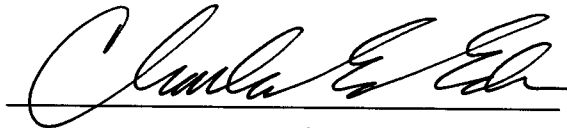

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000