

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004200

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 385 psi Regulator: 400 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value \leq 0.03 AC: 0.015 AC
Sim. Ser #: MP3064 Lot #: 201805C Exp. Date: 5.30.20
- b. High AC (Level 1,C): Known Value \geq 0.25 AC: 0.300 AC
Sim. Ser #: MP3069 Lot #: 201902D Exp. Date: 2.20.21


Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: ICS 5 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 34418080A2 Cyl #: 5 Exp. Date: 2.5.21

Test 1	<u>0.081</u>	Test 4	<u>0.081</u>	Test 7	<u>0.082</u>
Test 2	<u>0.082</u>	Test 5	<u>0.081</u>	Test 8	<u>0.081</u>
Test 3	<u>0.081</u>	Test 6	<u>0.081</u>	Test 9	<u>0.081</u>
Average	<u>0.081</u>				

C. Remarks/Maintenance: REPLACED SIMULATOR RETURN O-RING,
ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

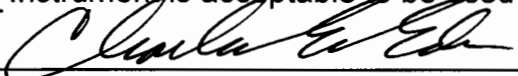
Breath Analyst Signature

NA

4.15.20
Date

NA

Reviewed by _____

Date _____

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = TOXL 8164.14.00 09/16
04/15/2020 13:03

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%\$^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMN0PQRSTUVWXYZ!@#%\$^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 34418080A2
Standard Cyl #? 5
Standard Expiration? 02/05/2021
Oper No? 666666

Flow Cal. Date: 06/25/2019
Slope 614
Intercept -551153

IR Calibration Date: 06/25/2019
 3um 9um

0th Coef(*100): -18792 -22164
1st Coef(*100): 267001 140777
2nd Coef(*100): 1903 1083
H2O adj(mg/l*10k): 403 445

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = TOXL 8164.14.00 09/16
04/15/2020 13:16

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:17
02 Std. Sol.	0.015	13:18
03 Room Air	0.000	13:18
04 Std. Sol.	0.015	13:19
05 Room Air	0.000	13:20
06 Std. Sol.	0.014	13:20
07 Room Air	0.000	13:21


08 Sim Temp = 34.0°C

Simul Ser No = MP3064

Std Sol No = 201805C

County = 08

Oper No. = 666666

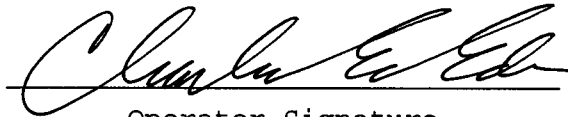

Operator Signature

CHARLES EDER

Remarks:

Low AC
0.015 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = TOXL 8164.14.00 09/16
04/15/2020 13:22

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:23
02 Std. Sol.	0.303	13:24
03 Room Air	0.000	13:24
04 Std. Sol.	0.304	13:25
05 Room Air	0.000	13:26
06 Std. Sol.	0.303	13:26
07 Room Air	0.000	13:27

08 Sim Temp = 34.0°C

Simul Ser No = MP3069
Std Sol No = 201902D
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AK
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = TOXL 8164.14.00 09/16
04/15/2020 13:03

Test	AC	Time
01 Room Air	0.000	13:04
02 *Subject Test	INT*	13:05
03 Room Air	0.000	13:06

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 5
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

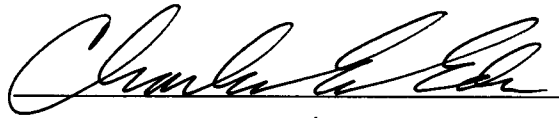

Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = TOXL 8164.14.00 09/16
04/15/2020 13:01

Test	AC	Time
01 Diagnostic	OK	13:02
02 Room Air	0.000	13:02
03 *Subject Test	RFI*	13:02
04 Room Air	0.000	13:03

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 5
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = TOXL 8164.14.00 09/16
04/15/2020 13:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:29
02 Std. Gas	0.081	13:30
03 Room Air	0.000	13:30
04 Std. Gas	0.082	13:30
05 Room Air	0.000	13:31
06 Std. Gas	0.081	13:31
07 Room Air	0.000	13:32

Lot No = 34418080A2
Cyl No = 5
Exp Date = 02/05/2021
County = 08 Oper No. = 666666

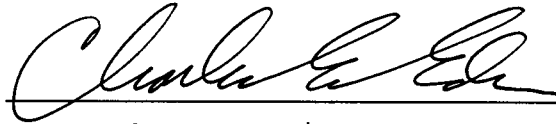

Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = TOXL 8164.14.00 09/16
04/15/2020 13:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:33
02 Std. Gas	0.081	13:33
03 Room Air	0.000	13:34
04 Std. Gas	0.081	13:34
05 Room Air	0.000	13:34
06 Std. Gas	0.081	13:35
07 Room Air	0.000	13:35

Lot No = 34418080A2
Cyl No = 5
Exp Date = 02/05/2021
County = 08 Oper No. = 666666

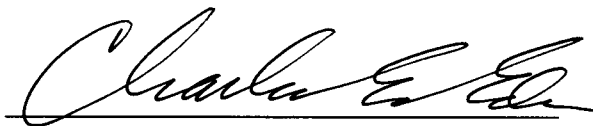

Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004200
Location = TOXL 8164.14.00 09/16
04/15/2020 13:35

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:36
02 Std. Gas	0.082	13:36
03 Room Air	0.000	13:37
04 Std. Gas	0.081	13:37
05 Room Air	0.000	13:38
06 Std. Gas	0.081	13:38
07 Room Air	0.000	13:39

Lot No = 34418080A2
Cyl No = 5
Exp Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-18000