

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004199

Location: Tox2

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 398 psi Regulator: 400 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value \leq 0.03 AC: 0.015 AC
Sim. Ser #: MP3064 Lot #: 201805C Exp. Date: 5.30.20
- b. High AC (Level 1,C): Known Value \geq 0.25 AC: 0.300 AC
Sim. Ser #: MP3069 Lot #: 201902D Exp. Date: 0.300

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: JCS5 Exp. Date: NA

Display reads "Interferent Detect"

3. RFI Check (CMS Mode) Display reads "RFI Detect"

4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 34418080A2 Cyl #: 5 Exp. Date: 2.5.21

Test 1	<u>0.079</u>	Test 4	<u>0.080</u>	Test 7	<u>0.080</u>
Test 2	<u>0.080</u>	Test 5	<u>0.079</u>	Test 8	<u>0.079</u>
Test 3	<u>0.079</u>	Test 6	<u>0.080</u>	Test 9	<u>0.080</u>
Average	<u>0.080</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ed
Breath Analyst Signature

4-15-20
Date

NA
Reviewed by

NA
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.14.00 09/16
04/15/2020 14:12

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:13
02 Std. Sol.	0.015	14:14
03 Room Air	0.000	14:14
04 Std. Sol.	0.015	14:15
05 Room Air	0.000	14:16
06 Std. Sol.	0.016	14:16
07 Room Air	0.000	14:17


08 Sim Temp = 34.0°C

Simul Ser No = MP3064

Std Sol No = 201805C

County = 08

Oper No. = 666666

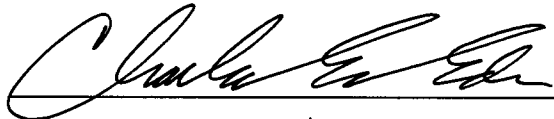

Operator Signature

CHARLES EDER

Remarks:

Low AC
0.015 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.14.00 09/16
04/15/2020 14:20

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:20
02 Std. Sol.	0.304	14:21
03 Room Air	0.000	14:22
04 Std. Sol.	0.306	14:23
05 Room Air	0.000	14:23
06 Std. Sol.	0.306	14:24
07 Room Air	0.000	14:24

08 Sim Temp = 34.0°C

Simul Ser No = MP3069

Std Sol No = 201902D

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.14.00 09/16
04/15/2020 14:10

Test	AC	Time
01 Room Air	0.000	14:10
02 *Subject Test	INT*	14:11
03 Room Air	0.000	14:11

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 5
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

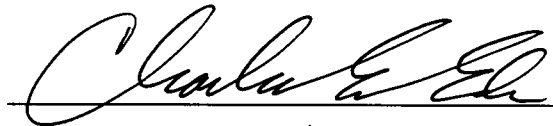

Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.14.00 09/16
04/15/2020 14:08

Test	AC	Time
01 Diagnostic	OK	14:08
02 Room Air	0.000	14:09
03 *Subject Test	RFI*	14:09
04 Room Air	0.000	14:10

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 5
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

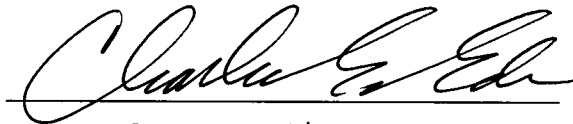

Operator Signature
CHARLES EDER

Remarks:

R RFI CHECK
CEE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.14.00 09/16
04/15/2020 14:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:26
02 Std. Gas	0.079	14:27
03 Room Air	0.000	14:27
04 Std. Gas	0.080	14:28
05 Room Air	0.000	14:28
06 Std. Gas	0.079	14:29
07 Room Air	0.000	14:29

Lot No = 34418080A2
Cyl No = 5
Exp Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.14.00 09/16
04/15/2020 14:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:30
02 Std. Gas	0.080	14:31
03 Room Air	0.000	14:31
04 Std. Gas	0.079	14:31
05 Room Air	0.000	14:32
06 Std. Gas	0.080	14:32
07 Room Air	0.000	14:33

Lot No = 34418080A2
Cyl No = 5
Exp Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-18000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004199
Location = TOXL 8164.14.00 09/16
04/15/2020 14:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:34
02 Std. Gas	0.080	14:34
03 Room Air	0.000	14:35
04 Std. Gas	0.079	14:35
05 Room Air	0.000	14:36
06 Std. Gas	0.080	14:36
07 Room Air	0.000	14:36

Lot No = 34418080A2
Cyl No = 5
Exp Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000