

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-003061

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 378 psi Regulator: 375 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.015 AC
Sim. Ser #: MP3064 Lot #: 201805C Exp. Date: 5.30.20
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP3069 Lot #: 201902D Exp. Date: 2.20.21

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: ICSS Exp. Date: NA

Display reads "Interferent Detect"

3. RFI Check (CMS Mode) Display reads "RFI Detect"

4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC

Gas Cyl Lot #: 34418080A2 Cyl #: 5 Exp. Date: 2.5.21

Test 1 0.080 Test 4 0.080 Test 7 0.080

Test 2 0.080 Test 5 0.080 Test 8 0.079

Test 3 0.080 Test 6 0.080 Test 9 0.080

Average 0.080

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ede
Breath Analyst Signature

4.20.20
Date

NA
Reviewed by

NA
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/20/2020 10:54

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:54
02 Std. Sol.	0.014	10:55
03 Room Air	0.000	10:56
04 Std. Sol.	0.013	10:56
05 Room Air	0.000	10:57
06 Std. Sol.	0.013	10:58
07 Room Air	0.000	10:58

08 Sim Temp = 34.0°C

Simul Ser No = MP3064
Std Sol No = 201805C
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

LOW AC
0.015 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/20/2020 11:07

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:08
02 Std. Sol.	0.300	11:09
03 Room Air	0.000	11:09
04 Std. Sol.	0.302	11:10
05 Room Air	0.000	11:10
06 Std. Sol.	0.301	11:11
07 Room Air	0.000	11:12


08 Sim Temp = 34.0°C

Simul Ser No = MP3069

Std Sol No = 201902D

County = 08

Oper No. = 666666

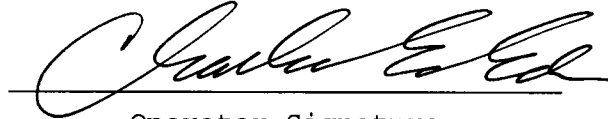

Operator Signature

CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/20/2020 10:44

Test	AC	Time
01 Room Air	0.000	10:44
02 *Subject Test	INT*	10:45
03 Room Air	0.000	10:46

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 5
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

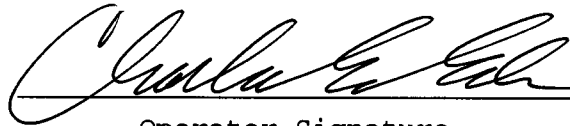

Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/20/2020 10:46

Test	AC	Time
01 Diagnostic	OK	10:47
02 Room Air	0.000	10:47
03 *Subject Test	RFI*	10:47
04 Room Air	0.000	10:48

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 5
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/20/2020 11:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:14
02 Std. Gas	0.080	11:15
03 Room Air	0.000	11:15
04 Std. Gas	0.080	11:16
05 Room Air	0.000	11:16
06 Std. Gas	0.080	11:16
07 Room Air	0.000	11:17

Lot No = 34418080A2
Cyl No = 5
Exp Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/20/2020 11:17

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:18
02 Std. Gas	0.080	11:18
03 Room Air	0.000	11:19
04 Std. Gas	0.080	11:19
05 Room Air	0.000	11:20
06 Std. Gas	0.080	11:20
07 Room Air	0.000	11:20

Lot No = 34418080A2
Cyl No = 5
Exp Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003061
Location = TOXL 8164.14.00 09/16
04/20/2020 11:21

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:21
02 Std. Gas	0.080	11:22
03 Room Air	0.000	11:22
04 Std. Gas	0.079	11:23
05 Room Air	0.000	11:23
06 Std. Gas	0.080	11:23
07 Room Air	0.000	11:24

Lot No = 34418080A2
Cyl No = 5
Exp Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000