Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

Intoxilyzer[®] 8000 Serial Number: 80-003066

Location: / UXL

- Α. General Setup and Checks:
 - Diagnostic Tests Pass and Instrument in Ready Mode 1.
 - Breath tube heated 2.
 - Date, time and location code (Level 2,E). Re-set if necessary. 3.
 - X Print test (Level 1,P). Sign and attach test record. 4.
 - Tank monitor (Level 3,D,G). Display and Regulator within 50 psi. 5.
 - Display: 346 psi Regulator: 375 psi a.
- Tests (Sign and attach test records): Β.

Configure simulator for the following tests (Level 1.S).

- Wet Calibration Check 1.
 - X Low AC (Level 1,C): Known Value ≤ 0.03 AC: <u>0.015</u> AC Sim. Ser # $\frac{20069}{2009}$ Lot #: <u>201805</u> Exp. Date: <u>5.30.20</u> a.
 - \square High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC b. Sim. Ser #: MP3069 Lot #: 201902 Exp. Date: 2. RO. 21

 \boxtimes Configure dry gas standard for the following tests (Level 1,S).

- Interferent Check (Level 1,B): Known Value: 0./OAC + 0.05% Acetone 2. Sim. Ser #: DR3451 Lot #: ICS 5 Exp. Date: NA Display reads "Interferent Detect"
- 3.
- 4 Gas Cyl Lot #: 24119080 A1 Cyl #: 9 Exp. Date: 11. 5.21

 Test 1
 0.080
 Test 4
 0.080
 Test 7
 0.080

 Test 2
 0.080
 Test 5
 0.080
 Test 8
 0.080

 Test 3
 0.079
 Test 6
 0.080
 Test 9
 0.080
- Average 0.080
- Remarks/Maintenance: ANNUAL INSPECTION С.

Hunstrument is acceptable to be used in the field.

Breath Analyst Signature

Reviewed by

 $\frac{7 \cdot 27 \cdot 20}{\text{Date}}$

Date

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-003066 Location = TOXL 8164.14.00 09/16 04/29/2020 16:06 abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +? abcdefghijklmnopqrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +? Current Instrument Setup Data Entry Mode: Enabled Start Test Sequence: Display Prelim Rslt? DABACABA Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 24119080A1 Standard Cyl #? 9 Standard Expiration? 11/05/2021 Oper No? 666666 Flow Cal. Date: 07/29/2010 Slope 806 Intercept -895033 IR Calibration Date: 07/29/2010 3um 9um 0th Coef(*100): -14075 -5808
 1st Coef (*100):
 272113

 2nd Coef (*100):
 1637
 138204 986 H2O adj(mg/l*10k): 289148

Call El

Operator Signature CHARLES EDER

Remarks:

RINT TES

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-003066Location = TOXL8164.14.00 09/1604/29/202016:08

	WET CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Sol. 03 Room Air 04 Std. Sol. 05 Room Air 06 Std. Sol.	0.000 0.014 0.000 0.014 0.000 0.015	16:09 16:09 16:10 16:11 16:11 16:12
07 Room Air	0.000	16:1

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = MP3064 Std Sol No = 201805C County = 08

Oper No. = 666666

U al

Operator Signature CHARLES EDER

Low Ac 0.015 AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-003066Location = TOXL8164.14.00 09/1604/29/202016:14

	WET CAL CHECK]
Test	AC	Time
01 Room Air	0.000	16:15
02 Std. Sol.	0.304	16:16
03 Room Air	0.000	16:16
04 Std. Sol.	0.305	16:17
05 Room Air	0.000	16:18
06 Std. Sol.	0.303	16:18
07 Room Air	0.000	16:19

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = MP3069 Std Sol No = 201902D County = 08

Oper No. = 666666

un

Operator Signature CHARLES EDER HIGH M

Remarks:

Form 106-I8000

0.300 ,

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-003066 Location = TOXL 8164.14.00 09/16 04/29/2020 16:24 Time Test AC 01 Room Air 0.000 16:24 02 *Subject Test INT* 16:25 03 Room Air 0.000 16:25 *Invalid Test Interferent Detected Sub Name = DISCOVER, THE SPIRIT Sub DOB = 02/01/1992Sub Sex = Female Weight = 150Test = DUI Cit = INTERFERENT Dr. Lic. = ND/DIS921456Lot No = 24119080A1Cyl No = 9Expiration Date = 11/05/2021County = 08Oper No. = 666666

Operator Signature CHARLES EDER

Interferent Check D. 10AC + D. 05% ACETONE

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-003066 Location = TOXL 8164.14.00 09/16 04/29/2020 16:21 Test AC Time 01 Diagnostic OK 16:22 02 Room Air 0.000 16:23 03 *Subject Test RFI* 16:23 04 Room Air 0.000 16:23 *Invalid Test Inhibited - RFI Sub Name = DISCOVER, THE SPIRIT Sub DOB = 02/01/1992Sub Sex = Female Weight = 150Cit = RFI CHECK Test = DUI Dr. Lic. = ND/DIS921456Lot No = 24119080A1Cyl No = 9Expiration Date = 11/05/2021County = 08Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

FCK

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-003066 Location = TOXL 8164.14.00 09/16 04/29/2020 16:26

	DRY CAL CHECK]
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.080 0.000 0.080 0.000 0.079 0.000	16:26 16:27 16:27 16:27 16:28 16:28 16:28 16:29

Lot No = 24119080A1Cyl No = 9Exp Date = 11/05/2021County = 08

Oper No. = 666666

Operator Signature CHARLES EDER ALIBRAMON CHECK

0.030 Ac

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-003066Location = TOXL8164.14.00 09/1604/29/202016:29

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.080 0.000 0.080 0.000 0.080 0.080 0.000	16:30 16:30 16:31 16:31 16:31 16:32 16:32

Lot No = 24119080A1 Cyl No = 9 Exp Date = 11/05/2021 County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

ALIBRATION CHECK

0-080 AC

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer
North Dakota Model 8000Alcohol Analyzer
SN 80-003066Location = TOXL
04/29/20208164.14.00 09/16
16:33

	DRY CAL CHECK	
Test	AC	Time
01 Room Air	0.000	16:33
02 Std. Gas	0.080	16:34
03 Room Air	0.000	16:34
04 Std. Gas	0.080	16:35
05 Room Air	0.000	16:35
06 Std. Gas	0.080	16:35
07 Room Air	0.000	16:36

Lot No = 24119080A1 Cyl No = 9 Exp Date = 11/05/2021 County = 08

Oper No. = 666666

Operator Signature CHARLES EDER MUBRAMON CHECK

0.080 AC

Form 106-18000