Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

Intox	ilyzer® 8000 Serial Number: 80-004/93 Location: 70×L
Α.	General Setup and Checks: 1.
B.	Tests (Sign and attach test records): Configure simulator for the following tests (Level 1,S). Wet Calibration Check a. ★Low AC (Level 1,C): Known Value ≤ 0.03 AC:
	 Configure dry gas standard for the following tests (Level 1,S). 2.
C.	Remarks/Maintenance: ANNUAL INSPECTION LASSROOM ONLY UNTIL EVALUATED/REPAIRED, AUTORAGE VALUELOW
□ Ins	trument is acceptable to be used in the field. Analyst Signature NA Wed by Date

02/18

Issuing Authority - CEE

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-004193 Location = TOXL 8164.14.00 09/16 05/04/2020 11:29

******* Printer Test ********

abcdefghijklmnopgrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

abcdefghijklmnopqrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled Start Test Sequence: Display Prelim Rslt? DABACABA Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? Standard Cyl #? 05620080A1 47 Standard Expiration? 04/05/2022 Oper No? 666666

Flow Cal. Date: 06/18/2019 Slope 622 Intercept

IR Calibration Date: 06/18/2019

Oth Coef(*100): -14782 1st Coef(*100): 267622 2nd Coef(*100): 2469 -15525 134211 H20 adj (mg/1*10k): 264225

3um

-723749

******* Printer Test End ********

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 05/04/2020 Alcohol Analyzer SN 80-004193 8164.14.00 09/16 11:33

WET CAL CHECK

Test			AC	Time
01	Room	Air	0.000	11:33
02	Std.	Sol.	0.013	11:34
03	Room	Air	0.000	11:35
04	Std.	Sol.	0.014	11:35
05	Room	Air	0.000	11:36
06	Std.	Sol.	0.014	11:37
07	Room	Air	0.000	11:37

08 Sim Temp = 34.0°C

Simul Ser No = MP3064 Std Sol No = 201805C

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

LOW AC

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
05/04/2020

Alcohol Analyzer SN 80-004193 8164.14.00 09/16 13:26

WET CAL CHECK

Test			AC	Time
01	Room	Air	0.000	13:26
02	Std.	Sol.	0.304	13:27
03	Room	Air	0.000	13:28
04	Std.	Sol.	0.305	13:28
05	Room	Air	0.000	13:29
06	Std.	Sol.	0.302	13:30
07	Room	Air	0.000	13:30

08 Sim Temp = 34.0°C

Simul Ser No = MP3069 Std Sol No = 201902D County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

0.300

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-004193 Location = TOXL 8164.14.00 09/16 05/04/2020 13:33

Test	AC	Time
01 Room Air	0.000	13:34
02 *Subject Test	INT*	13:34
03 Room Air	0.000	13:35

^{*}Invalid Test

Interferent Detected

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female

Weight = 150 Test = DUI Cit = INTERFERENT

Dr. Lic. = ND/DIS921456

Lot No = 05620080A1

Cyl No = 47

Expiration Date = 04/05/2022

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Interferent Check 0.10 AC + 0.05 V. ACETONE

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004193
Location = TOXL 8164.14.00 09/16
05/04/2020 13:35

Test	AC	Time
01 Diagnostic	OK	13:36
02 Room Air	0.000	13:36
03 *Subject Test	RFI*	13:37
04 Room Air	0.000	13:37

*Invalid Test Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK

Dr. Lic. = ND/DIS921456

Lot No = 05620080A1

Cyl No = 47

Expiration Date = 04/05/2022

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
05/04/2020

SN 80-004193 8164.14.00 09/16 13:49

DRY CAL CHECK

Test			AC	Time
01	Room	Air	0.000	13:49
02	Std.	Gas	0.078	13:49
03	Room	Air	0.000	13:50
04	Std.	Gas	0.079	13:50
05	Room	Air	0.000	13:51
06	Std.	Gas	0.078	13:51
07	Room	Air	0.000	13:52

Lot No = 05620080A1

Cyl No = 47

Exp Date = 04/05/2022

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

ALIBRATION CHECK

0.080 AC

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 05/04/2020

Alcohol Analyzer SN 80-004193 8164.14.00 09/16 13:52

DRY CAL CHECK

Test		AC	Time
01 Roc	om Air	0.000	13:52
02 Std	l. Gas	0.079	13:53
03 Roc	om Air	0.000	13:53
04 Std	l. Gas	0.079	13:54
05 Roc	om Air	0.000	13:54
06 Std	l. Gas	0.079	13:55
07 Roc	om Air	0.000	13:55

Lot No = 05620080A1

Cyl No = 47

Exp Date = 04/05/2022

County = 08

Oper No. = 666666

CHARLES EDER

CACIBRATION CHECK

0.080 AC

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
05/04/2020

Alcohol Analyzer SN 80-004193 8164.14.00 09/16 13:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:56
02 Std. Gas	0.079	13:56
03 Room Air	0.000	13:57
04 Std. Gas	0.079	13:57
05 Room Air	0.000	13:58
06 Std. Gas	0.079	13:58
07 Room Air	0.000	13:59

Lot No = 05620080A1

Cyl No = 47

Exp Date = 04/05/2022

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

0,080 AC