## INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-00 6-50 3

Location: ToXL

- Α. General Setup and Checks:
  - Diagnostic Tests Pass and Instrument in Ready Mode 1.
  - Breath tube heated 2.
  - Date, time and location code (Level 2,E). Re-set if necessary. 3.
  - Print test (Level 1,P). Sign and attach test record. 4.
  - X Tank monitor (Level 3, D, G). Display and Regulator within 50 psi. 5.
    - Display: 353 psi Regulator: 375 psi а.
- Tests (Sign and attach test records): Β.

 $\triangle$  Configure simulator for the following tests (Level 1,S).

- Wet Calibration Check 1.
  - f Low AC (Level 1,C): Known Value ≤ 0.03 AC:  $Q_{1015}$  AC a. Sim. Ser #: MP 3064 Lot # 20/805C Exp. Date 2020
  - X High AC (Level 1,C): Known Value ≥ 0.25 AC: <u>0.300</u> AC b. Sim. Ser #: 493069 Lot #: 201902 D Exp. Date: 2.20.21

 $\cancel{1}$  Configure dry gas standard for the following tests (Level 1,S).

- ✓ Interferent Check (Level 1,B): Known Value: O./O AC + 0.05% Acetone 2. Sim. Ser #: DR3451 Lot #: ICS 5 Exp. Date: NA Display reads "Interferent Detect"
- 🛛 RFI Check (CMS Mode) 🛛 🗖 Display reads "RFI Detect" 3.
- Dry Calibration Check (Level 1,C): Known Value 0.080 AC 4. Gas Cyl Lot #:05620080A1 Cyl #: 47 Exp. Date: 4.5.22 

   Test 1
   0.082
   Test 4
   0.082
   Test 7
   0.082

   Test 2
   0.082
   Test 5
   0.082
   Test 8
   0.083

   Test 3
   0.082
   Test 6
   0.083
   Test 9
   0.083
- Average 0.082
- С.

Remarks/Maintenance: <u>ANNUAL INSPECTION</u>

Anstrument is acceptable to be used in the field.

Breath Analyst Signature

Reviewed by

02/18 Issuing Authority - CEE

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006503 Location = TOXL 8164.14.00 09/16 05/06/2020 16:15 abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&\*() +? abcdefghijklmnopgrstuvwxyz1234567890-= ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&\*() +? Current Instrument Setup Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 05620080A1 Standard Cyl #? 47 Standard Expiration? 04/05/2022 Oper No? 666666 Flow Cal. Date: 08/20/2015 Slope 663 Intercept -633300 IR Calibration Date: 06/12/2017 3um 9um 0th Coef(\*100):-67411st Coef(\*100):2565832nd Coef(\*100):3028 -15632 132194 1131 H2O adj(mq/l\*10k): 354358 

Operator Signature CHARLES EDER

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-006503Location = TOXL8164.14.00 09/1605/06/202016:17

	WET CAL CHECK	
Test	AC	Time
01 Room Air	0.000	16:18
02 Std. Sol.	0.013	16:19
03 Room Air	0.000	16:19
04 Std. Sol.	0.013	16:20
05 Room Air	0.000	16:20
06 Std. Sol.	0.013	16:21
07 Room Air	0.000	16:22

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$ 

Simul Ser No = MP3064 Std Sol No = 201805C County = 08 Oper No. = 666666

Operator Signature CHARLES EDER

0.015 AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-006503Location = TOXL8164.14.00 09/1605/06/202016:23

			WET CAL CHECK	
Te	est		AC	Time
01	Room .	Air	0.000	16:23
02	Std.	Sol.	0.294	16:24
03	Room .	Air	0.000	16:25
04	Std.	Sol.	0.299	16:26
05	Room .	Air	0.000	16:26
06	Std.	Sol.	0.299	16:27
07	Room .	Air	0.000	16:27

08 Sim Temp = 34.0°C

Simul Ser No = MP3069 Std Sol No = 201902D County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

HGH AC H

0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006503 Location = TOXL 8164.14.00 09/16 05/06/2020 16:29 Test AC Time 01 Room Air 0.000 16:30 02 \*Subject Test INT\* 16:30 03 Room Air 0.000 16:31 \*Invalid Test Interferent Detected Sub Name = DISCOVER, THE SPIRIT Sub DOB = 02/01/1992Sub Sex = Female Weight = 150Test = DUI Cit = INTERFERENT Dr. Lic. = ND/DIS921456Lot No = 05620080A1Cyl No = 47Expiration Date = 04/05/2022County = 08Oper No. = 666666

Operator Signature CHARLES EDER

INTERFERENT ('HECK 0.10 AC + 0.05% ACETONE Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006503 Location = TOXL 8164.14.00 09/16 05/06/2020 16:32 Test AC Time 01 Diagnostic 16:33 OK 02 Room Air 0.000 16:33 03 \*Subject Test RFI\* 16:34 04 Room Air 0.000 16:34 \*Invalid Test Inhibited - RFI Sub Name = DISCOVER, THE SPIRIT Sub DOB = 02/01/1992Sub Sex = Female Weight = 150Test = DUI Cit = RFI CHECK Dr. Lic. = ND/DIS921456Lot No = 05620080A1Cyl No = 47Expiration Date = 04/05/2022County = 08Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

I CHECK

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006503 Location = TOXL 8164.14.00 09/16 05/06/2020 16:35

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.082 0.000 0.082 0.000 0.082 0.000	16:35 16:36 16:36 16:37 16:37 16:37 16:38

Lot No = 05620080A1Cyl No = 47Exp Date = 04/05/2022County = 08

Oper No. = 666666

Operator Signature

CHARLES EDER CHARLES EDER ALIBRATION CHECK

0.080AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer<br/>North Dakota Model 8000Alcohol Analyzer<br/>SN 80-006503Location = TOXL<br/>05/06/20208164.14.00 09/16<br/>16:44

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.082 0.000 0.082 0.000 0.083 0.000	16:45 16:45 16:46 16:46 16:46 16:47 16:47

Lot No = 05620080A1 Cyl No = 47 Exp Date = 04/05/2022County = 08 Oper No

Oper No. = 666666

Operator Signature CHARLES EDER ALIBRATION CHECK

0.080 AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-006503Location = TOXL8164.14.00 09/1605/06/202016:48

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.082 0.000 0.083 0.000 0.083	16:48 16:49 16:50 16:50 16:50 16:51

Lot No = 05620080A1Cyl No = 47 Exp Date = 04/05/2022County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

CALIBRATION CHECK

0.080AC

Form 106-18000